



Midmark M9 and M11 Steam Sterilizers

DESIGNED FOR SIMPLICITY, RELIABILITY AND SAFETY

Midmark sterilizers are designed to be safe, easy to use and dependable. Every sterilizer is inspected and ASME-certified by a third-party-licensed inspector, ensuring the reliability for which Midmark is known. The Midmark M9® and M11® Steam Sterilizers are designed, manufactured and supported in the US, with decades of manufacturing experience built into each one.

MULTIPLE SIZES TO FIT YOUR NEEDS

Choose from two sizes to accommodate your instrument processing workflow. Select the M11 for maximum capacity and efficiency. Use the M9 when you need a faster cycle with high capacity but are tighter on space.

PRE-PROGRAMMED CYCLES

Once you select a pre-programmed cycle, the unit is designed to sterilize automatically without any operator assistance.

SMART DRYING TECHNOLOGY

The sterilizer will automatically stop and alert you to take proper action if the door isn't closed or the water level is too low. Once sterilization is complete, the door will open automatically and quietly to dissipate steam and dry your instruments.

EASY OPERATION AND MAINTENANCE

Simple operation makes it easy for your team to do the right thing every time, minimizing breaks in safety protocols. With easy-access drain and fill points, Midmark sterilizers lead the way in exceptional quality and streamlined maintenance.



- 94% of dental service technicians would recommend a Midmark sterilizer.¹
- Midmark sterilizers are ASME certified.
- Chamber sizes are available up to 11" x 18".

CYCLE DETAILS

Pre-Programmed Cycle	Sterilization Temperature	Hot Cycle Time (Fill, Heat, and Vent; in minutes)		Sterilization Time (in minutes)		Dry Time (in minutes)		Total Hot Cycle Time w/o Dry (in minutes)		Total Hot Cycle Time w/ Dry (in minutes)	
		M11	M9	M11	M9	M11	M9	M11	M9	M11	M9
Unwrapped	270°F (132°C)	19	11	3	3	30	30	22	14	52	44
Pouches	270°F (132°C)	20	12	4	4	30	30	24	16	54	46
Packs/Low Temp	250°F (121°C)	18	10	30	30	30	30	48	40	78	70
Handpieces	270°F (132°C)	26	15	4	4	30	30	30	19	60	49

STERILIZER SPECIFICATIONS

Model/Order No.	M9®	M11®
Overall length with plug	20.38" (51.8 cm)	23.8" (60.5 cm)
Overall width	15.3" (38.9 cm)	17.8" (45.2 cm)
Overall height with printer	16.1" (40.9 cm)	18.1" (50.0 cm)
Minimum countertop area	17.88" (45.4 cm) D x 15.3" (38.9 cm) W	21" (53.3 cm) D x 17.8" (45.2 cm) W
Chamber	9" diameter (22.9 cm), 15" depth (38.1 cm) 3.5 gal usable volume (13.4 liters)	11" diameter (28 cm), 18" depth (45.7 cm) 6.5 gal usable volume (24.6 liters)
Trays	Two large – 7.3" W x 12" L x 0.8" D (18.6 cm x 30.5 cm x 2.2 cm) Two small – 5.6" W x 12" L x 0.8" D (14.3 cm x 30.5 cm x 2.2 cm)	Two large – 9" W x 15" L x 1.1" D (22.9 cm x 38 cm x 2.9 cm) Two small – 6.6" W x 15" L x 1.1" D (16.8 cm x 38 cm x 2.9 cm)
Unit weight	73 lb (33.1 kg)	99 lb (44.9 kg)
Shipping weight	81 lb (36.7 kg)	131 lb (59.4 kg)
Water reservoir capacity	1.1 gal (4.1 liters)	1.4 gal (5.3 liters)
Electrical requirements	See midmark.com/IP for details	See midmark.com/IP for details

ACCESSORIES

M9 and M11 Sterilizers

M9/M11 Printer	9A259001
Cool Hand Tool	9A307001
6 Slot Vertical Pouch Rack	002-2108-00
5 Slot Vertical Pouch Rack	002-2108-01
Sterilizer Datalogger	9A682001

M11 Sterilizer

Vertical Cassette Rack	9A215002
Horizontal Cassette Rack	9A215001