



# Hand/Foot Control Cord Bracket Kit [002-0962-0x]

- Components:**  
 Cord Bracket ..... 1  
 Screws: ..... 2  
 Nuts ..... 2
- Special Tools:**  
 Drill  
 11/64" drill bit

### Bracket installation for rear inlet:


(The illustration shows the standard base. The procedure for models with rotational base is similar).




#### Equipment Alert

The PC board cover **must** be removed. Failure to do so will result in damage to the PC board when mounting holes are drilled (metal shavings).

**Step 1:** Remove screws & partially separate PC board cover.

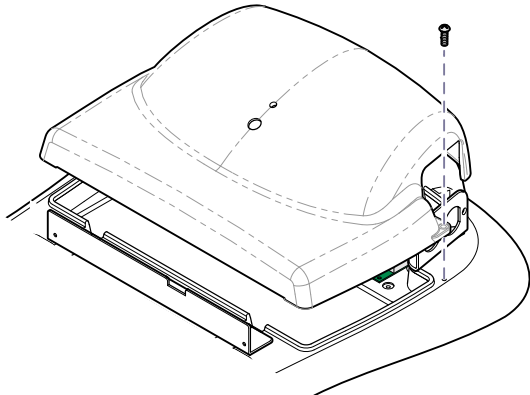

**Step 2:** Disconnect wire harnesses from J9 & J12 on main PC board.  
**Go to Step 3.**

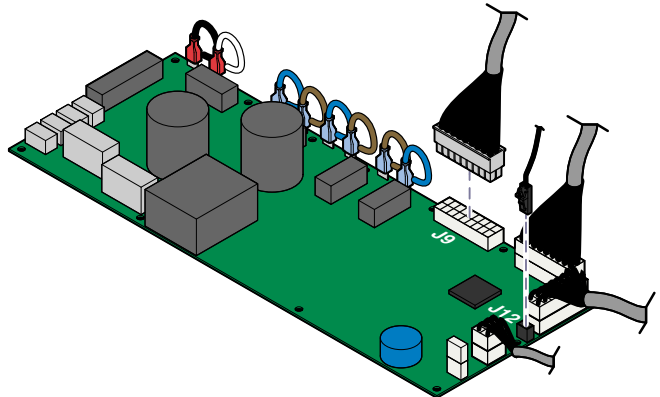


MA720200i

### Bracket installation for front inlet:

(Models w/ standard base only.  
Models w/rotational base, go to Step 3))

**Note:** Newer units have nutserts in shroud spacer to secure control cord bracket.

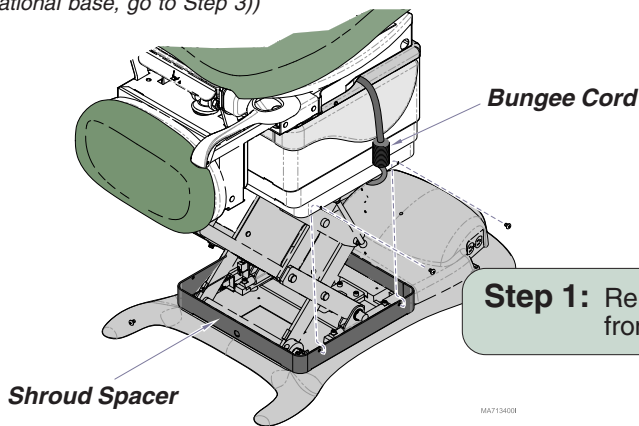

**Step 2:** Lift shrouds, then stretch bungee cord under seat and attach hooks under shrouds on both sides.  
**Go to Step 3.**

**Note:** It may be necessary to run Tilt UP function to route bungee cord under seat section.

**Step 1:** Remove bottom screws from inner shrouds.

MA713402i

**Step 3:** Slide narrow portion of cord connector into bracket as shown.


Locking Ring

**Step 4:** Plug cord connector into desired inlet on table. Tighten locking ring.

**Note:** This step must be done with the control cord positioned in the bracket slot.

**Attention:** Newer tables may have mounting holes. If so, go to Step 6.


**Step 5:** Mark location of mounting holes. Drill mounting holes (11/64").

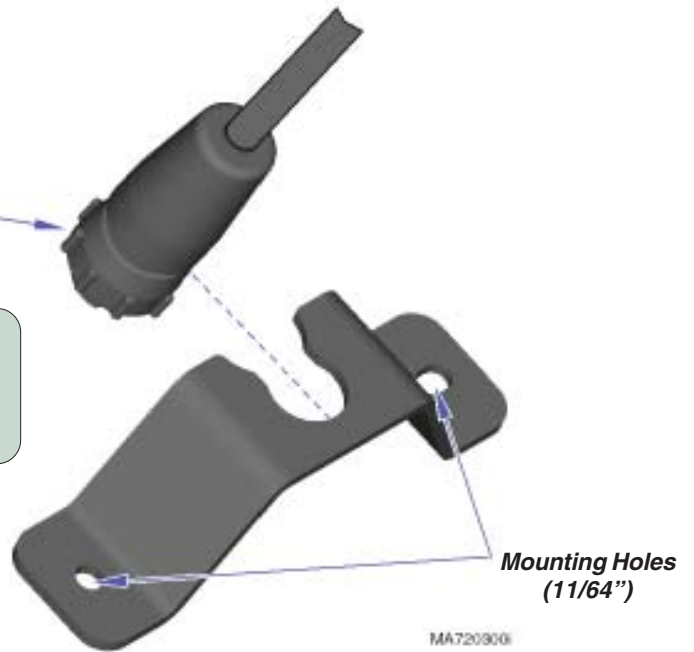

**Bracket installation for rear inlet:**

(The illustration shows the standard base. The procedure for models with rotational base is similar).


**Step 6:** Secure bracket with two screws.

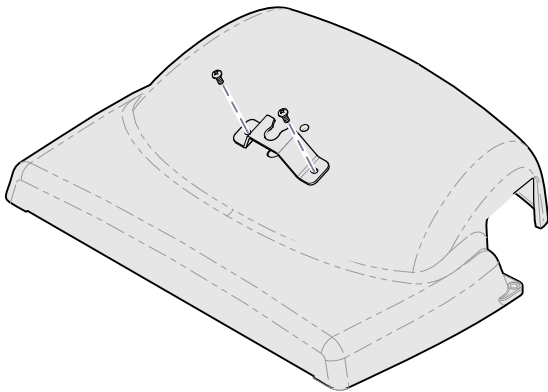

**Bracket installation for front inlet:**

(Models w/ standard base)


**Step 6:** Secure bracket with two screws & two nuts.

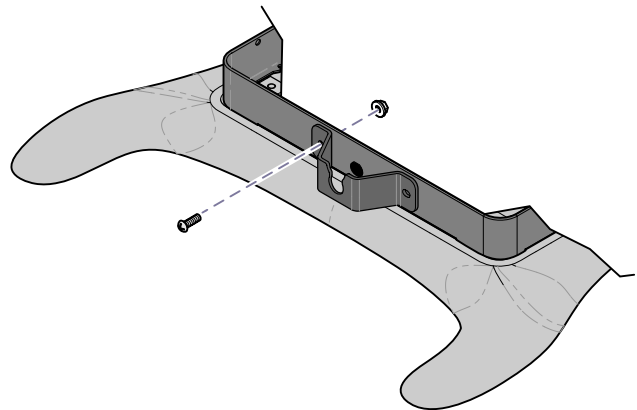

**Bracket installation for front inlet:**

(Models w/ rotational base)

**Step 6:** Secure bracket with two screws.

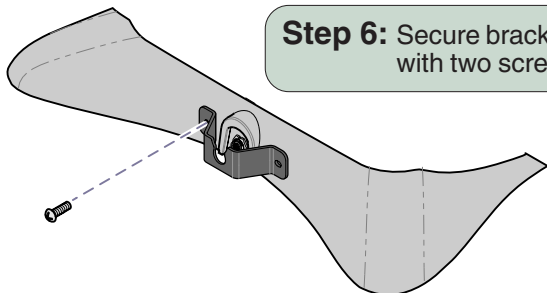

**Step 7:** Replace all covers, shrouds, etc. Connect control cord (see steps 3 & 4).

MA720400i