

To operate the Base / Back / Tilt / Foot functions...

Press & hold the appropriate button.

Example: The Foot Up button is called out in the illustration.



Note

If the chair “beeps” and one or more functions will not operate, move the foot extension to the stowed position - then retry function. Refer to the ‘Troubleshooting’ section in the User Guide for further information regarding the Crash Avoidance System.




QuickChair™ Function (w/foot control only)

This function activates the Base Down, Tilt Down, Foot Down, & Back Up functions at the same time.

To activate the QuickChair™ function...

Press & hold the QuickChair™ button on the foot control.

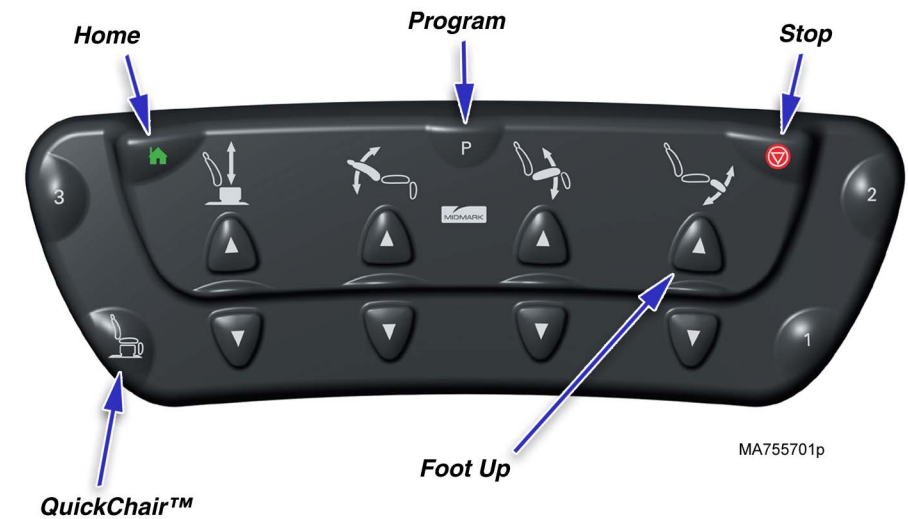

Programming Chair Positions (programmable models only)

This feature allows you to easily recall frequently used chair positions.

To program a chair position...

- A. Move the chair to desired position.
 - B. Press the Program button. [single “beep”]
 - C. Press the desired Position button (1, 2, 3, or 4). [three “beeps”]
- Tip: You must press the Position button within 5 seconds of pressing the Program button.

To recall a programmed position...

Press & hold the desired Position button (1, 2, 3, or 4).

Note

To stop movement of any automatic chair function... Press the Stop button on the hand or foot control.

Home Function

This function automatically lowers the base so that your patient may easily mount or dismount the chair.

To lower the base to 23 in. (58 cm)...

Press, then immediately release the Home button.

To lower the base to 19 in. (48 cm)...

Press & hold the Home button until you hear a single “beep”.

Maintenance

Quick Reference Guide 641 Power Procedures Chair



CAUTION

Headrest must be securely locked in place before performing any exam or procedure.

To adjust the pivot angle of the headrest...


- A. Loosen knob (A).
- B. Pivot headrest to desired position.
- C. Tighten knob (A) to lock in place.

To adjust the height of the headrest...

- A. Loosen knob (B).
- B. Slide headrest out / in to desired position.
- C. Tighten knob (B) to lock in place.

To adjust the side rail (out / in)...

- A. Loosen knob (C) for desired side rail.
- B. Slide side rail out / in to desired position.
- C. Tighten knob (C) to lock in place.

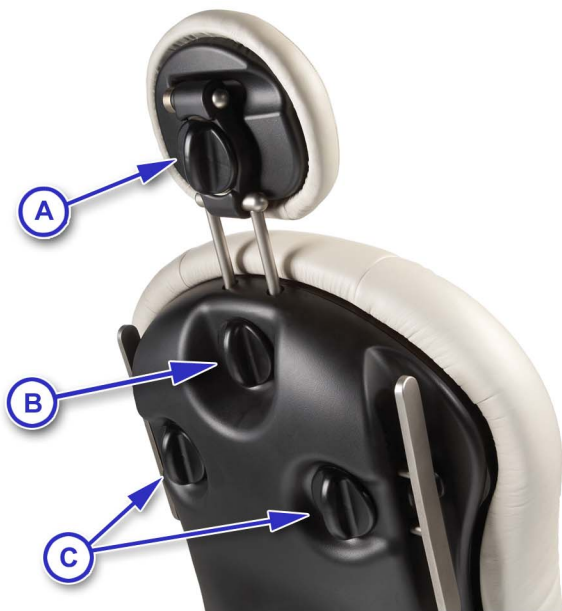

MA775100p

Periodically inspect the following:

- Power cord must be free of visible damage.
- All fasteners must be present and secure.
- All mechanical functions should operate.

Cleaning

Upholstery

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on www.midmark.com in the Technical library under the User Information tab for your product.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner. Periodic applications of common furniture wax will ease cleaning, and maintain the finished luster of the surface. Use of chlorine based disinfectant can cause damage to the hand control.

Calling for Service

If service is required, contact your Midmark dealer.

To contact Midmark directly:
1-800-Midmark (1-800-643-6275)
8:00 am until 5:00 pm Mon-Fri (EST)



Français : Référez-vous à www.midmark.com

Midmark Corporation
60 Vista Drive
Versailles, OH 45380-0286
Phone: 937-526-3662
Fax: 937-526-5542
midmark.com



Because we care.

Model Number:	<input type="text"/>
Serial Number:	<input type="text"/>
Dealer Info:	<input type="text"/>
Purchase Date:	<input type="text"/>
Upholstery Color:	<input type="text"/>

003-1707-00 Rev. D 7/9/15

This guide outlines basic operation of the chair. The enclosed User Guide provides additional instructions on operation and care, as well as regulatory compliance information. If service is required, contact your authorized Midmark dealer.



To extend / retract the foot section..

- A. Squeeze release lever on either side of foot section.
- B. Pivot foot extension up / down until it locks into place.



Release Lever

