




12", 15", 18", 24", Side Storage Unit Installation

Applies to Models:
OSC012, 015, 018, 024

Special Tools:
Level
1/4" deep well socket

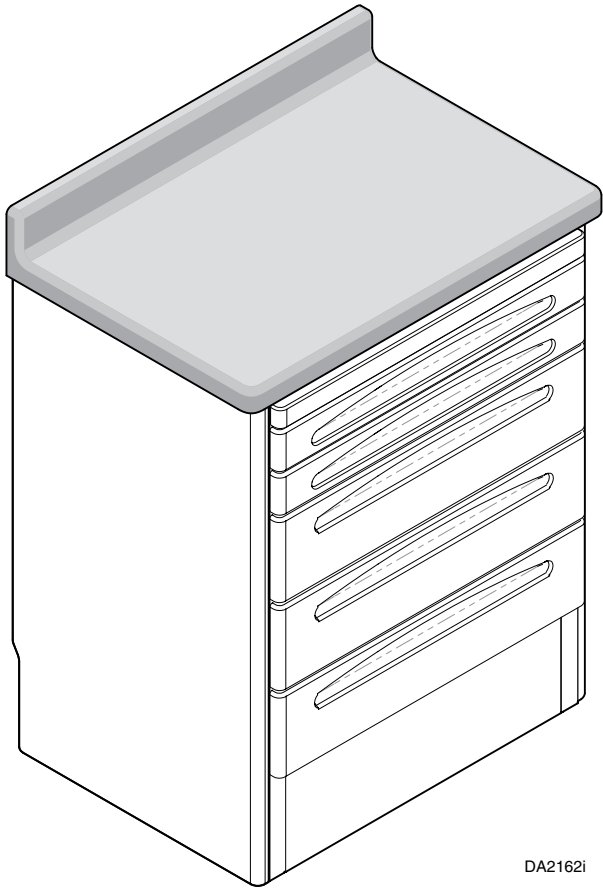

DA2162i

OSC024 (Shown)

- OSC012 (12" Side Storage Unit)
- OSC015 (15" Side Storage Unit)
- OSC018 (18" Side Storage Unit)
- OSC024 (24" Side Storage Unit)

DRAWER REMOVAL *(If Required for Access)*

**2" Metal Frame Drawers
or
Writing Surface Assembly**

Step 1: Lift or push down release levers (one per side) and slide out drawer assembly.


All Molded Drawers

Step 1: Pull drawer outward until it hits stop.

Step 2: Pull sides of drawer outward while lifting to free back of drawer from hooks and remove.

DA2163i

POSITIONING & LEVELING

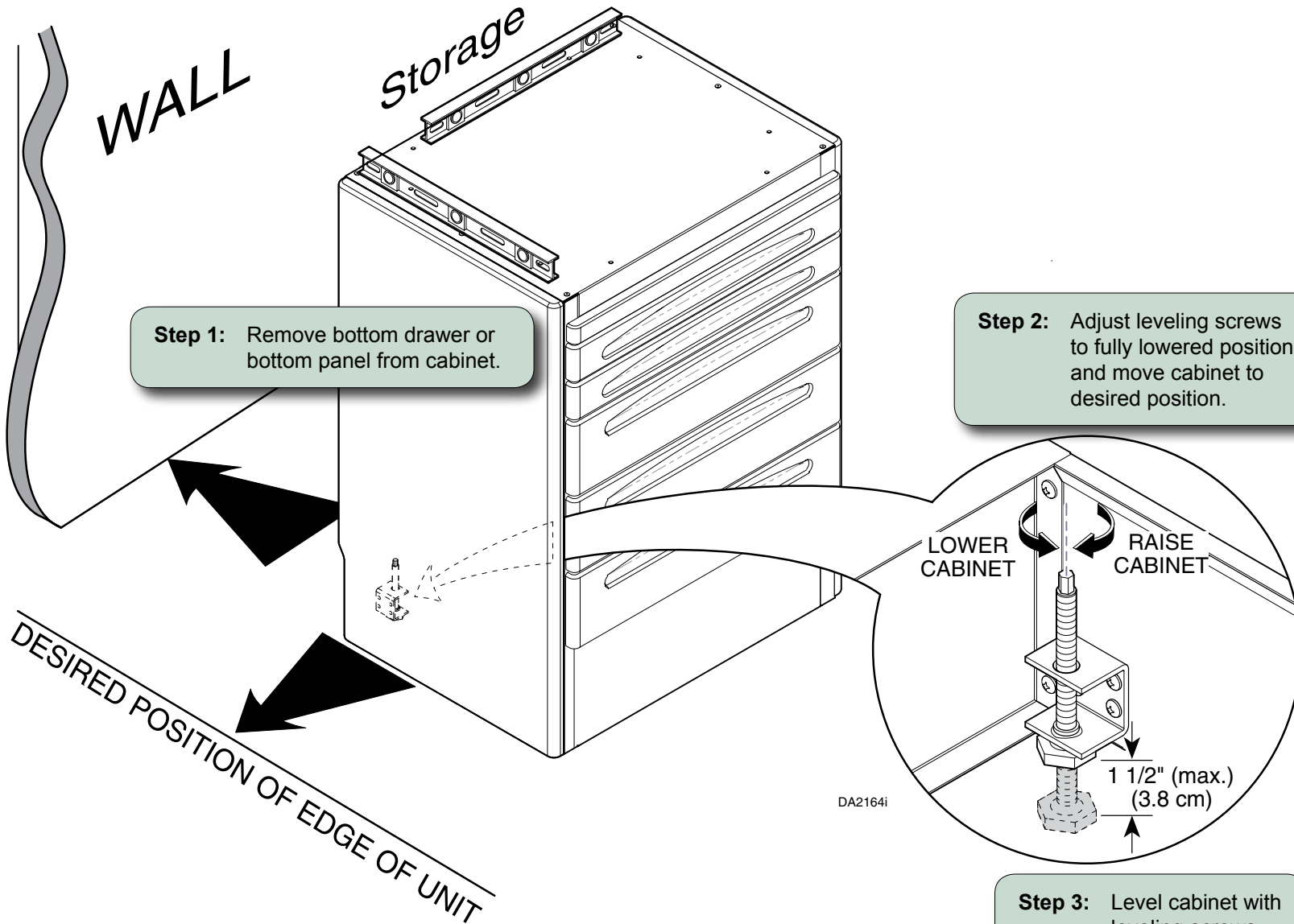


CONNECTING ADDITIONAL UNITS & COUNTER TOP INSTALLATION

Note
If necessary, sand
counter top to form
to wall.

Step 2: Position counter top so it extends 3/8" (0.95 cm) to 1/2" (1.3 cm) over each end of cabinet(s) and secure with provided screws.

Step 1: If necessary, level and connect any additional storage units.

Special hex bolts
(#10-24 x 3/4")

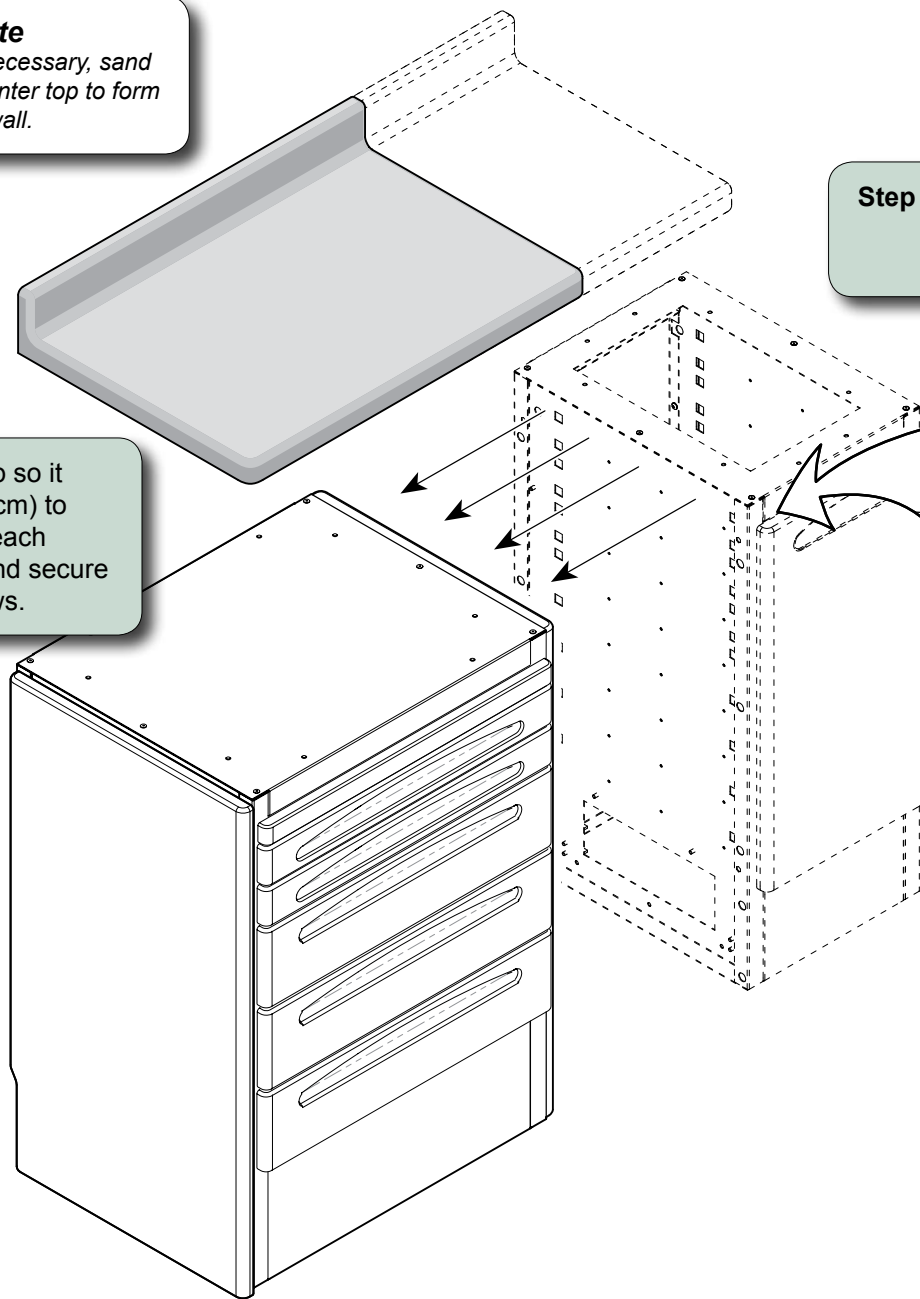

DA2165i

DOOR ADJUSTMENT

