Midmark Dental Light, LR Mounted on an Elevance Chair Installation Guide

Applies to Models:
153901 (-002 & -003, -007 & -008)
153818 (-002 & -003)

WARNING
Equipment is not suitable for use in the presence of a flammable anesthetic mixture.
Clarification: Equipment is suitable for use in the presence of oxygen, air, or nitrous oxide.

Install the Lower Arm Weldment

Step 1: Raise Chair to a comfortable working height of 10" - 15" above floor.

NOTE: This install guide applies to Halogen and LED-style lights.

Halogen

LED

DA3426i

LR Arm Casting

Lower Arm Weldment

Greased Thrust Bearings

LR Arm Bearing Cap

Hex Bolt


WARNING
Disconnect chair from power during installation. Failure to do so may result in personal injury.

Step 2: Position lower arm weldment beneath LR mounting casting with thrust bearings positioned as shown. Place bearing cap onto lower bearing and secure with hex bolt.

DA2092i

Assemble Light Arm Components

Step 3: Insert the stop pin, position stop ring on square flex arm post so tab faces light head, then insert flex arm assembly into the light arm post. Secure with set screw.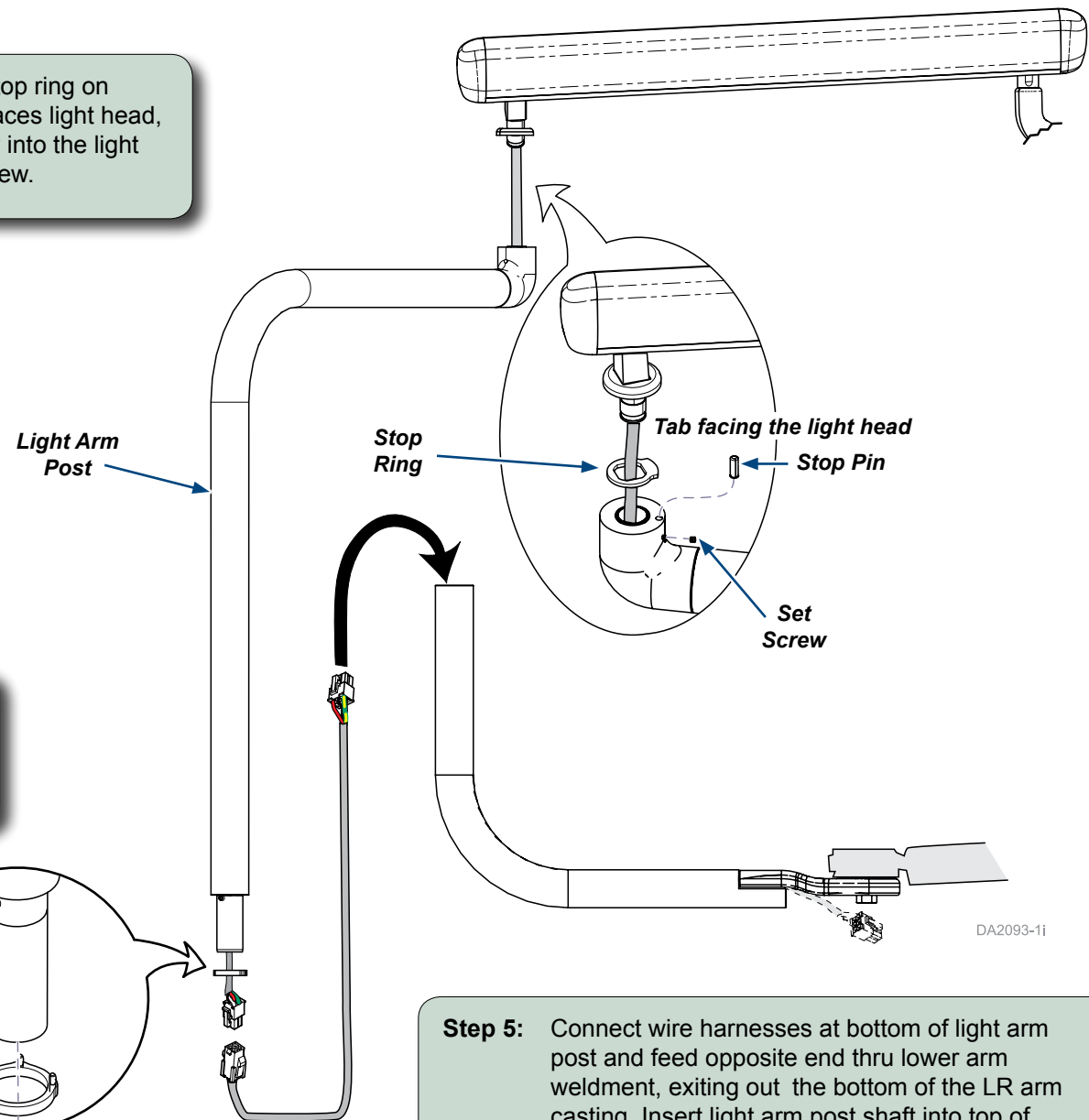


Step 4: Install rotation stop plate (pins up) on lower light arm shaft so that pins are located on the opposite side of stop screw on shaft.

Step 5: Connect wire harnesses at bottom of light arm post and feed opposite end thru lower arm weldment, exiting out the bottom of the LR arm casting. Insert light arm post shaft into top of lower arm.


Electrical Connection

Step 6: With seat upholstery removed from chair, feed arm light harness into chair cavity and connect to chair light harness.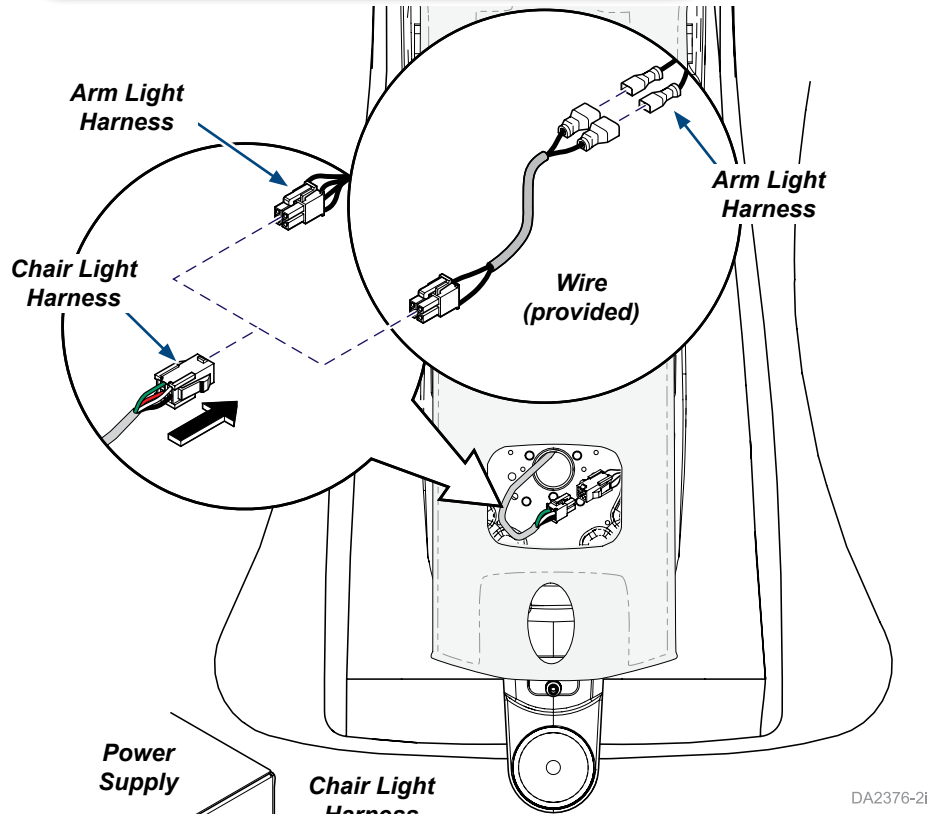

DA2376-2i


Step 7: Replace seat upholstery.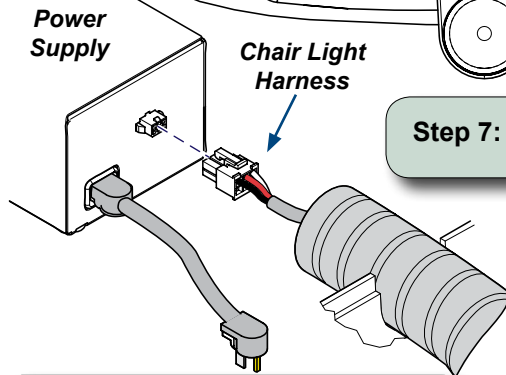

Step 8: Connect chair light harness to power supply.

Arm Leveling

Step 9: Loosen the 4 mounting bolts 1 or 2 turns.

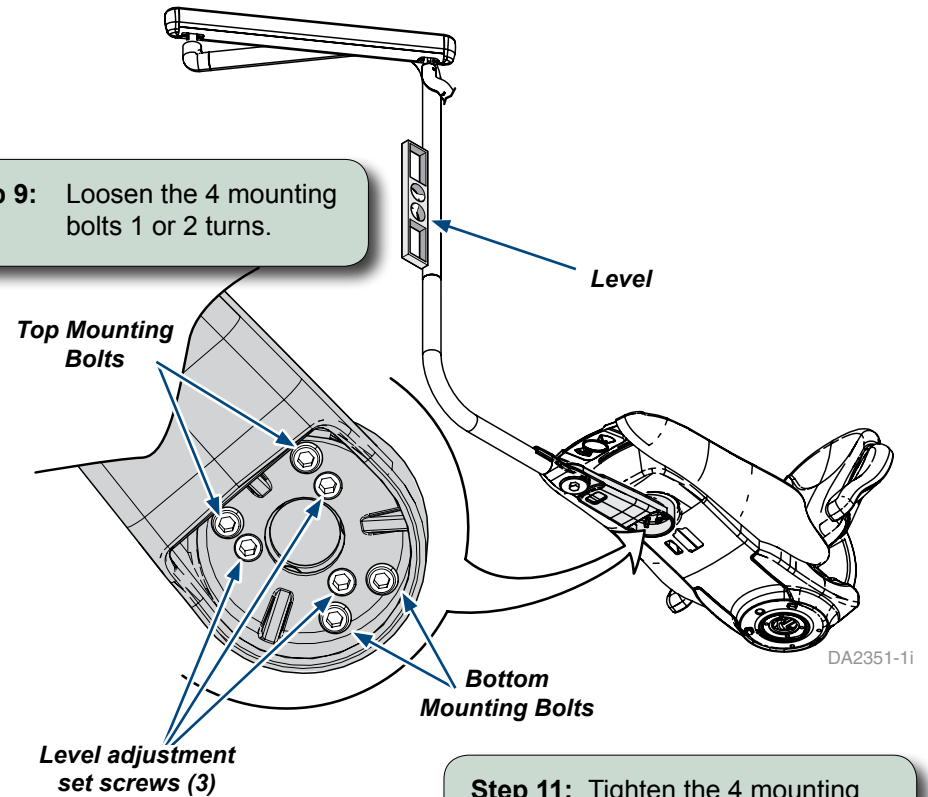DA2351-1i

Step 10: Adjustment set screws to level the Light Arm Assembly.

Step 11: Tighten the 4 mounting bolts 1 or 2 turns.

Operating the Power Supply (Light) Without Air


Equipment Alert

If the installation location does not have a pressurized air supply line to connect to the power supply, it will be necessary to bypass the air pressure switch. Bypassing the air pressure switch is not necessary when a pressurized air supply line is connected.


WARNING

To prevent the possibility of electrical shock, always disconnect facility power to the Power Supply before removing cover. Failure to comply with these instructions could result in severe personal injury or death.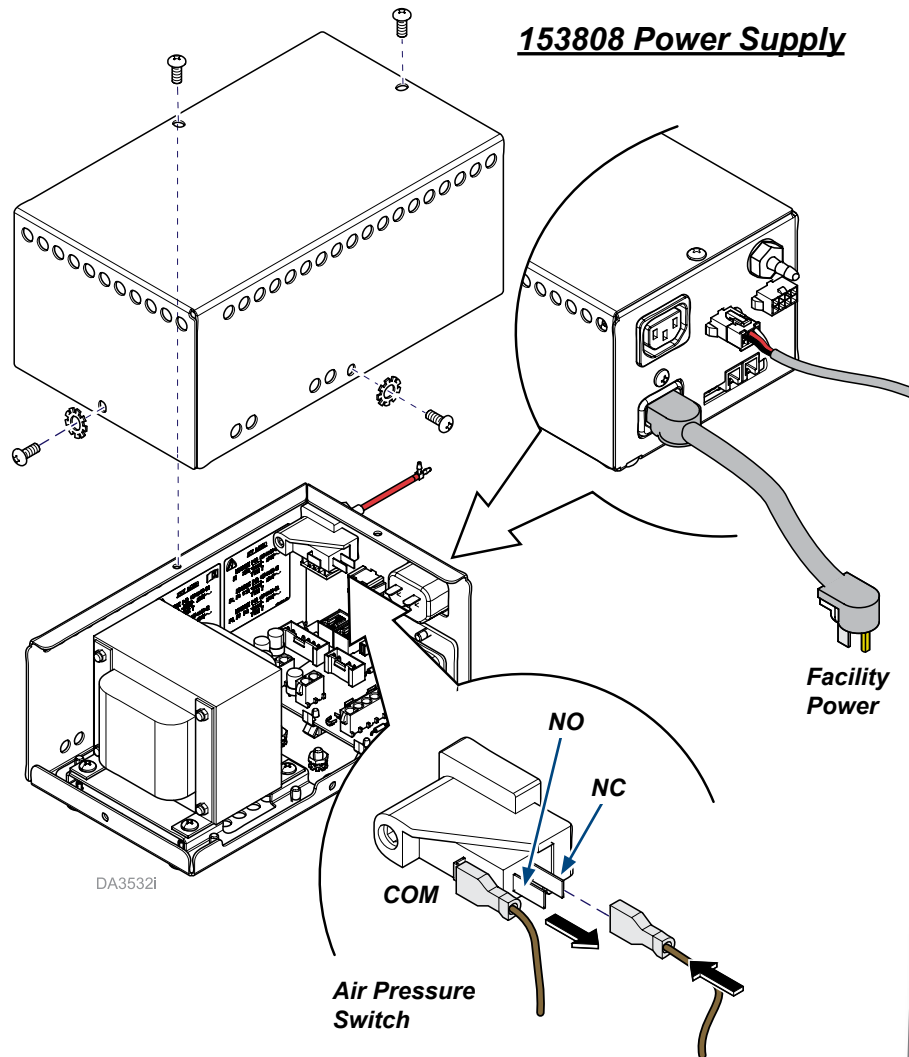

153941 Power Supply

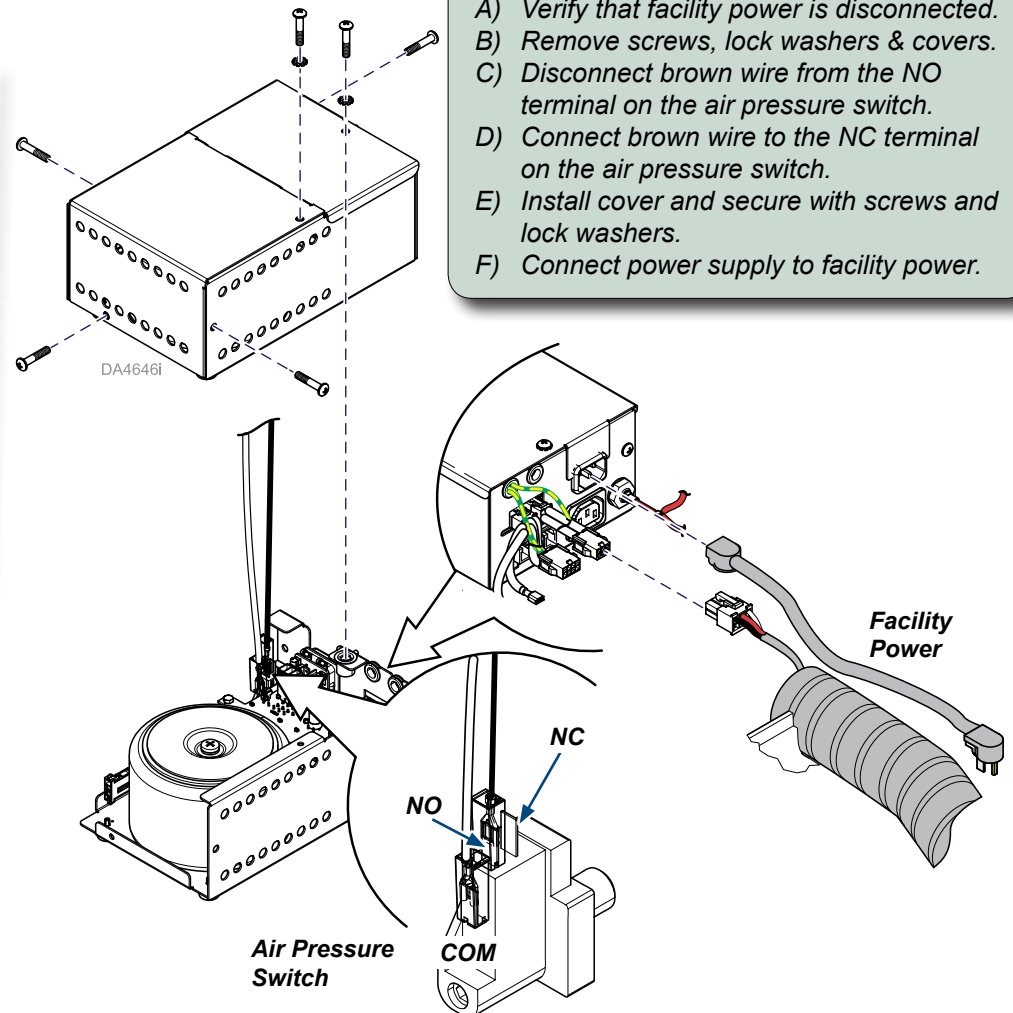To Bypass the Air Pressure Switch...

- Verify that facility power is disconnected.
- Remove screws, lock washers & covers.
- Disconnect brown wire from the NO terminal on the air pressure switch.
- Connect brown wire to the NC terminal on the air pressure switch.
- Install cover and secure with screws and lock washers.
- Connect power supply to facility power.

Tension Adjustments



To adjust tension at Light pivot joint...

- Remove screw cover as shown.
- Loosen set screw.
- Loosen or tighten light pivot adjustment screw.
- Tighten set screw.

To adjust tension at Arm / Yoke pivot joints...

- Remove neck cover as shown (see Note).
- Loosen or tighten appropriate screws to adjust tension.

Note: Some halogen models have a short neck on the lighthouse. It is not necessary to remove the neck cover on these models.

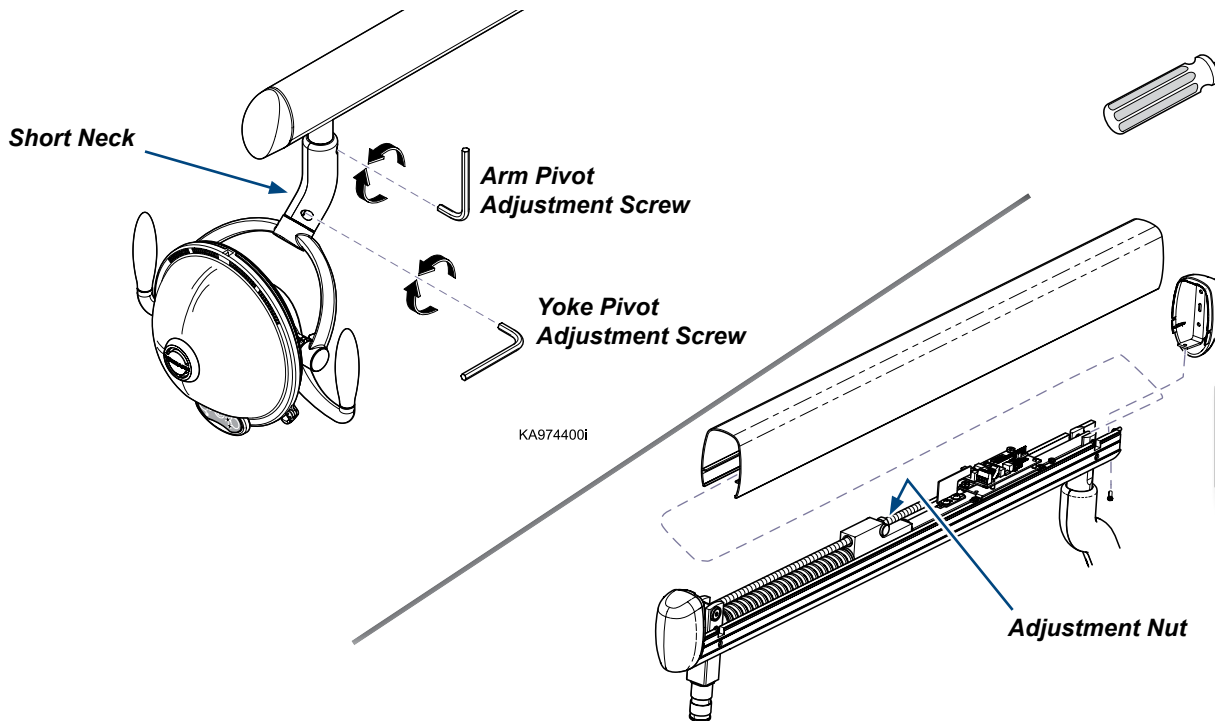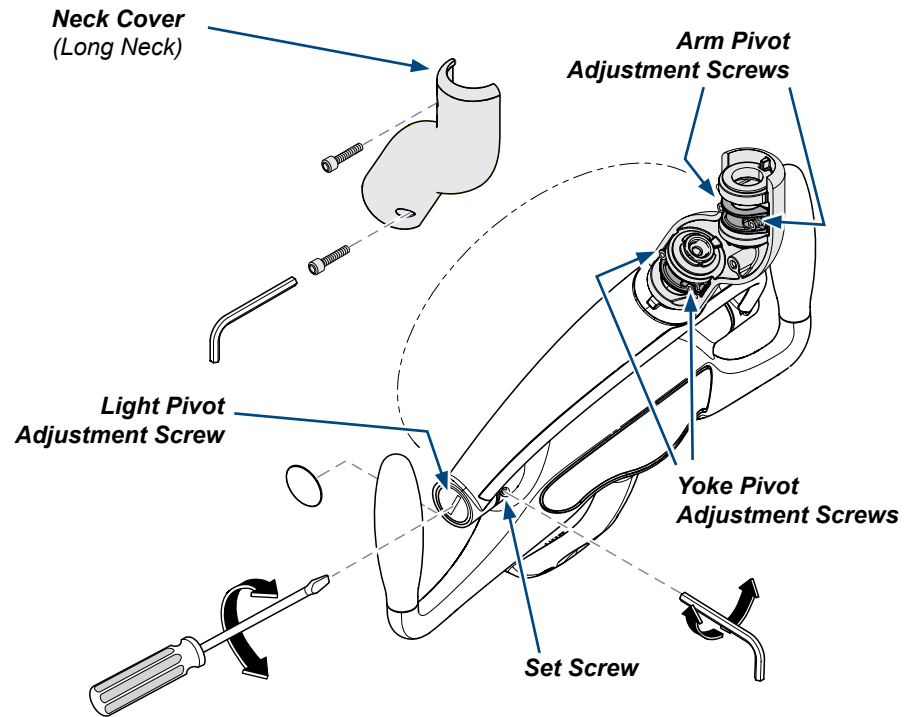To adjust Flex Arm tension...

- Remove end cap and top cover as shown.
- Loosen or tighten adjustment nut as desired.

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