

# Gas Disposal Kit Installation Instructions

## Intended Use:

Midmark Post or Wall Mounted Veterinary Active Scavenging Systems.

## Requirements / Restrictions:

For installation through outside walls only. Do not install in roof.

## Components:

|                                 |         |
|---------------------------------|---------|
| Gas Disposal Kit (P/N 92515144) | ..... 1 |
| {kit includes the following}    |         |
| gas disposal tube assembly      | ..... 1 |
| inner wall plate                | ..... 1 |
| tube clamp                      | ..... 1 |
| worm gear clamp                 | ..... 1 |
| #10 wood screw                  | ..... 1 |

## Special Tools:

|                    |                 |
|--------------------|-----------------|
| drill w/ 5/32" bit | screwdriver     |
| hole saw           | caulk gun/caulk |




### WARNING

Care must be taken to ensure that the installation of the gas disposal kit is such that it will not interfere with items normally found between the walls of a building or structure where this unit will be installed. A qualified contractor may be required to perform the procedures necessary for the proper installation of the gas disposal kit. Before cutting into any walls, be sure that there are no electrical wires, telephone wires, gas pipes, or anything else within or on the walls that can be damaged during cutting. Midmark is not responsible for any damage to the walls or anything else located on or within the walls, or any injury to a person(s) that may occur as a result of this installation.



### WARNING



Excess anesthetic gas shall be vented to a disposal point that is normally isolated from personnel, doors, windows, and any air intakes or outlets. Do not install the vent assembly through the roof or on or near the floor.

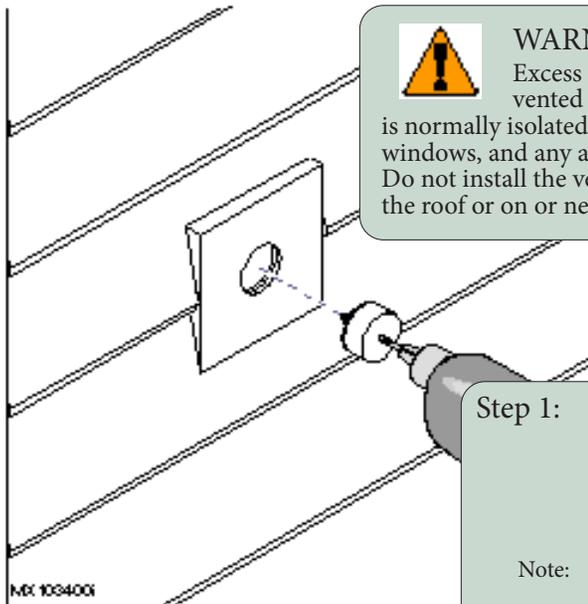

**Step 1:** Determine best location for Gas Disposal System and cut 1.38" (35mm) diameter hole through inside/outside wall.


**Note:** A board should be added to create a flat surface if uneven covering exist on outside wall.

**Step 2:** Fully insert gas disposal tube assembly from outside with vent facing downward and then mark mounting hole location.

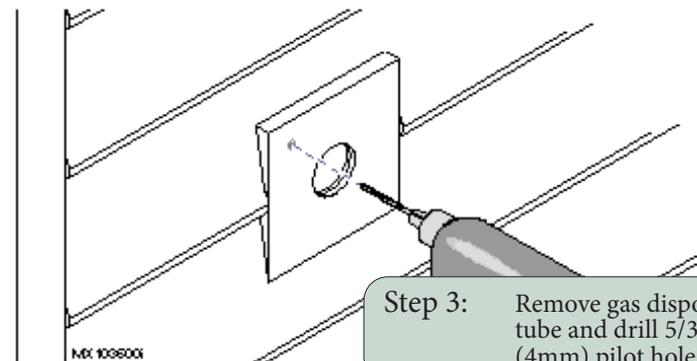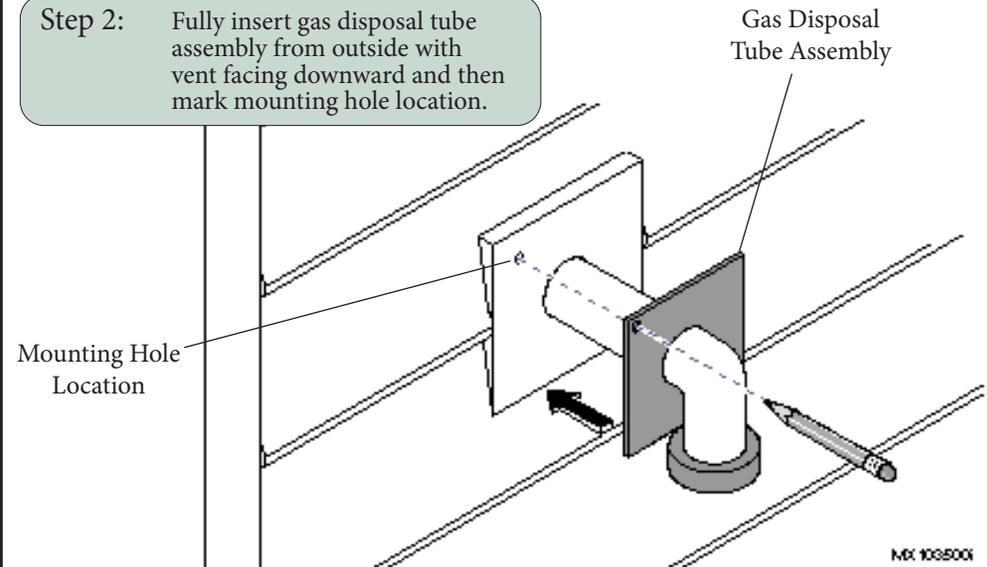

**Step 3:** Remove gas disposal tube and drill 5/32" (4mm) pilot hole.


Step 4: Place a bead of caulk around tube on inside of tube mounting plate.

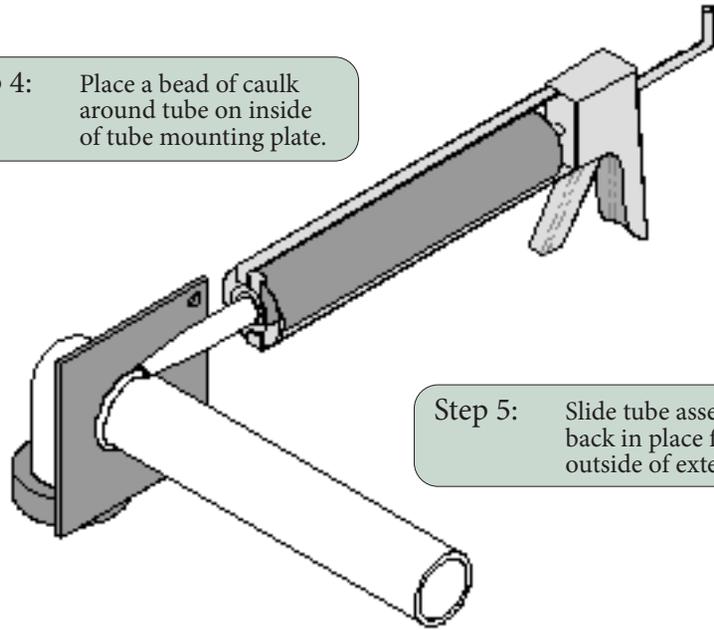

Step 5: Slide tube assembly back in place from outside of exterior wall.


MX 1037001

Step 8: Slide inner wall plate then tube clamp over gas disposal tube.

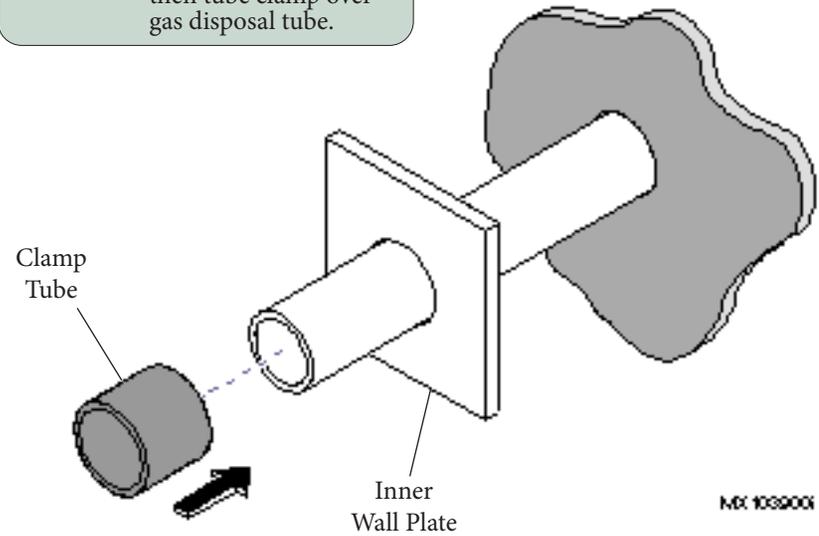

MX 1039001

Step 7: Place a bead of caulk around mounting plate on outside wall.

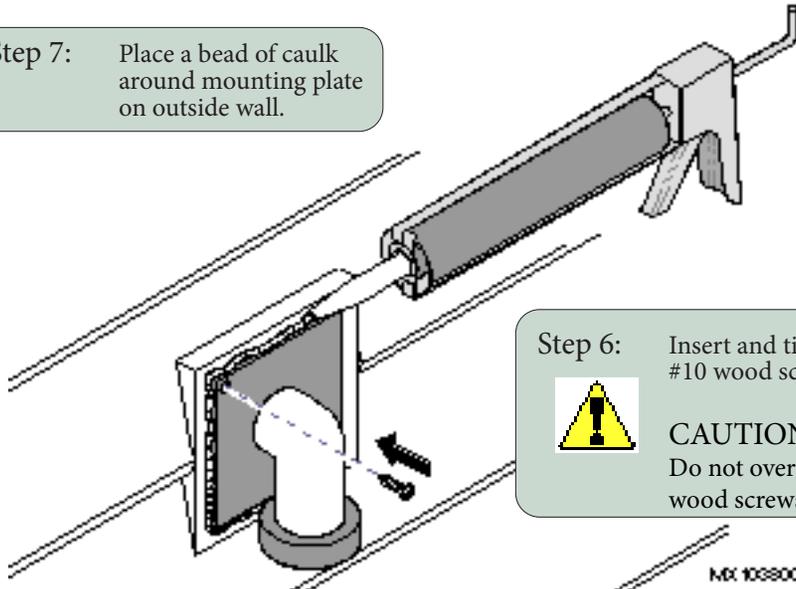


Step 6: Insert and tighten #10 wood screw.



**CAUTION**  
Do not overtighten wood screws.

MX 1038001

Step 9: Enlarge worm gear clamp if necessary then place over tube clamp and secure wall plate snugly in place.

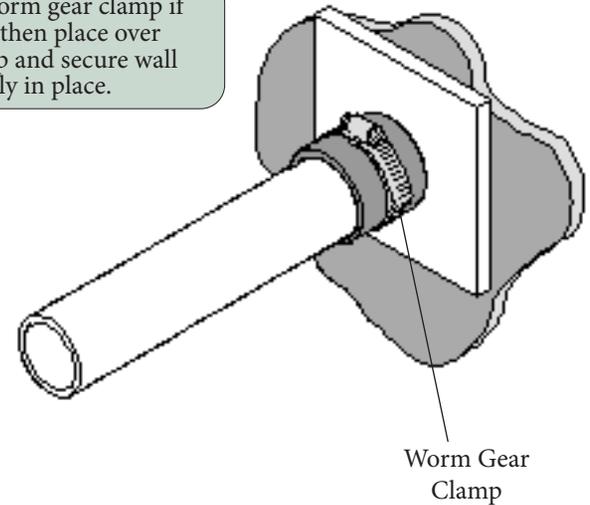

MX 1040001