

Daily

A. Replace Collection Canister

Midmark collection canisters are designed for single patient use only. Refer to collection canister manufacturer directions for replacement.

Bi-Annually

A. Clean Air Vents on Motors

Disconnect power. Wipe with a clean cloth to remove dust & debris.

Annually

A. Replace Vacuum Filter

Procedure is located in this guide.

B. Drain Moisture from Tank

Open service valve to drain any accumulated moisture in tank.


As Necessary

A. Vacuum Pump Flushing

Vacuum pump(s) can be flushed to remove deposits which build up on the pump vanes over time. Removal of these deposits will help to maintain proper vane performance. Contact Midmark Service Technician.

B. Adjust Vacuum Setpoints

Adjust vacuum setpoints at initial setup or when needed. Procedure is located in this guide.

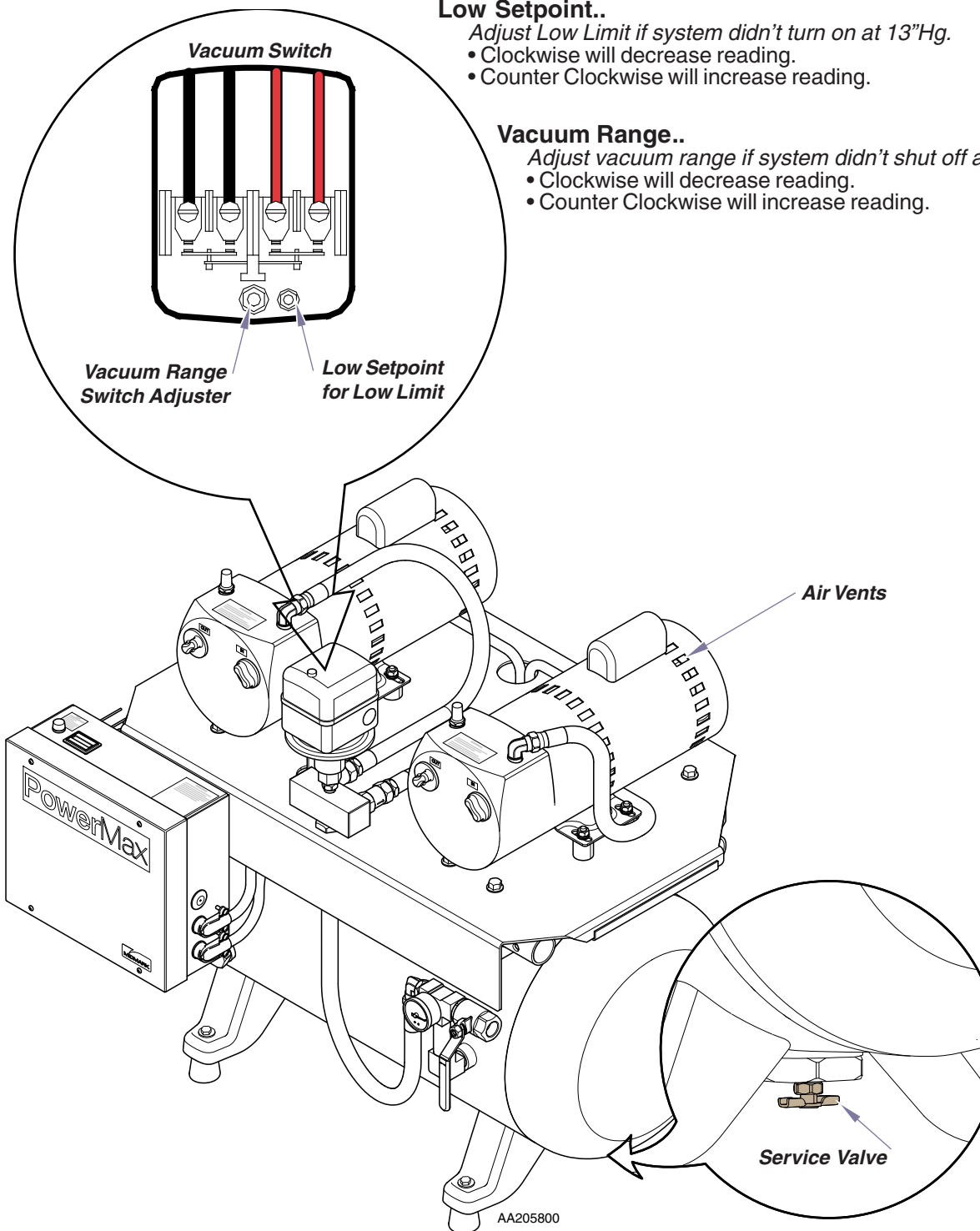

Low Setpoint..

Adjust Low Limit if system didn't turn on at 13"Hg.

- Clockwise will decrease reading.
- Counter Clockwise will increase reading.

Vacuum Range..

Adjust vacuum range if system didn't shut off at 20"Hg.

- Clockwise will decrease reading.
- Counter Clockwise will increase reading.

Maintenance Kits

Vacuum Filter

Order #: 77000796

Includes:

- 1 Filter (Order 2 per Head)

Flushing Solvent

Order #: 77000699

Accessories

Exhaust

(PM-1 & PM-3) Order #: 77000894

(PM-4) Order #: 77000895

Includes:

Exhaust kit to vent PowerMax to the outside.

Surgical Operatory Recovery Room Package

Order #: 77008037

Includes:

- Pkg of 5 Disposable Canisters (1200 cc)
- DISS Free Flow Regulator
- Outlet Station: Vacuum DISS
- Conductive Tubing - 6'
- Vacuum Slide and Ring
- Replacement Elbow

Disposable Canisters

(1200 cc)

Order #: 002-1882-00

Includes:

- Pkg of 5 Canisters

Midmark Corporation
60 Vista Drive
Versailles, OH 45380
Customer Service Phone: 1-800-643-6275 x 8923
Technical Service Phone: 1-800-643-6275 x 8890
Fax: 1-877-725-6495
www.midmark.com

