



The most tender care possible

Weigh, measure and examine infants safely and securely in one location. The Midmark 640—because your patients deserve the most tender, loving care.

Weighing infants can be efficient and accurate using the Midmark 640 with integrated scale. Press the Zero/On button and within seconds the scale display will read, "Scale ready." Place the infant on the scale and the weight will be automatically displayed. When you want to measure the patient, simply pull out the infantometer and slide it back into place when finished. Additional features include:

Reweigh allows you to recapture weight with the press of a button—no need to pick the infant back up.

A surface height of 36 inches provides easy access and reduces the need for repeated bending during the exam.

Upholstered end panels, beveled sides and flat surfaces with no sharp edges help ensure patient safety.

For storage needs, there is an oversized drawer as well as a bulk storage area designed for keeping diapers and equipment.

A hidden paper roll holder with a paper tear strap at one end adds convenience.

Specifications

Patient Weight Capacity: 4.5 lb (2 kg) - 40 lb (18 kg)

Scale Resoluton:
0.5 oz for pounds-ounces
0.02 lb for pounds
0.01 kg for kilograms

Scale Accuracy: < 22 lb (10 kg) +/- 1.0 oz (28 g) ≥ 22 lb (10 kg) +/- 2.0 oz (56 g)

Working Height: 35.8" (90.8 cm)

Table Top Length: 41.9" (106.4 cm)

Table Top Depth: 20.8" (52.7 cm)

Paper Roll Holder: Upper portion of table accommodates one 14" (35.6 cm) long, 3" (7.6 cm) diameter roll

Drawer.

33" x 15" x 4.125" (83.8 cm x 38.1 cm x 10.5 cm)

Drawer is constructed of high impact polystyrene with rounded corners for easy cleaning and is supported by side-mounted ball bearing steel slides.

Bulk Storage: 33.5" x 17" x 18.38" (85 cm x 43.2 cm x 46.7 cm)

Safety Standards

ANSI/AAMI ES 60601-1:2005 CAN/CSA C22.2 NO. 60601-1-08

Options

Door/Drawer Locks

Built-in Scale

Measures in pounds or kilograms with unit lock to assure consistency of reading. New reweigh feature allows for a quick weigh recheck with the push of a button.

Automatically shuts off after 30 seconds to conserve energy.

Standard Upholstery Colors

- Mist-813
- Stone-814
- Restful Path-858
- Iron Ore-851
- Lunar Gray-845
- Obsidian-857
- Oasis-843
- Soothing Blue-855
- Dream-847
- Mineral-844
- Healing Waters-848
- Shaded Garden-853
- Citrus-849
- Sandy Retreat-850
- Curative Copper-846
- Cranberry-859
- Dark Linen-856
- Latte-860
- Robust Brown-852
- Deep Earth-854

UltraFree* Upholstery Colors

- UltraFree Stone-865
- UltraFree Restful Path-868
- UltraFree Obsidian-867
- UltraFree Harbor-861
- UltraFree Wheatgrass-862
- UltraFree Spice-863
- UltraFree Cranberry-869
- UltraFree Dark Linen-866
- UltraFree Latte-870
- UltraFree Branch-864

*Ultraleather® by Ultrafabrics

Midmark is an ISO 13485 and ISO 9001 Certified Company For more information or a demonstration, contact your Midmark dealer or call: 1-800-MIDMARK Fax: 1-800-365-8631 Outside the U.S.A. call: 1-937-526-3662 Fax: 1-937-526-8392 or visit our website at midmark.com

© 2010 Midmark Corporation Midmark Corporation, Dayton, OH. Products subject to improvement changes without notice Litho in U.S.A. 007-0674-00 Rev. G1 (1/18)

