



## Daily

### A. Clean Vacuum Piping System.


Clean piping system with a non-foaming vacuum system cleaner. Midmark recommends Precision Clense™, Order No. 77000570.

### B. Clean Vacuum Inlet Strainer.

Turn vacuum "OFF". Remove, replace and or clean filter bowl, screen and gasket. Reassemble. Turn vacuum "ON".

### Note - Recycler Models Only

If you have low to no suction due to a clog by residue building up over time, there is a feature on vacuum to keep it running until a service technician can troubleshoot the problem. Turn the water by-pass valve to the OPEN position. This will by-pass the recycler and may keep the pump running.

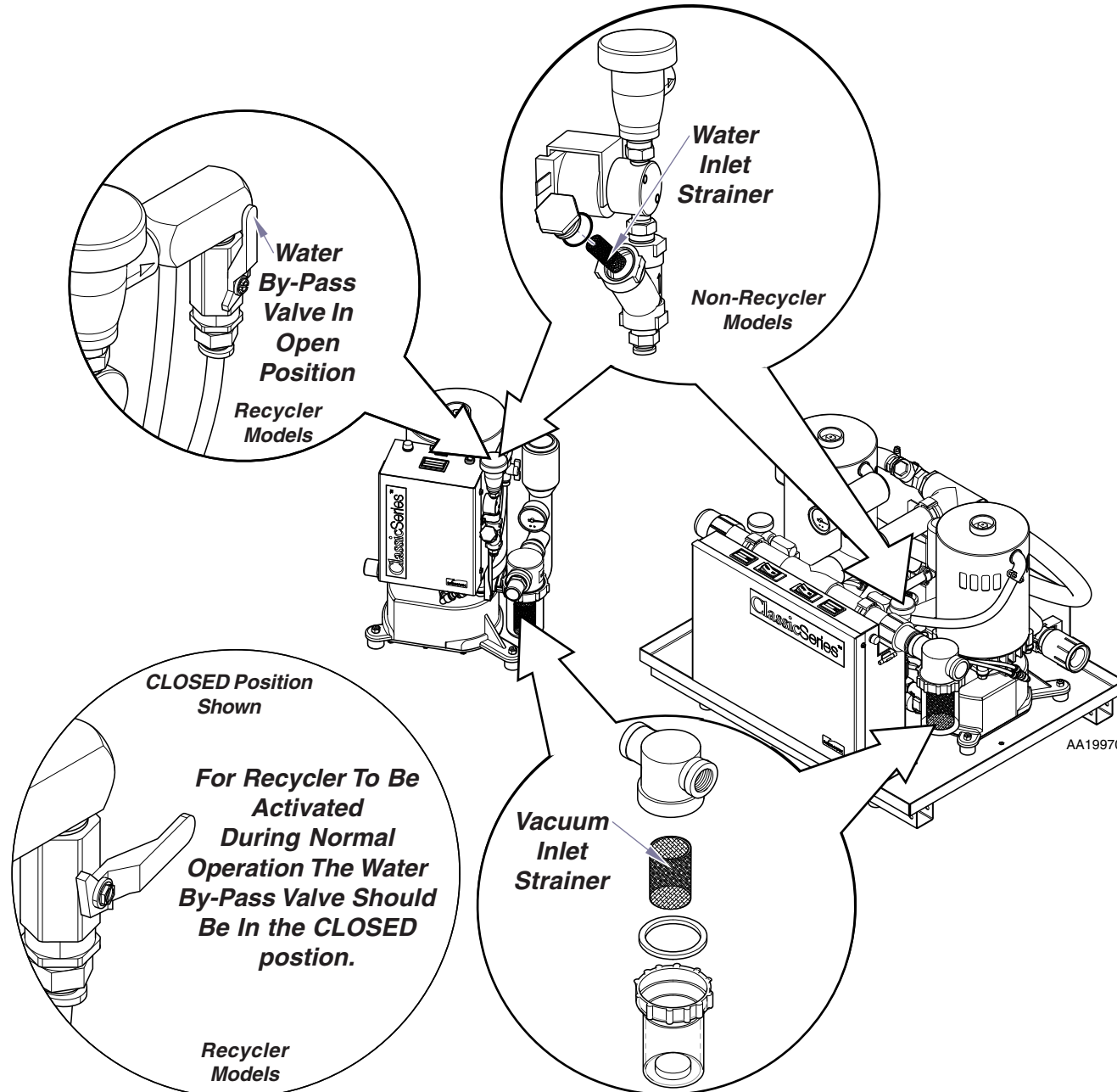

## Annually

### A. Clean and or Replace Vacuum Relief Filter.

Pull filter out of end cap. Blow off filter and or replace if damaged. Replace filter in VRV end cap.

### B. Clean Water Inlet Strainer.

Turn vacuum "OFF". Remove cap on "Y" strainer housing and pull out the wire intake strainer. Rinse with water and blow off. Replace strainer and cap. Turn vacuum "ON".

### C. Clean / Dust Vacuum Pump(s).

Wipe with a clean cloth to remove dust & debris.


## Maintenance Kits



### Vacuum Relief Valve Filter

Order #: 62935601

Includes:  
• Filter

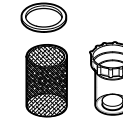

### Vacuum Inlet Strainer


Order #:

Single Models – 62937301

Twin Models - 77005155

Includes:

• Filter bowl, screen and o-ring.

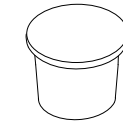

### Dispos-A-Bowl

Order #:

Single Models – 77005167

Twin Models - 77005168

Includes:

• Container, lid and red safety tape.

## Accessories




### Exhaust Silencer

Order #: 77000040

Includes:

• Exhaust Silencer

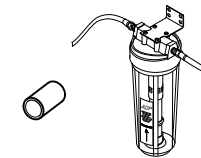

### Mineral Reducer Kit

Order #: AVA8200

Includes:

• Installation instructions, housing, cartridge, all necessary mounting and installation hardware.

Order #: PFM8200

Includes:

• Cartridge for Mineral Reducer.