



MEDICAL

630 HUMANFORM[®] Procedure Chair



Healthcare is changing. Are you changing with it?



Healthcare has been changing at an unprecedented rate. There are more patients, fewer providers, and patients are trending away from hospitals into office settings. Having the right combination of technology and workflow is more important now than it ever has been.



Position yourself + your patients for **in-office procedures**


Close to half of all procedures in the US are now performed in the office setting.¹ With advancements in technology, anesthesia and pain management, it is expected that the number of procedures being performed in an office setting will continue to rise.² Is your facility equipped to handle this migration of care?




In-office procedures are on the rise

60
million

The number of in-office procedures performed in 2024.¹

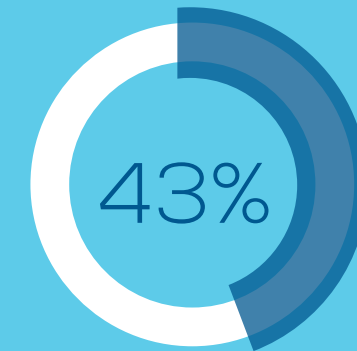

In 2024, nearly half of all procedures in the US were performed in the office setting.¹

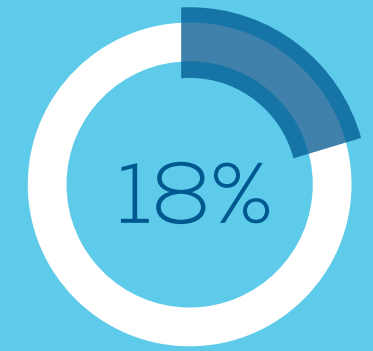

The estimated increase in the volume of in-office procedures by 2032.²

MIDMARK CAN HELP

The Midmark 630 HUMANFORM® Procedure Chair is designed with precise positioning and more humanlike movement to provide better accessibility and ergonomics for patients and caregivers.

Bringing patient + provider together comfortably + safely

Since the inception of the Americans with Disabilities Act (ADA), accessibility has become a legal requirement. But more importantly, accessible design is instrumental in providing better care.

The Midmark 630 HUMANFORM® Procedure Chair can lower to a height of 19" (non-rotational models) and can be moved into a low, flat position. The low seat height and wide, flat transfer surface can help enable patients to transfer onto and off of the procedure chair with minimal assistance, improving safety for the patient and caregiver.



Natural, humanlike movement
patients will appreciate



01 BARRIER-FREE DESIGN

helps address the needs of a changing patient demographic with a 19" low seat height and 650-pound weight capacity.

02 MOTION PROFILING®

adjusts the chair as the patient moves to help maintain head and back support. This exclusive feature is designed to eliminate the need for patients to reposition as the chair back section raises or lowers.

03 FOOT EXTENSION

extends the chair footrest to accommodate taller patients with the touch of a button.

04 PREMIUM COMFORT SYSTEM®

combines unique, contoured upholstery and an adjustable footrest to cradle the patient for a soothing, chair-like feel.



Better position caregivers for in-office procedures


No matter the type of procedure, proper positioning is necessary for unobstructed access to the patient and a clear view of the surgical site. The Midmark® 630 has 8-way powered positioning (height, back, foot and tilt) giving caregivers the freedom to effortlessly move patients into the most effective positions.

1 out of 5

Of all injuries reported in the US is healthcare related.³

3x

Healthcare workers are almost three times as likely to suffer from work-related injuries than construction workers.³

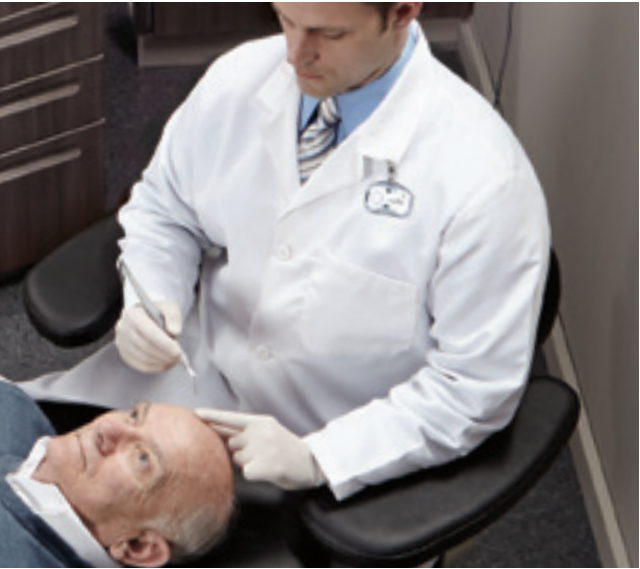

CAREGIVER ERGONOMICS

When performing in-office procedures, it's important to choose a procedure chair that provides clear access to the patient and procedure site. But have you considered provider ergonomics? The Midmark 630 HUMANFORM® Procedure Chair is designed with features that help ensure ample access to the patient while maintaining a proper working position whether seated or standing.

PRECISE MEDICAL POSITIONING



LITHOTOMY

Position patients for lower-body procedures



ELEVATED FLAT

Can improve access and visibility to the patient's head, neck and torso



CHAIR ENTRY

Positions the patient in the upright position for entry and exit



TRENDELENBURG

Can be used as a first-line treatment of acute hypotension and shock

ADJUSTABLE HEADREST DESIGN



Articulating Headrest

Tilt the patient's head or adjust for different procedures, improving access and patient comfort.



Tool-Free Adjustment

Simply loosen the knob, rotate the headrest up or down and retighten.



Intelligent features make time with patients more efficient

With more procedures being performed in office settings and patient volumes on the rise, efficiency and comfort are more important than ever. The Midmark 630 HUMANFORM® Procedure Chair helps create a better care experience—for you and your patients.



Programmability
is positioning
made easy

The Midmark® 630 can store up to four of your most used positions. With the touch of a button, all sections of the chair move simultaneously to quickly and precisely move the patient to the desired position. (Programmability is an optional feature.)



MAKE THE MOST OF THE SPACE YOU HAVE WITH **ROTATION**

With the optional rotation feature, you no longer have to drag heavy devices around the chair or the patient. You can simply move the patient for better access.



DUPLEX RECEPTACLES

Hospital-grade, duplex receptacles on each side of the chair allow you to easily power ancillary devices at the point of care. (Not available on rotational or 230 V models.)



WIRELESS CONTROLS

Locate your optional wireless hand and foot control anywhere in the room without having to worry about trip hazards or cords. A low battery audible indicator will sound when it's time for a battery replacement.



ACTIVE SENSING TECHNOLOGY®

Avoid impact or disruption while moving the chair. Active sensing technology pauses chair movement when disrupting impact is detected.



Accessories

ACCESSORIES HELP ADD VERSATILITY + FUNCTION TO YOUR PROCEDURE CHAIR

OVAL HEADREST



12.7" H x 12.3" W

EXTENDED HEADREST



12.7" H x 12.4" W

RECTANGULAR HEADREST



12" H x 18" W

U-SHAPED HEADREST



8" H x 11.5" W

DOUBLE ARM
INSTRUMENT TRAY



Attaches to either side of
the chair

RITTER® 253 LED EXAM LIGHT



Available in chair mount (spline or
rail), wall mount and mobile options

ARTICULATING KNEE
CRUTCHES



LOCKING CASTERS



3 1/2" added height to chair

SECURITY SIDE PANELS



ACCESSORIES

CHAIR ARMS



Fits 28" and 32" upholstery widths

SEAT RAILS



Surgical size 1 1/8"

BACK + BASE ACCESSORY RAILS



Surgical size 1 1/8" and standard size 1"

GYNECOLOGY DEBRIS TRAY



*Larger option debris tray
13" W x 12" L x 1/2" D*

HAND SURGERY ARMBOARD



8.5" W x 26" L

SECURITY STRAPS



Adjustable width and location

ARTICULATING ARMBOARD



7" W x 24" L

VISION BLOCK SCREEN

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


Blocks patient's line of sight



COLOR OPTIONS

STANDARD UPHOLSTERY COLORS

	Mist 813
	Stone 814
	Restful Path 858
	Iron Ore 851
	Lunar Gray 845
	Obsidian 857
	Oasis 843
	Soothing Blue 855
	Dream 847
	Mineral 844
	Healing Waters 848
	Shaded Garden 853
	Citrus 849
	Sandy Retreat 850
	Curative Copper 846
	Cranberry 859
	Dark Linen 856
	Latte 860
	Robust Brown 852
	Deep Earth 854

ULTRAFREE UPHOLSTERY COLORS

	UltraFree Stone 865
	UltraFree Restful Path 868
	UltraFree Obsidian 867
	UltraFree Harbor 861
	UltraFree Wheatgrass 862
	UltraFree Spice 863
	UltraFree Cranberry 869
	UltraFree Dark Linen 866
	UltraFree Latte 870
	UltraFree Branch 864



UPHOLSTERY

Available in 28" and 32" widths to accommodate a range of patient sizes.

OPTIONS + ACCESSORIES

Options

- Premium Upholstery Top: Seamless design
- Premium UltraFree Top: Seamless, PVC-free design
- Programmability: Four customizable positions
- Wireless Controls: Hand and foot
- Electrical Outlets: Duplex, hospital grade (on each side)*
- Rotation: Allows table to rotate 350° and adds 3.5" to the minimum and maximum seat height specifications
- CE marked chairs available

Accessories

- Ritter® 253 LED Exam Light
- Back and base accessory rails - Surgical size 1 1/8" and Standard size 1"
- Seat rails - Surgical size 1 1/8"
- Adjustable arm system - Includes left and right arms
- Articulating armboard
- Hand surgery armboard
- Knee crutches - Articulating
- Gynecology debris tray
- Security straps
- Security side panels
- Instrument tray - Double arm
- IV pole
- Vision Block Screen
- Tamper resistant receptacles*
- Casters*
- 3' power cord
- Additional hand and foot controls
- U-shaped headrest
- Rectangular headrest
- Extended headrest
- Oval headrest
- * Not available on rotational or 230 V models

SERVICES + SUPPORT

TECHNICAL SPECIFICATIONS

View technical specifications for the Midmark® 630 Procedure Chair.



SCAN TO VIEW SPECS.

MIDMARK LIVE DESIGN

Our in-house design experts are ready to help you every step of the way.



SCAN TO VIEW VIDEO.

REPAIR + SERVICE

Explore available support and service plans so you can obtain the level of coverage you need to keep your medical equipment and devices operational. This includes periodic maintenance and periodic inspection plans, as well as extended warranty plans.



SCAN FOR SUPPORT.

MIDMARK DELIVERY SERVICES

Coordination, delivery and setup right on time, every time.



SCAN TO VIEW YOUR OPTIONS.

SOURCES

1 Retrieved November 2024 from: <https://clarivate.com/coronavirus-resources/?lid=d>
See "Impact on Medical Procedure... tab"

2 <https://www.ascfocus.org/ascfocus/content/articles-content/articles/2022/digital-debut/sg2-annual-report-projects-double-digit-asc-growth>

3 <https://www.bls.gov/charts/injuries-and-illnesses/number-and-rate-of-nonfatal-work-injuries-by-industry.html>



Designing better care.®



CARB 93120.2 Phase 2 Compliant and TSCA Title VI Compliant

The color examples shown are the best representation of the original material. Actual color may vary slightly. We strongly recommend that you contact Midmark Customer Experience at 1.800.MIDMARK to request a sample before placing your order.

Countries within the EU should direct all questions, incidents and complaints to the Midmark Authorized Representative listed below:

CEpartner4U, Esdoornlann 13, 3951 DB Maarn, The Netherlands
Phone: +31.343.442.524 Website: cepartner4u.com

Midmark is an ISO 13485 and ISO 9001 Certified Company.
Certain products are not included. See the complete list at:
midmark.com/ISO

For more information, contact your authorized Midmark distributor or call: 1.800.MIDMARK

Outside the US call: 1.937.526.3662 or visit our website: midmark.com

USGBC® and the related logo are trademarks owned by the U.S. Green Building Council® and are used with permission.

© 2026 Midmark Corporation, Versailles, Ohio USA
Products subject to improvement changes without notice.

007-10305-00 Rev. D1 (7/26)

