



DENTAL

Sterilizers



Reduce stress and improve retention



Instrument processing is a highly regulated, time-consuming and labor-intensive task that can become a primary contributor to teammate stress and burnout. That's why we designed **Midmark® Steam Sterilizers** to help make instrument processing and adherence to clinical best practices as easy and as automated as possible.



Midmark® Steam Sterilizers

SIMPLIFY STERILIZATION AND GET YOUR TEAM BACK TO THE WORK THEY LOVE. CHOOSE TECHNOLOGY DESIGNED WITH THEM IN MIND.

Midmark Steam Sterilizers are designed with intuitive onscreen guidance, helpful notifications and step-by-step touchscreen prompts that can streamline compliance, cycle operation, and routine care. Save time, simplify teammate training and build instrument processing confidence with automated mechanical monitoring and chemical indicator reporting.

INTUITIVE USER INTERFACE:

Navigate cycle setup and other processes in your preferred language—English, French or Spanish—with a clear, fingerprint-resistant 5-inch touchscreen you can utilize while wearing gloves. Choose a style—light or dark mode.



SCAN TO
VIEW VIDEO.



PROTECT YOUR INVESTMENT

Register your warranty today at midmark.com/protectyourinvestment to receive prompt support when you need it and opt in to receive future notifications for available software updates.



01



02



01 MIDMARK M11® STEAM STERILIZER

The 11" x 18" chamber makes the M11 one of the largest of any standard tabletop sterilizers on the market.

02 MIDMARK M9® STEAM STERILIZER

Pack all the reliable sterilizing power you need into a compact unit perfect for areas with limited space.

Midmark sterilizers are designed to help make instrument processing as easy and as automated as possible.



WRAPPED
270°F, 4 MIN



WRAPPED
275°F, 3 MIN



DELICATE
250°F, 30 MIN

Simplify clinical workflow with dynamic-air-removal steam sterilizers that provide unique preset cycles to meet both the CDC and AAMI sterilization cycle recommendations for wrapped instruments.¹




UNWRAPPED
270°F, 3 MIN




5 CUSTOM CYCLES*
250°F–275°F,
3 MIN–45 MIN


Determine cycle status—cycle in progress, cycle complete and cycle error—from a distance with a progressing color-coded LED light bar, large cycle-countdown clock and audible cues.

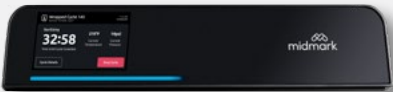

CYCLE IN PROGRESS

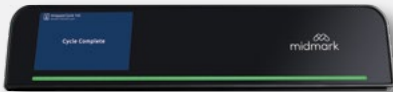

CYCLE COMPLETE

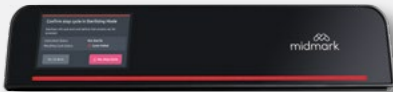

CYCLE ERROR

* Custom cycles are not FDA cleared and validation of sterility of items processed using a custom cycle is the responsibility of the user.



STREAMLINED COMPLIANCE RECORDKEEPING:
 Stay audit-ready with device reminders and notifications, user authentication, unlimited storage of routine care events for the life of the sterilizer, and automated cycle recordkeeping.



SCAN TO
 VIEW VIDEO.



DURABILITY: Midmark sterilizers have been designed for durability, including an increased cycle life of 25,000 cycles thanks to a completely reengineered chamber. This enhanced durability can reduce the frequency of required maintenance by users and certified technicians by more than half.



SCAN TO
 VIEW VIDEO.

ACCESSORIES



9A307001 COOL HAND TOOL
 (Tray and cassette removal tool)



9A694001 AUTOFILL 115V
 (9A694002 Autofill 230V available)



9A698001 USB KIT
 Download cycle records including time, temperature and pressure. Includes USB drive, USB cable and PC board (technician installation required).



9A703001 VERTICAL CASSETTE RACK
 (M11 Steam Sterilizer models only)



**9A586002 DOUBLE VISTACOO™
 DIRECT-TO-DRAIN SYSTEM**

SERVICES + SUPPORT

TECHNICAL SPECIFICATIONS
 View technical specifications for Midmark M9® and M11® Steam Sterilizers.



SCAN TO
 VIEW SPECS.

MIDMARK LIVE DESIGN
 Our in-house design experts are ready to help you every step of the way.



SCAN TO
 VIEW VIDEO.



SOURCES

1 FDA - <https://www.fda.gov/media/80265/download>



Designing better care.®



CARB 93120.2 Phase 2 Compliant and TSCA Title VI Compliant

The color examples shown are the best representation of the original material. Actual color may vary slightly. We strongly recommend that you contact Midmark Customer Experience at 1.800.MIDMARK to request a sample before placing your order.

Midmark is an ISO 13485 and ISO 9001 Certified Company. Certain products are not included. See the complete list at: midmark.com/ISO

For more information, contact your authorized Midmark distributor or call: 1.800.MIDMARK

Outside the USA call: 1.937.526.3662 or visit our website: midmark.com

© 2025 Midmark Corporation, Versailles, Ohio USA
Products subject to improvement changes without notice.
Litho in USA.

007-10489-00 Rev. B1 (3/25)

