



MEDICAL

Patient Support Rails



Supporting accessibility and compliance in examination and procedure rooms



Midmark 626 Barrier-Free® Examination Chair



01



02



03

US ACCESS BOARD COMPLIANT TRANSFER SUPPORTS FOR EXAMINATION + PROCEDURE CHAIRS

The Department of Justice and the Department of Health and Human Services are actively enforcing the 2017 US Access Board Standards, requiring fully compliant exam and procedure chairs to have transfer supports that are designed to meet the standards. Midmark Patient Support Rails and Patient Support Rails+ are designed to meet the US Access Board Standards and allow patients to safely transfer on each side of the exam or procedure chair.



Ritter 228 + 229 Barrier-Free® Examination Chairs

01 ROTATION/LOCKING POINTS

Patient Support Rails can be locked into three positions—front horizontal or two elevated positions—or moved to a fully back horizontal position.

02 SAFETY RAILS

When rotated to the back horizontal position, the support rails can be used as safety rails, designed to improve patient security and provide a gripping point during movement into supine, prone, left or right lateral positions.

03 CONTINUOUS GRIPPING SURFACE

Patient Support Rails provide patients with a 1 ¼ inch diameter continuous gripping surface. They can be used as an affixed gripping point for patients to independently raise themselves off of the exam or procedure chair.



Midmark 631 Procedure Chair

This document references MDE Standards' Specifications for Accessibility M302 and M305. For more information, visit: <https://www.access-board.gov/mde/>

Proper arm support for more precise and accurate blood pressure readings



01 BETTER BP

For a [more accurate blood pressure measurement](#), the patient's arm should be supported with cuff at heart level. If the upper arm is below heart level or unsupported, the readings will be too high. If the upper arm is above heart level, the readings will be too low.¹

02 BETTER POSITIONING

The Patient Support Rails+ accessory functions similarly as the Patient Support Rails with the added benefit of helping providers achieve proper positioning of the patient's arm for blood pressure capture and blood draw applications.

03 BETTER ARM SUPPORT

Easily support the patient's arm with cuff at heart height with the Patient Support Rails+ accessory. The unique articulating arm pad design allows for more precise positioning for blood pressure capture or other similar applications.

03



Articulating Arm Pad

SPECIFICATIONS



SCAN TO VIEW
USER GUIDE.

¹ <https://www.aafp.org/afp/2005/1001/p1391.html>



Designing better care.®

Midmark is an ISO 13485 and ISO 9001 Certified Company.
Certain products are not included. See the complete list at:
midmark.com/ISO

For more information, contact your Midmark distributor
or call: 1.800.MIDMARK

Outside the USA call: 1.937.526.3662 or visit our
website: midmark.com

© 2026 Midmark Corporation, Versailles, Ohio USA
Products subject to improvement changes without notice.

007-10145-00 Rev. C1 (7/26)

