



A guide for:

PVC Coated Floor Care & Cleaning

SHOR-LINE
SCHROER MFG. CO.

{Save This Guide For Future Reference}

PN: 006.0098.00
Updated: 4.25.2012



Introduction

Thank you for purchasing Shor-Line products. As a leader in animal care equipment, our commitment to you is exactly the same as it was in 1927.

We hope this guide answers any questions you might have in regards stainless steel care and cleaning. If you can't find the answers to your questions, have further questions or would like to purchase additional products, please contact us at 800.444.1579 or 913.281.1500. If in Europe phone +44 1446 77 20 41.

Consider keeping this guide in a safe and convenient place for future assistance.

{Shor-Line}

Schroer Manufacturing Company
511 Osage Ave.
Kansas City, Kansas 66105, USA
phone 800.444.1579 fax 913.281.5339
email guides@shor-line.com
www.shor-line.com

{Shor-Line Limited}

Vale Business Park
Llandow,
Vale of Glamorgan CF71 7PF
United Kingdom
phone +44 1446 77 20 41 fax +44 1446 77 36 68
email quality@shor-line.co.uk
www.shor-line.co.uk

▲CAUTION

Remove all tags and collars from animal before placing on the floor. Place towel or matting under head of any anesthetized animal.

▲WARNING

Shor-Line products are designed for animal use only.

▲NOTICE

Dropping, Dragging and/or excessive weight will impair the coating & structure of this floor and void its warranty. Handle with care.

California Proposition 65 Warning



WARNING This product can expose you to chemicals including chromium, which are known to the state of California to cause cancer. For more information go to: www.P65Warnings.ca.gov



PVC Coated Floor Care & Cleaning

{Quality & Life of the Floor}

IMPORTANT: Routine maintenance will ensure quality and life of your long-term investment and protect animals from transmittable disease and infection.

IMPORTANT: To safeguard your investment and rights to the Shor-Line's Warranty, it is important that you know factors that can degrade the PVC Coated Floor. Permanent damage or discoloration of the PVC Coating on your floors can occur from the following:

- Cleaning with high pressure washers. (Prolonged exposure can damage the coating.)
- Cleaning with hot water exceeding 180° F for longer than 10 minutes at a time.
- Exposure to heavy concentrations of corrosive cleaning agents such as: Zylene, Toluene, Powder/Liquid Bleach, Chlorine Shock, Peroxide or any type of solvent/cleaning material containing Bleach or Chlorine.
- Improper and careless handling such as throwing, sliding and cutting.
- Excessive weight being placed on the floors.

{Care & Cleaning Best Practices}

IMPORTANT: You play an essential role in safeguarding your investment and rights to Shor-Line's Warranty. We **strongly recommend** using only the following green cleaning agents to maintain the quality and durability of the PVC:

- Liquid Dishwashing Detergent (not Dishwasher Detergent)
- A cleaner suitable for Vinyl sheet flooring
- A cleaner suitable for Vinyl pool liners
- A cleaner suitable for flexible Vinyl outdoor furniture

Note: All cleaning agents must be used at their recommended strengths or damage could occur to the PVC coating.

SHOR-LINE[®]

SCHROER MFG. CO.

toll free	local	fax
800	913	913
444.1579	281.1500	281.5339

Shor-Line/Schroer Manufacturing Co.
511 Osage Avenue
Kansas City, KS 66105 U.S.A.

www.shor-line.com