



Midmark®
Smart View™

Release 2.4



SUMMARY OF RELEASE 2.4

This release introduces enhanced functionality for myMidmark® Smart Air Compressor and Smart G-Vacuum systems.

Key features of this release include the following new capabilities:

- Extended firmware update window for easier maintenance and setup
- Improved Wi-Fi and network stability for more reliable connectivity
- New 5-minute sleep timer helps keep the pump dry to help enhance reliability on Smart G-Vacuum
- Added purge cycle capability (VFD 5.15 or higher) to help enhance system longevity

IMPORTANT

Preparing Your Smart G-Vacuum for the Update

Put your Smart G-Vacuum into standby mode by following these steps:

- 1. Do Not Turn the Device Off Completely:** The device must remain in standby mode for the update.
- 2. Locate the On/Off Toggle Switch:** Go to the mechanical room and find the switch on the front of the device.
- 3. Activate Standby Mode:** Toggle the switch to the "Off" position. This will put the device into standby mode, rather than shutting it down.
- 4. Complete the Update:** Once the update is complete, switch the device out of standby mode by toggling the switch to the "On" position.



RELEASE 2.4 INCLUDES THE FOLLOWING FEATURES

Smart Air Compressor

- The firmware download grace period has been extended
- Improvement in network stability and Wi-Fi connectivity

Smart G-Vacuum

- The vacuum sleep timer now runs for 5 minutes instead of 1 minute when in sleep mode
- A purge cycle is now available (requires VFD version 5.15 or higher)
- The firmware download grace period has been extended
- Improvement in network stability and Wi-Fi connectivity