

Midmark®
Smart View

Release 1.2



SUMMARY OF RELEASE 1.2

This release includes connectivity enhancements for Midmark Smart Air Compressors and Smart G-Vacuums. It also makes registering and provisioning next generation Midmark® Steam Sterilizers easier than ever.

Key features of this release include the following new capabilities:

- Wi-Fi enhancements for Smart Compressors and Smart G-Vacuums to improve network stability
- Next generation Midmark Steam Sterilizers added to Smart View

*IMPORTANT

Preparing Your Smart G-Vacuum for the Update

To put your Smart G-Vacuum into standby mode, follow these steps:

1. Do Not Turn the Device Off Completely:

The device must remain in standby mode for the update.

2. Locate the On/Off Toggle Switch: Go to the mechanical room and find the switch on the front of the device.

3. Activate Standby Mode: Toggle the switch to the "Off" position. This will put the device into standby mode, rather than shutting it down.

4. Complete the update: Once the update completes, switch the device out of standby mode by switching to the "On" position.



RELEASE 1.2 INCLUDES THE FOLLOWING FEATURES

Wi-Fi Enhancements:

Wi-Fi enhancements for Smart Compressors and Smart G-Vacuums to improve network stability.

Next generation Midmark Steam Sterilizers added to Smart View:

- Quick device setup and connection to the cloud
- On-device update cancellation, providing flexibility to prioritize critical operations
- Proactive notifications that alert users when the device goes offline, enabling prompt action to minimize downtime



Remote Software Update Cancellation for Next Generation Sterilizers:

Users can cancel software updates directly from the sterilizer touchscreen, ensuring that immediate operational needs are prioritized over pending software updates.

This feature is particularly valuable when an administrator schedules an update via the online portal, but a device user needs to cancel the update to operate the sterilizer. Update cancellation not only halts the update on the device but also communicates this change back to the cloud so the update will be available when the user is ready to reschedule.

This feature ensures that the sterilizer remains operational when the user needs it most.

Device display:

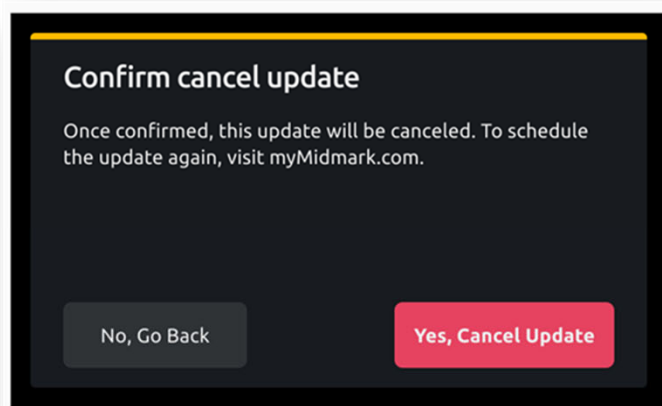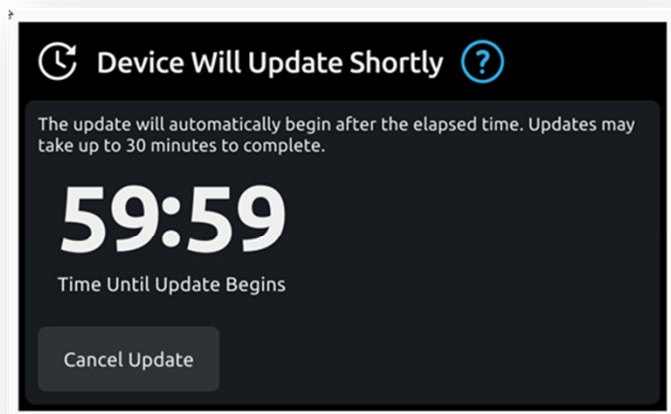


Email Notifications When the Sterilizer Goes Offline:

Now, Smart View can alert customers when their connected sterilizer goes offline and requires reconnection. Keeping users informed about the status of their sterilizer can help reduce downtime and operational disruptions.

- Improves awareness of potential issues while empowering users to take immediate action, fostering a sense of control and responsiveness
- Supports a seamless and connected experience, enhancing overall satisfaction with both the product and the associated portal
- Helps ensure sterilization processes remain uninterrupted and efficient, contributing to a smoother and more reliable operation

Users can receive notifications within the myMidmark.com portal and via email. (They can turn on or off emails based on user preferences in the portal.)


Email Notifications When the Sterilizer Goes Offline:

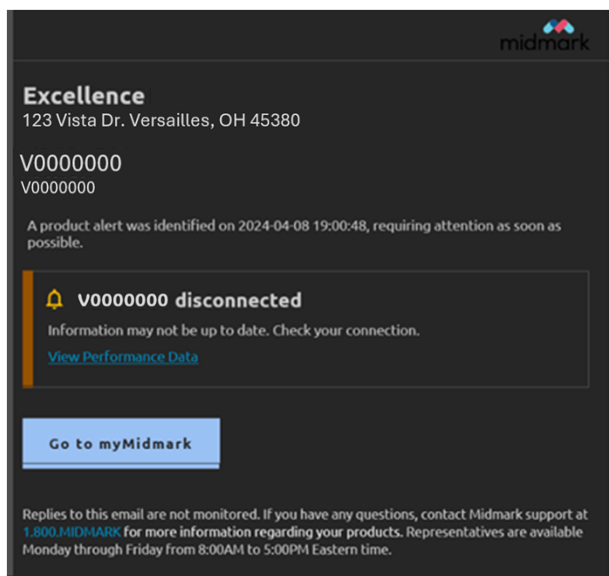

Remote Updates:

Remote updates help ensure all connected devices are kept up to date for optimal performance, regardless of their locations.

- Access to the newest features and updates, minimizing maintenance
- Notifications of available software updates for all connected devices
- Remote management of updates for all connected devices

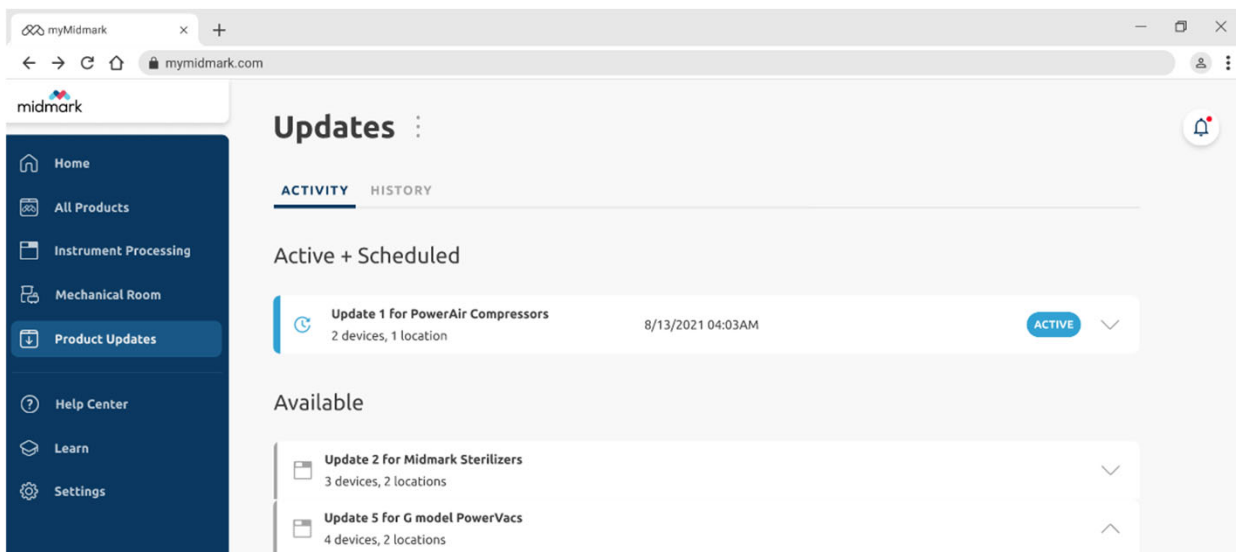


Slide 3

DM0


Is there content that is supposed to be ahead of these images? Or is that what the paragraph on the prior page was meant to do? If so, then I would suggest a page break after the Release Details and the start of the section with screenshots. Or some sort of visual separation that this is more detail versus the summary.

Diane Millis, 2024-11-08T21:08:19.888

View update details:

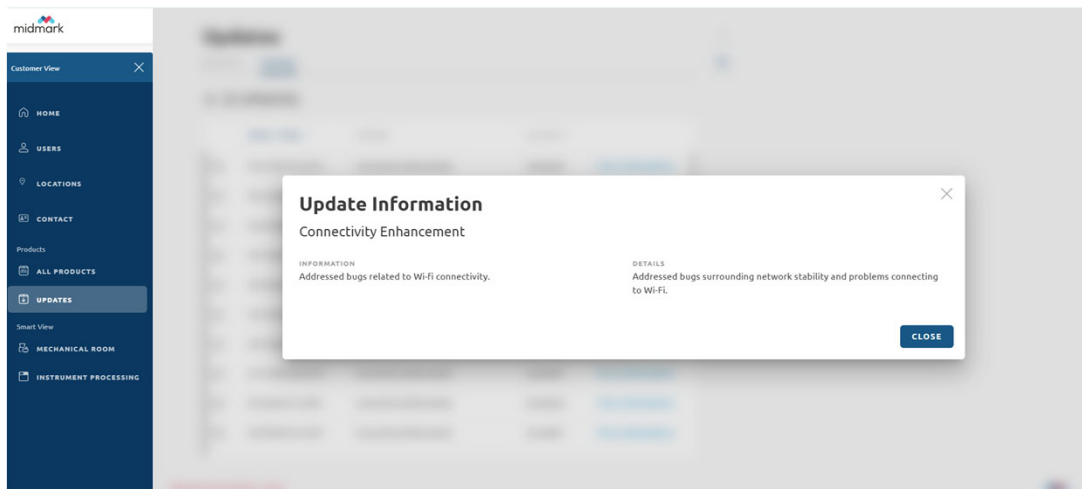

Schedule update:

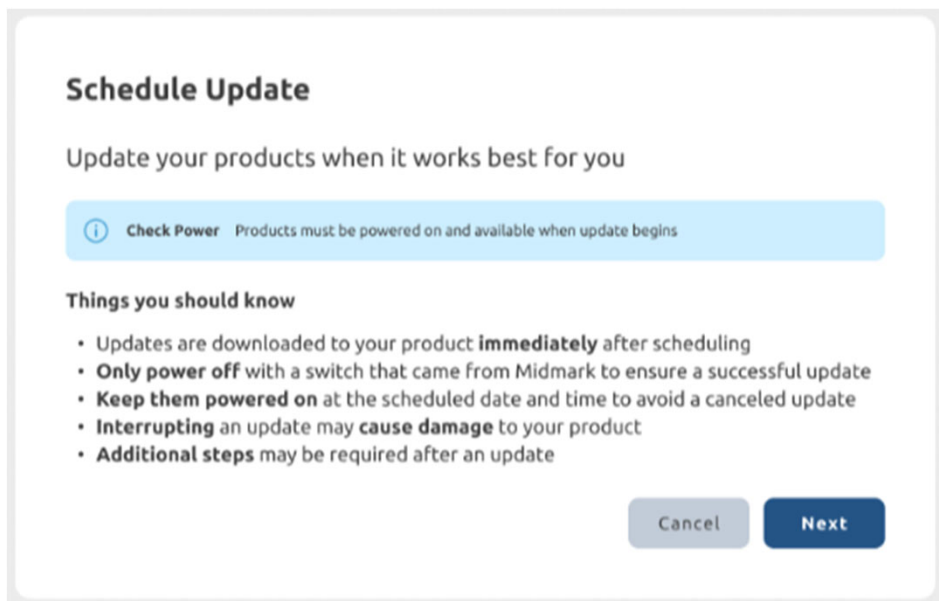

Confirm scheduled update:

