

# Maximize Handpiece Longevity with Essential Maintenance



Take optimal care of your handpieces by performing 3 essential processes after each use:



## 1. CLEANING

Clear debris and disinfect internal handpiece lines to reduce the risk of buildup.



## 2. LUBRICATION

Lubricate handpieces to reduce bearing friction, protect parts from the heat of sterilization, and prevent corrosion and rust on internal components.\*



## 3. STERILIZATION

Sterilize your handpieces in an easy-to-use reliable sterilizer.

## Simplify handpiece maintenance by pairing solutions from Midmark® and Bien-Air®.

Clean and lubricate handpieces more precisely and efficiently with the Lubricare 2® Handpiece Care and Maintenance System from Bien-Air. The clean air blow system ensures thorough cleaning and lubrication inside your handpieces and consistent, repeatable high standards without the variability that may occur when performing these processes manually.




### DID YOU KNOW?

Bien-Air's Spraynet® cleaning solution can reduce 99.99% of the microbial load inside your handpieces.



Simplify sterilization by pairing the easy-to-use Midmark M9® and M11® Steam Sterilizers with Bien-Air electric handpieces. Bien-Air handpiece cycle requirements align directly with the handpiece cycle preset on Midmark sterilizers with no cycle customization needed.

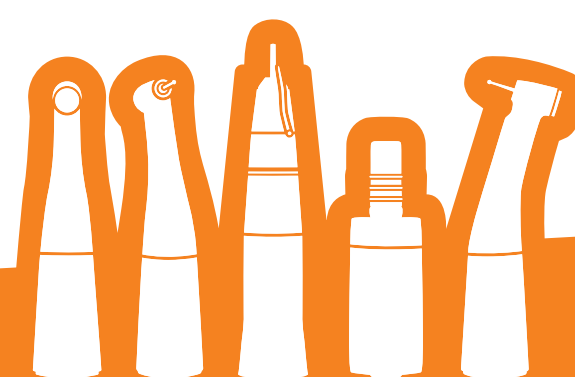

### DID YOU KNOW?

Each handpiece brand has its own instructions for use (IFUs), including parameters for sterilization. So, if you use handpieces from a variety of brands, you'll need to run separate sterilization cycles to meet each brand's IFUs.

## Simplify your handpieces.

Just two Bien-Air contra-angle handpieces can perform a broad range of clinical procedures compared to the competition, which can require twice as many attachments for the same procedures.

**DID YOU KNOW?** Bien-Air handpieces contain the only sterilizable anti-retraction valve on the market.



## Maximize efficiency.

Select a Midmark Procenter unit with Bien-Air handpieces already factory installed.

Regular cleaning, lubrication and sterilization are essential maintenance for the longevity of your handpieces. Simplify the process with solutions from Midmark and Bien-Air.



Source:  
\*<https://www.trutouchrepairs.com/how-to-lubricate-your-dental-handpieces-tips-to-increase-handpiece-life/>  
Bien-Air, Lubricare 2 and Spraynet are not trademarks of Midmark Corporation and are the sole property of Bien-Air SA.  
© 2023 Midmark Corporation, Versailles, Ohio USA  
MKT-00528

