



MEDICAL

# Sterilizers



# Balancing patient care with instrument processing challenges



Instrument processing is a highly regulated, time-consuming and labor-intensive task. This may make it challenging for staff to choose between prioritizing patient experience or properly sterilizing instruments. That's why we designed **Midmark® Steam Sterilizers** to help make instrument processing and adherence to clinical best practices as easy and as automated as possible.





# Midmark® Steam Sterilizers

WHEN YOU INVEST IN MIDMARK STERILIZERS, YOU CAN BE CONFIDENT YOU ARE GETTING THE BEST—OUR STERILIZERS ARE THE MARKET LEADERS YEAR AFTER YEAR.

Midmark Steam Sterilizers are designed with intuitive instructions that can simplify compliance, cycle operation, and performing and recording device care. This can help to **reduce transcription and documentation errors which could account for 56% of sterilization errors.**<sup>1</sup> Automated mechanical monitoring and chemical indicator reporting can save time, helping simplify staff training and giving them confidence to process instruments.

**INTUITIVE USER INTERFACE:**

Navigate cycle setup and other processes in your preferred language—English, French or Spanish—with a clear, fingerprint-resistant 5-inch touchscreen you can utilize while wearing gloves. Choose a style—light or dark mode.



SCAN TO  
VIEW VIDEO.



**PROTECT YOUR INVESTMENT**  
Register your warranty today to receive prompt support when you need it and opt in to receive future notifications for available software updates.



**01 MIDMARK M11 STEAM STERILIZER**

The 11" x 18" chamber makes this one of the largest of any standard countertop sterilizers on the market.

**02 MIDMARK M9 STEAM STERILIZER**

Pack all the reliable sterilizing power you need into a compact unit perfect for areas with limited space.

Midmark Sterilizers are designed to help make instrument processing as easy and as automated as possible.



**WRAPPED**  
270°F, 4 MIN



**WRAPPED**  
275°F, 3 MIN



**DELICATE**  
250°F, 30 MIN

Simplify clinical workflow with dynamic-air-removal steam sterilizers that provide unique preset cycles to meet both the CDC and AAMI sterilization cycle recommendations for wrapped instruments.<sup>2</sup>




**UNWRAPPED**  
270°F, 3 MIN




**5 CUSTOM CYCLES\***  
250°F–275°F,  
3 MIN–45 MIN


Determine cycle status—cycle in progress, cycle complete and cycle error—from a distance with a progressing color-coded LED light bar, large cycle-countdown clock and audible cues.

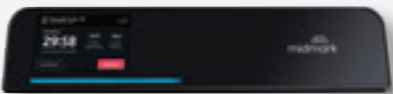

CYCLE IN PROGRESS

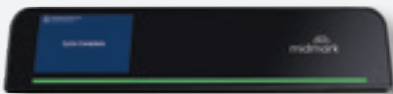

CYCLE COMPLETE

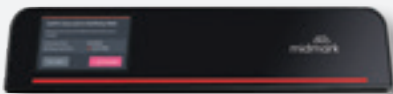

CYCLE ERROR

\* These cycles are not FDA cleared and validation of sterility of items processed using a custom cycle is the responsibility of the user.



**STREAMLINED COMPLIANCE RECORDKEEPING:**  
Stay audit-ready with device reminders and notifications, user authentication, unlimited storage of routine care events for the life of the sterilizer, and automated cycle recordkeeping.



SCAN TO  
VIEW VIDEO.



**DURABILITY:** Midmark sterilizers have been designed for durability, including an increased device life of 25,000 cycles thanks to a completely reengineered chamber. This can reduce the frequency of required maintenance by users and certified technicians by more than half.



SCAN TO  
VIEW VIDEO.

# ACCESSORIES



**9A307001 COOL HAND TOOL**  
(Tray and Cassette Removal Tool)



**9A694001 AUTOFILL 115V**  
(9A694002 Auto-Fill 230V available)



**9A698001 USB KIT**  
Download cycle records including time, temperature and pressure. Includes USB drive, USB cable and PC board (technician installation required).



**9A703001 VERTICAL CASSETTE RACK**  
(M11 Steam Sterilizer models only)



**9A586002 VISTACOOl™ UNIT**

# SERVICES + SUPPORT

**TECHNICAL SPECIFICATIONS**  
View technical specifications for Midmark M9® and M11® Steam Sterilizers.



SCAN TO  
VIEW SPECS.

**MIDMARK DELIVERY SERVICES**  
Coordination, delivery and setup right on time, every time.



SCAN TO  
VIEW YOUR  
OPTIONS.

**MIDMARK LIVE DESIGN**  
Our in-house design experts are ready to help you every step of the way.



SCAN TO  
VIEW VIDEO.

**CLINICAL EDUCATION**  
Trust in our expert educators to assist you in implementing your Midmark devices.



SCAN FOR  
TRAINING  
OPTIONS.

**REPAIR + SERVICE**  
Explore available support and service plans so you can obtain the level of coverage you need to keep your medical equipment and devices operational. This includes periodic maintenance and periodic inspection plans, as well as extended warranty plans.



SCAN FOR  
SUPPORT.

# SOURCES

- 1 <https://www.ncbi.nlm.nih.gov/pmc/articles/PMC6252925>
- 2 <https://www.fda.gov/media/80265/download>



Designing better care.®



USGBC® and the related logo are trademarks owned by the U.S. Green Building Council® and are used with permission.

Midmark is an ISO 13485 and ISO 9001 Certified Company. Certain products are not included. See the complete list at: [midmark.com/ISO](http://midmark.com/ISO)

For more information, contact your authorized Midmark distributor or call: 1.800.MIDMARK

Outside the USA call: 1.937.526.3662 or visit our website: [midmark.com](http://midmark.com)

VistaCool™ Direct-to-Drain System for Autoclave Wastewater is manufactured by Crosstex International, Inc., for distribution by Midmark Corporation, Versailles, OH.

VistaCool™ is a trademark of Crosstex International, Inc., a Cantel Medical Company, Hauppauge, NY.

© 2025 Midmark Corporation, Versailles, Ohio USA  
Products subject to improvement changes without notice.

007-10478-00 Rev. C1 (6/25)

