



Install Handpieces

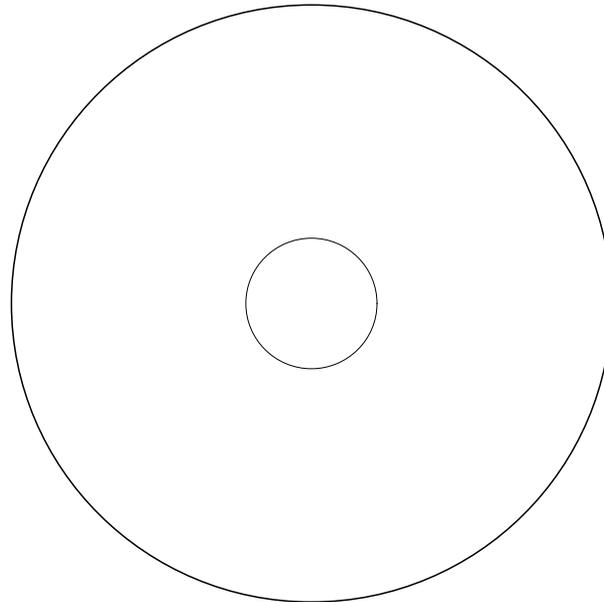

To activate warranty:

Insert CD, then select Product Registration from the menu.

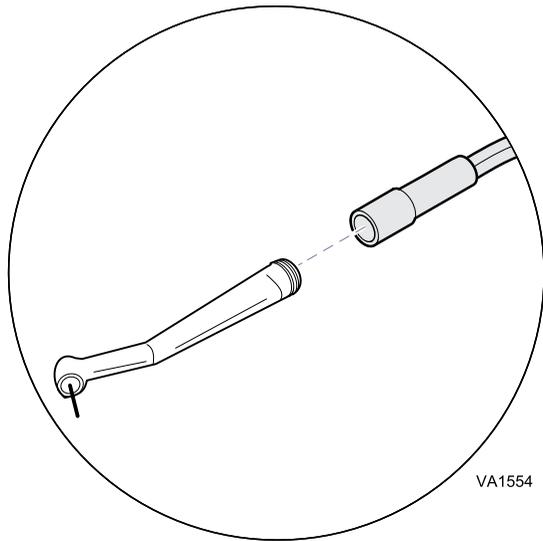

Equipment Alert

Complete manufactures instructions for "Infection Control / Sterilization found in the documentation included with the handpieces. Failure to do this will cause premature failure of handpieces.

Maintenance Kits

 **Water Bottle Gasket**
Order #: 014-0176-52

 **Plastic Muffler Filter**
Order#: 002-1149-00

VA1443

Calling for Service

If service is required, contact your Midmark dealer.

To contact Midmark directly:

*1-800-Midmark (1-800-643-6275)
8:00am until 5:00 pm Monday through Friday (EST)*

Midmark Corporation
60 Vista Drive
Versailles, OH 45380-0286
Phone: 937-526-3662
Fax: 937-526-5542
www.midmark.com



Because we care.



Operating & Care Guide VetPro® 5000

Mobile Dental Delivery System

Model Number:
Serial Number:

Dealer Information:

Purchase Date:

003-1949-00 Rev E

The operation of the VetPro® 5000 Wall/ Mobile Dental Delivery System is restricted to use by veterinary professionals.


This guide outlines basic care and maintenance of the VetPro® 5000. The enclosed CD provides additional instructions on operation and care. If service is required, contact your authorized Midmark dealer.

© Midmark Corporation 2008

To Flush Delivery System...


- A) Remove all handpieces that use water.
- B) Open water needle valve.
- C) Turn foot control toggle switch to WET.
- D) Hold end of tubing over a container and press foot control for 30 seconds. Then press and hold flush button for 30 seconds. Repeat (step D) for all handpieces.
- E) Press and hold syringe buttons (air & water) for 30 seconds.
- F) Install newly disinfected handpieces.
- G) Refill water bottle.

Note: When performing "Flush" procedure at the end of the day. Refer to: User's Guide 003-1950-00 on CD or www.Midmark.com for procedure.)

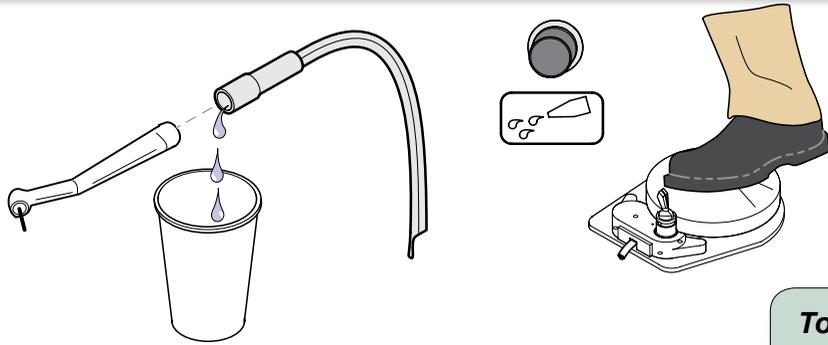

At the beginning of each work day...

- Fill water bottles with fresh distilled water.
- Flush handpieces.

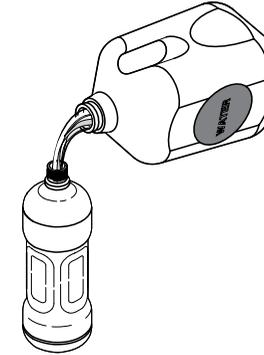

After Each Procedure

A. Clean.

Handpieces and syringe tip should be changed or disinfected between each patient according to manufacturer's instructions.

B. Flush Delivery System.

C. Fill Water Bottle.

Daily

A. Clean System

- Disinfect handpieces according to manufacturer's instructions.
- Wipe down delivery equipment with disinfectant.

B. Bleach Flush Delivery System

(End of Day Only - Refer to: User's Guide 003-1902-00 on CD or www.Documark.com for procedure.)

C. Drain Air Storage Tank


Periodically

A. Check air/oil separator fluid level.

- Change Gauze when saturated. Use 2 x 2 Gauze pads.


To clean the air / oil separator cup...

- A) Turn master On/Off switch Off.
- A) Unsnap cup.
- B) Dispose of saturated gauze.
- C) Disinfect cup.
- D) Install new gauze, then reinstall cup.

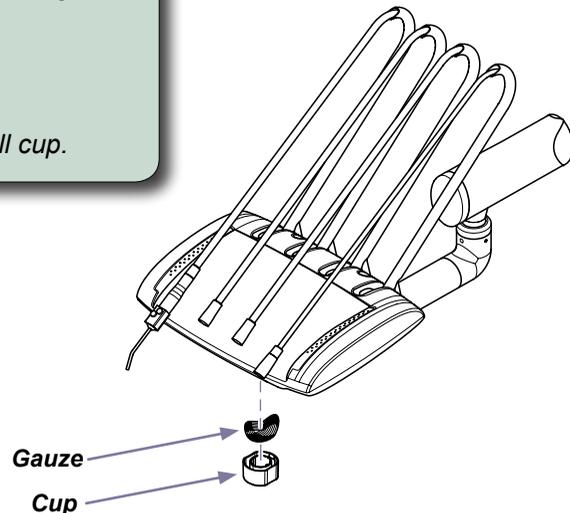

To Drain Air Storage Tank..

Hold absorbant cloth under drain. Open drain valve by turning Counter-clockwise. Tighten valve after tank empties of air and condensation.

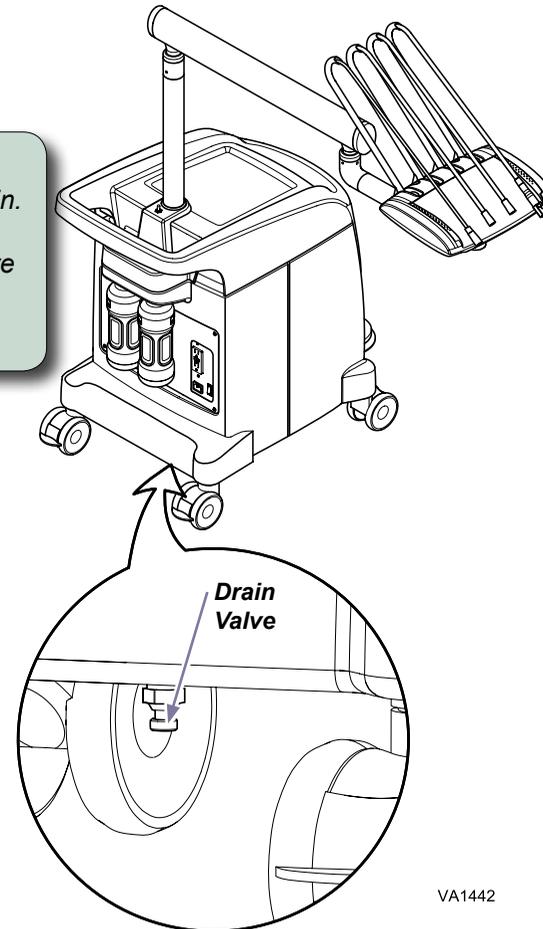

VA1442