

Operator Shutdown Procedures

If you need to shut down your operator for an extended period of time, please follow these simple steps.

Dental Chairs

- STEP 1:** Place the chair back into down position.
- STEP 2:** Lower the chair base to the lowest position.

Doctor's Delivery Units

- STEP 1:** Follow the end-of-day cleaning process as shown in the user manuals.
- STEP 2:** Remove the water supply from the unit (either city or bottle water system).
- STEP 3:** Purge all hand pieces and syringes until no fluid is visible.
- STEP 4:** Leave the water bottle off the unit during storage time.

Assistant's Delivery Units

- STEP 1:** Clean the evacuation lines as described by the vacuum system manufacturer's specifications.
- STEP 2:** Turn the vacuum off and clean both basket and housing.

Cuspidors

- STEP 1:** Ensure the unit is turned off.
- STEP 2:** Run the cup fill and bowl flush until all water is evacuated.

After the above processes have all been completed, turn off the manual shut-off valves located in the junction box, as shown to the right.

For further technical information and user guides, visit our technical library at midmark.com/technical-library/dental. To find the appropriate user guide, choose the product category and product, then select User Information.

