[English](#)

[Español](#)

[Français](#)

Assembly Instructions

Stools: Models 425 and 427

MODEL 425

MODEL 427

(The above models are shown with optional items attached.)

Introduction

These assembly instructions are intended to assist you in assembling Midmark's Model #425 and #427 Stools. Please follow these instructions to assure proper assembly.

Unpacking

Equipment Alert

To avoid damage, do not use a knife or any other sharp object for opening cartons.

Assembly

Remove the stool from the carton. Depending on the optional items that you purchased, assemble the stool in the following order:

- Optional Lab Height Kit Assembly
- Optional Armrest Assembly
- Optional Backrest Assembly
- Seat Assembly

Optional Lab Height Kit Assembly

Place the base stool on its side. Remove the roller base from the gas cylinder by using a hammer to pound the roller base from the gas cylinder. To prevent marking, use a piece of cardboard or wood over the roller base before striking it. Strike as close to the gas cylinder as possible and rotate to a different leg with each blow. Insert extension tube thru the foot ring. Tighten the knob firmly to prevent the foot ring from rotating around the tube. Insert the gas cylinder into the extension tube (see Figure 1). Finally, insert the extension tube into the roller base.

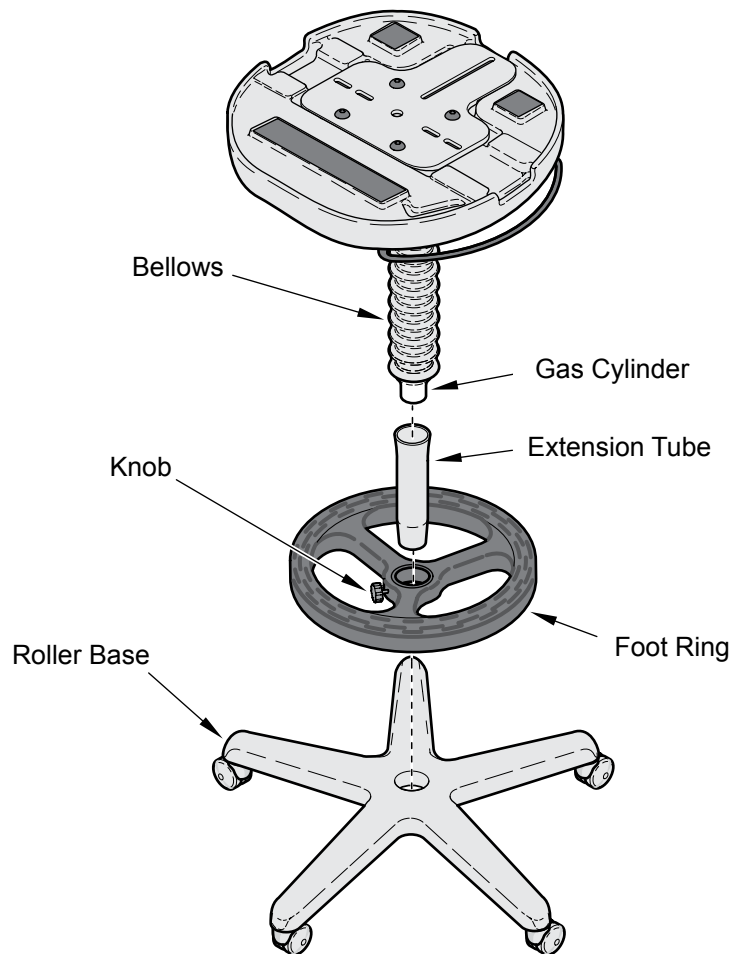
FIGURE 1

MA8374i

NOTE

If you have ordered a Lab Height Kit, please assemble the Lab Height Kit to the base stool first. We suggest that no armrests or backrests be mounted to the stool during the lab height assembly.

NOTE

The Lab Height Kit cannot be used on 427 Foot Release Models.

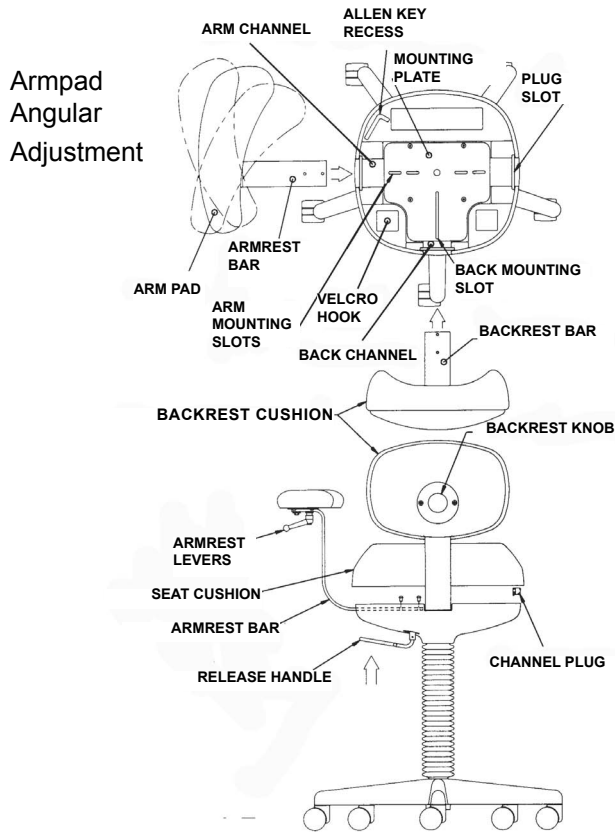
FIGURE 2
Optional Armrest Assembly

FIGURE 3
Insert the armrest bar into the arm channel and under the mounting plate (see Figure 2). Align the two tapped holes in the armrest bar with the two arm mounting slots in the mounting plate. Using the supplied 5/32 Allen Key, drive the two 1/4-20 x 3/8 screws through the mounting plate and into the armrest bar. Repeat the above procedure with the other armrest. After tightening the last screw, place the Allen Key into the Allen Key recess.

Optional Backrest Assembly

FIGURE 4
Insert the backrest bar into the back channel and under the mounting plate (see Figure 2). Align the tapped holes in the backrest bar with the back mounting slot in the mounting plate. Using the supplied 5/32 Allen Key, drive the two 1/4 - 20 x 3/8 screws through the mounting plate and into the backrest bar. Tighten the screws and place the Allen Key into the Allen Key recess.

Seat Assembly

Insert the three channel plugs into the plug slots--if

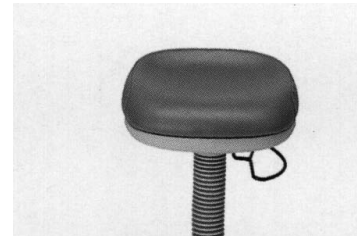
FIGURE 5
needed (see Figure 2). You'll find the channel plugs in the small parts bag.

Remove the seat cushion from its carton. Align the three velcro patches on the bottom of the seat cushion with the three velcro hook patches on the base stool. Then, place the seat cushion onto the base stool. If the cushion isn't centered on the base stool, pull the cushion up and try again.

Adjustments

Height Adjustment of Stool

Lower the 425 stool by grasping the release handle, (see Figure 2) and pull up. This will open the valve in the gas cylinder. The weight of the operator will compress the gas in the cylinder allowing the stool seat to lower. Once the desired height is reached, release the handle. The handle will snap back to its original position locking the cylinder's vertical position.

To raise the stool, grasp the handle and pull up. While holding the handle up, remove the operator's weight from the stool top. By removing the weight from the stool top, the compressed gas in the cylinder can exert enough thrust to force the seat up to the desired position. When the desired position is reached, release the handle.

In units with the foot release (Model 427), the valve in the gas cylinder is opened and closed by the action of the foot pedal instead of the release handle. The valve is opened by pressing down on the pedal and is closed by releasing the pedal.

Backrest Adjustment

The backrest has two adjustments: in-and-out and up-and-down. Adjust in or out by removing the cushion and loosening the two 1/4-20 x 3/8 screws with the supplied 5/32 Allen Key. Position the backrest by pushing or pulling the backrest bar at the base. Tighten the two 1/4-20 x 3/8 screws when the desired position is achieved.

Up-and-down adjustment is obtained by loosening the backrest knob (turning counterclockwise) and lifting or lowering the backrest cushion. When the desired position is reached, tighten the backrest knob (turning clockwise).

Armrest Adjustment

Each armrest has two adjustments: in-and-out and angular. In-and-out adjustment is obtained by removing the cushion and loosening the two 1/4-20 x 3/8 screws using the supplied Allen Key. Position the armrest by pushing or pulling the armrest bar at its base. When the desired position is reached, tighten the two 1/4-20 x 3/8 screws on each arm.

Angular adjustment of the arm pad can be obtained by turning the arm lever a half-turn in. Adjust the arm pad to the desired angular position and turn the arm lever a half-turn out to lock into place.

Cleaning

Upholstery

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on www.midmark.com in the Technical library under the User Information tab for your product.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner. Periodic applications of common furniture wax will ease cleaning, and maintain the finished luster of the surfaces.

Warranty Information

SCOPE OF WARRANTY Midmark Corporation ("Midmark") warrants to the original retail purchaser that it will, at Midmark's option, repair or replace components of the domestic and international medical products manufactured by Midmark (except for components not warranted under "Exclusions") that are defective in material or workmanship under normal use and service. The sole remedy under this limited warranty is the repair or replacement, at Midmark's option, of the applicable components. This limited warranty shall only apply to defects that are reported to Midmark within the applicable warranty period and which are determined to exist upon examination by Midmark. This warranty extends only to the original retail purchaser of a product and is not transferable or assignable. Replacement components or products may be used and/or refurbished components or products, provided they are of like quality and specifications as new components or products.

Midmark warrants to the original retail purchaser that during the applicable warranty period it will repair or replace software contained within the products manufactured by Midmark (except for those not warranted under "Exclusions") if: (1) the media on which the software is furnished exhibits defects in material or workmanship under normal use; or (2) the software does not substantially conform to its published specifications.

APPLICABLE WARRANTY PERIOD The applicable warranty period, measured from the date of invoice to the original retail purchaser of the product and shall be one (1) year for all warranted products and components.

OBTAINING WARRANTY SERVICE Warranty service must be obtained through either Midmark or an authorized dealer in the Midmark product line for which warranty service is requested. Midmark may be contacted for warranty service inquiries or issues via email at www.midmark.com; by phone at 1-800-MIDMARK or by mail to Midmark Corporation, 60 Vista Drive, Versailles, Ohio 45380. It is the retail purchaser's obligation to arrange for delivery of a product to Midmark or one of its authorized dealers for warranty service, which delivery shall be at retail purchaser's expense. It is also the retail purchaser's obligation to comply with the warranty service instructions provided either by Midmark or its authorized dealer. The retail purchaser must provide Midmark with completed warranty registration information within thirty (30) days after purchase in order to obtain the benefits of this limited warranty.

EXCLUSIONS: This limited warranty does not cover and Midmark shall not be liable for the following:

- (1) Defects, damage or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, negligent storage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- (2) Products which are not installed, used, and properly cleaned and maintained as required or recommended in the Midmark "Installation" and/or "Installation/Operation Manual" for the applicable product, including the specified structural and operational environmental conditions and electrical requirements;
- (3) Products considered to be of a consumable or sterile nature;
- (4) Accessories or parts not manufactured by Midmark;
- (5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing in advance by Midmark;
- (6) Costs and expenses of routine maintenance and cleaning; and
- (7) Representations and warranties made by any person or entity other than Midmark.
- (8) Matching of color, grain or texture except to commercially acceptable standards;
- (9) Changes in color caused by natural or artificial light;
- (10) Custom manufactured products;
- (11) Alterations or modifications to the product by any person or entity other than Midmark; and
- (12) Products that would otherwise be covered under this limited warranty, but are acquired: (i) from a person or entity that is not Midmark or one of its authorized dealers; or (ii) from a Midmark dealer that is not authorized to sell the product at issue in the geographic territory where the purchaser is located, or is not authorized to sell the product at issue within the medical market.

SOFTWARE; WITH RESPECT TO SOFTWARE THAT IS A PRODUCT OR COMPONENT THEREOF, MIDMARK DOES NOT WARRANT THAT THE SOFTWARE: (1) IS ERROR FREE; (2) CAN BE USED WITHOUT PROBLEMS OR INTERRUPTIONS; OR (3) IS FREE FROM VULNERABILITY TO INTRUSION OR ATTACK BY VIRUSES OR OTHER METHODS.

EXCLUSIVE REMEDY; CONSEQUENTIAL DAMAGES DISCLAIMER MIDMARK'S ONLY OBLIGATION UNDER THIS LIMITED WARRANTY IS THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS. MIDMARK SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES OR DELAYS, INCLUDING, BUT NOT LIMITED TO, DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, LOSS OF DATA, DOWNTIME, COVER AND EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS. THIS DISCLAIMER SHALL SURVIVE ANY FAILURE OR ASSERTED FAILURE OF THE ESSENTIAL PURPOSE OF THIS LIMITED WARRANTY OR ITS REMEDIES SPECIFIED HEREIN. WARRANTY DISCLAIMER THIS WARRANTY IS MIDMARK'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. MIDMARK MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS.

STATUTE OF LIMITATIONS No action may be brought against Midmark for breach of this limited warranty, an implied warranty, if any, or for any other claim arising out of or relating to the products, more than ninety (90) days following expiration of the limited warranty period.

NO AUTHORIZATION No person or firm is authorized to create or approve for Midmark any other obligation or liability in connection with the products.

Midmark Corporation
60 Vista Drive
Versailles, OH 45380-0286
1-800-MIDMARK
Fax 877-249-1793

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