


Thermometer Kit

002-0242-00

M7 Sterilizer

Kit Components:

	Qty
Thermometer	1
Sleeve	1
Male Connector (3/16" x 1/8" NPT)	1
Compression Nut (1/4")	1
Installation Instruction	1

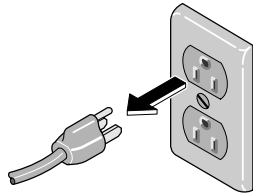

Warning

Disconnect Power and allow unit to cool before removing parts.

Installation


Step 2: Orient gauge face properly. Tighten panel nut.

Installation

Step 3: Slide rubber sleeve back against fitting. Tighten compression nut. (*Do not overtighten.*)

Removal

Step 1: Loosen panel nut. Slide retainer bracket back.

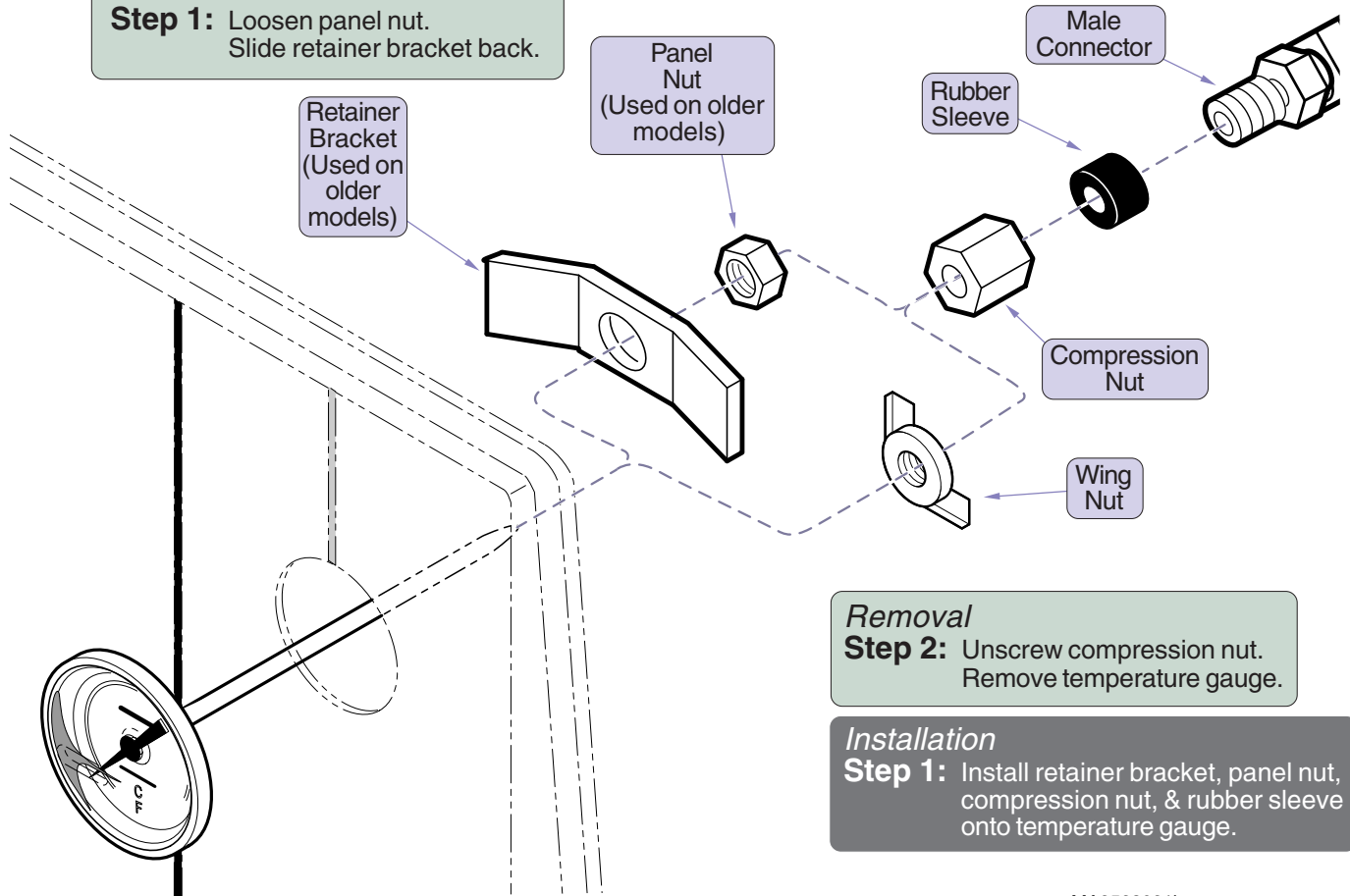

Removal

Step 2: Unscrew compression nut. Remove temperature gauge.

Installation

Step 1: Install retainer bracket, panel nut, compression nut, & rubber sleeve onto temperature gauge.

MA2523021i