

9A77001 ADJUSTABLE IV ASSEMBLY

KIT COMPONENTS

Part Number	Description	Qty.
9A77001	Adjustable IV Pole Assembly.....	1
003-0934-00	Installation / Operation Instructions	1

REQUIRED TOOLS

None

INTRODUCTION


To provide secure support for fluid for an IV infusion. Must be mounted on base rail of exam table or side rail of stretcher.



WARNING

Only mount 9A77001 on base rail of exam table or side rail of stretcher. Assure table or stretcher is clear of any obstacles that may interfere with operation of IV Pole. IV Pole may interfere with 8 ft. (20.3 cm) ceiling. Always support IV Pole assembly when changing positions on it.


INSTALLATION

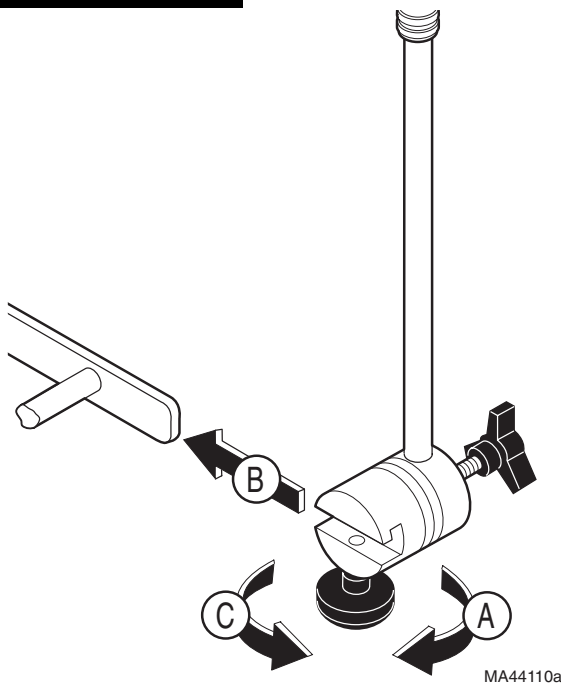

MA44110a

- Loosen lower knob on swivel block
- Slide swivel block onto exam table base rail or stretcher side rail.
- Position at desired location and tighten knob.


OPERATION (ROTATIONAL POSITIONING)

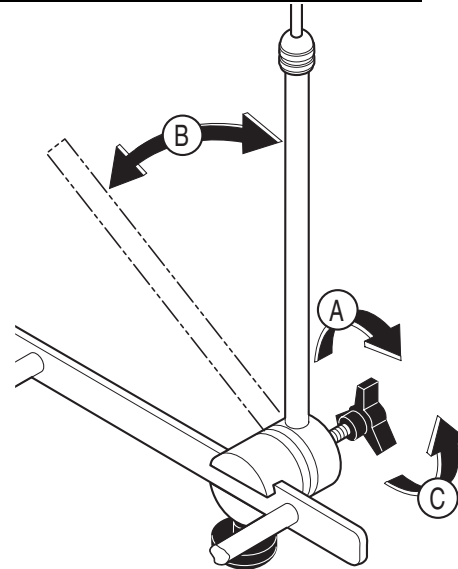

MA44110b

- Loosen knob.
- Rotate IV pole to desired position.
- Tighten knob.

OPERATION (HEIGHT ADJUSTMENT)

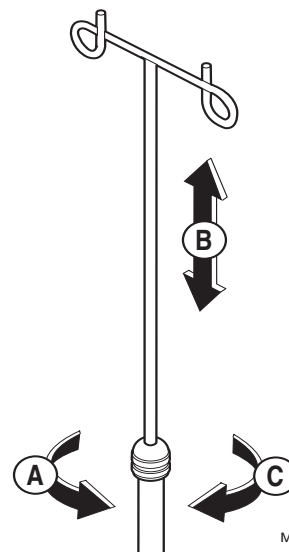

MA44110c

- Support top IV rod, loosen ferrule.
- Raise or lower top IV rod to desired position.
- Tighten ferrule.


MIDMARK[®]

Midmark Corporation, Versailles, Ohio 45380 U.S.A.
(937) 526-3662 FAX (937) 526-5542

003-0934-00 Rev. B (7/08)

I.V. Pole

Order 
 Medical and
 Animal Health
 Parts Online

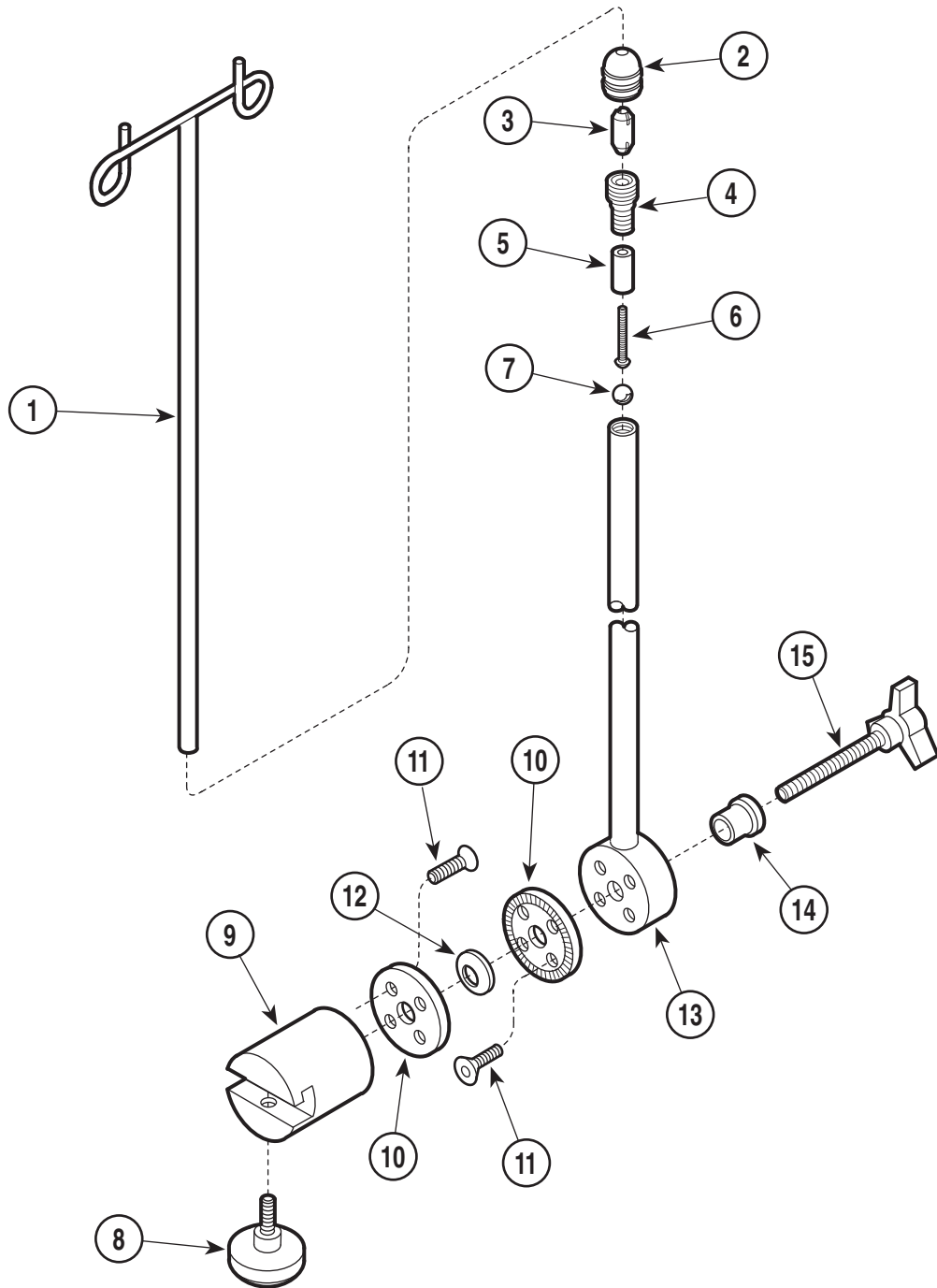

MA222900

Item	Part No.	Description	Qty.	Item	Part No.	Description	Qty.
	9A77001	I.V. Pole Assembly (Includes Items 1 thru 15)	1	7	• 553-0114-02	• Elastomer Spring	1
1	• 530-0326-00	• Chrome Plated I.V. Rod	1	8	• 016-0175-00	• Knob	1
2	• K101037	• Ferrule	1	9	• 057-0180-00	• Swivel Block	1
3	• K101040	• Bushing	1	10	• 057-0084-00	• Serrated Ring	2
4	• 557-0189-00	• Threaded Sleeve (Apply Loctite #042-0024-00)	1	11	• 040-0008-27	• Screw	8
5	• 557-0187-00	• Small Stop	1	12	• 025-0021-00	• Spring Washer	6
6	• 040-0250-98	• Screw (Apply Loctite #042-0025-00)	1	13	• 030-0708-00	• Tube Weldment	1
				14	• 053-0114-00	• Nylon Bearing	1
				15	• 016-0310-00	• Knob	1

Always Specify Model & Serial Number