



NOTES:  
 1. MATERIAL: 60# WHITE OFFSET PAPER.  
 2. INK COLOR: BLACK  
 3. PART TO BE SUPPLIED FOLDED; APPROX. 8.25" X 10.75"  
 4. FULL SIZE PRINT ON SHEET 2.

| ITEM                               | QTY      | PART NUMBER                        | DESCRIPTION               |
|------------------------------------|----------|------------------------------------|---------------------------|
| MATERIAL SPEC                      |          |                                    |                           |
| INCH                               |          | INCH (MILLIMETERS)                 |                           |
| UNSPECIFIED DIMENSIONAL TOLERANCES |          | UNSPECIFIED DIMENSIONAL TOLERANCES |                           |
| ANGULAR +/-                        |          | 2.0°                               |                           |
| FRACTIONAL +/-                     |          | 0.1                                |                           |
| DECIMAL .XX +/-                    |          | 0.03                               |                           |
| DECIMAL .XXX +/-                   |          | 0.015                              |                           |
| ECO NO                             | DATE     | CH BY                              | CHANGE                    |
| ECO052440                          | 5/16/17  | KFS                                | RELEASED                  |
| REV                                | DATE     | BY                                 | DESCRIPTION               |
| 1                                  | 04/16/17 | KSCHMITZ                           | TEMPLATE, CUTOUT, QC3R-0X |

  

|             |              |      |          |
|-------------|--------------|------|----------|
| SCALE       | 1:000        | DATE | 04/16/17 |
| PREPARED BY | KSCHMITZ     | DATE | 04/16/17 |
| APPROVED BY |              | DATE |          |
| PART NUMBER | 003-10019-00 | REV  | AA1      |



WIDMARK CORPORATION, VERMILION, OHIO 43080  
 SHEET 1 OF 2

003-10019-00

WALL

BACKSPLASH

BACK EDGE



**QC3R-01 QUICKCLEAN ULTRASONIC CUTOUT INSTRUCTIONS**

1. ENSURE THAT THE MODEL NUMBER REFERENCED ON THIS TEMPLATE AND THE MODEL NUMBER OF THE MIDMARK ULTRASONIC CLEANER MATCH APPROPRIATELY.
2. LOCATE TEMPLATE TO PROPER POSITION (LEFT-RIGHT) ON THE COUNTERTOP. USE THE CENTERLINE ON THE TEMPLATE TO REFERENCE POSITIONING.
3. SLIDE THE TEMPLATE SO THAT THE EDGE MARKED "BACK EDGE" IS FLUSH AGAINST THE BACKSPLASH (ALLOW THE TEMPLATE TO RIDE UP THE CORNER RADIUS), AND THEN TAPE THE TEMPLATE IN PLACE.
4. USING A DRILL AND 1/2" BIT, DRILL THE (4) CORNER HOLES AT THE LOCATIONS SHOWN.
5. REMOVE THE TEMPLATE AND DRAW A STRAIGHT LINE CONNECTING THE OUTSIDE TANGENT POINTS OF EACH 1/2" DIA. HOLE TO REPRESENT THE CUTOUT.
6. APPLY A LAYER OF PAPER TAPE TO THE BOTTOM SURFACE OF A SABER SAW, MAKING CERTAIN THAT NO METAL IS EXPOSED. THIS WILL PREVENT SCRATCHING OF THE COUNTERTOP.
7. USING THE SABER SAW AND A SMOOTH CUT WOOD BLADE (32 TEETH/INCH), FOLLOW AND CUT THE CUTOUT LINE PERIMETER.

FRONT

003-10019-00 REV AA1