



QC6R-01 QUICKCLEAN ULTRASONIC CUTOUT INSTRUCTIONS

1. ENSURE THAT THE MODEL NUMBER REFERENCED ON THIS TEMPLATE AND THE MODEL NUMBER OF THE MIDMARK ULTRASONIC CLEANER MATCH APPROPRIATELY.
2. LOCATE TEMPLATE TO PROPER POSITION (LEFT-RIGHT) ON THE COUNTERTOP. USE THE CENTERLINE ON THE TEMPLATE TO REFERENCE POSITIONING.
3. SLIDE THE TEMPLATE SO THAT THE EDGE MARKED "BACK EDGE" IS FLUSH AGAINST THE BACKSPLASH (ALLOW THE TEMPLATE TO RIDE UP THE CORNER RADIUS), AND THEN TAPE THE TEMPLATE IN PLACE.
4. USING A DRILL AND 1/2" BIT, DRILL THE (4) CORNER HOLES AT THE LOCATIONS SHOWN.
5. REMOVE THE TEMPLATE AND DRAW A STRAIGHT LINE CONNECTING THE OUTSIDE TANGENT POINTS OF EACH 1/2" DIA. HOLE TO REPRESENT THE CUTOUT.
6. APPLY A LAYER OF PAPER TAPE TO THE BOTTOM SURFACE OF A SABER SAW, MAKING CERTAIN THAT NO METAL IS EXPOSED. THIS WILL PREVENT SCRATCHING OF THE COUNTERTOP.
7. USING THE SABER SAW AND A SMOOTH CUT WOOD BLADE (32 TEETH/INCH), FOLLOW AND CUT THE CUTOUT LINE PERIMETER.

NOTES:
 1. MATERIAL: 60# WHITE OFFSET PAPER.
 2. INK COLOR: BLACK
 3. PART TO BE SUPPLIED FOLDED; APPROX. 8.25" X 10.75"
 4. FULL SIZE PRINT ON SHEET 2.

ITEM	QTY	PART NUMBER	DESCRIPTION
MATERIAL SPEC			
INCH (MILLIMETERS)		INCH (MILLIMETERS)	
UNSPECIFIED DIMENSIONAL TOLERANCES:		UNSPECIFIED DIMENSIONAL TOLERANCES:	
ANGULAR +/- 2.0°		SCALE 1.000	
FRACTIONAL +/- 0.1		DATE	
DECIMAL .XX +/- 0.03		PREPARED BY	
DECIMAL .XXX +/- 0.015		DATE	
ECO NO	DATE	CH BY	CHANGE
ECO NO	DATE	CH BY	CHANGE
THIS PRINT AND THE INFORMATION CONTAINED THEREIN ARE PROPERTY OF MIDMARK CORPORATION AND ARE LOANED TO THE USER IN CONFIDENCE AND ARE NOT TO BE REPRODUCED, USED, OR DISCLOSED TO OTHERS WITHOUT THE EXPRESS WRITTEN PERMISSION OF MIDMARK CORPORATION.		COMMERCIAL TOLERANCES SHALL APPLY ON DRG, ROD, WIRE, SHEET, ETC.	
APPROVED BY		DATE	
PART NUMBER		PART NAME	
003-10020-00		TEMPLATE, CUTOUT, QC6R-0X	
REV		REV	
AA1		AA1	

KSCHEMITZ 5/12/2017 12:06:40 PM

WALL

BACKSPLASH

BACK EDGE

QC6R-01 QUICKCLEAN ULTRASONIC CUTOUT INSTRUCTIONS

1. ENSURE THAT THE MODEL NUMBER REFERENCED ON THIS TEMPLATE AND THE MODEL NUMBER OF THE MIDMARK ULTRASONIC CLEANER MATCH APPROPRIATELY.
2. LOCATE TEMPLATE TO PROPER POSITION (LEFT-RIGHT) ON THE COUNTERTOP. USE THE CENTERLINE ON THE TEMPLATE TO REFERENCE POSITIONING.
3. SLIDE THE TEMPLATE SO THAT THE EDGE MARKED "BACK EDGE" IS FLUSH AGAINST THE BACKSPLASH (ALLOW THE TEMPLATE TO RIDE UP THE CORNER RADIUS), AND THEN TAPE THE TEMPLATE IN PLACE.
4. USING A DRILL AND 1/2" BIT, DRILL THE (4) CORNER HOLES AT THE LOCATIONS SHOWN.
5. REMOVE THE TEMPLATE AND DRAW A STRAIGHT LINE CONNECTING THE OUTSIDE TANGENT POINTS OF EACH 1/2" DIA. HOLE TO REPRESENT THE CUTOUT.
6. APPLY A LAYER OF PAPER TAPE TO THE BOTTOM SURFACE OF A SABER SAW, MAKING CERTAIN THAT NO METAL IS EXPOSED. THIS WILL PREVENT SCRATCHING OF THE COUNTERTOP.
7. USING THE SABER SAW AND A SMOOTH CUT WOOD BLADE (32 TEETH/INCH), FOLLOW AND CUT THE CUTOUT LINE PERIMETER.

FRONT

003-10020-00 REV. AA1