

QuickClean™ Ultrasonic Cleaner Recessed Installation Guide

Applies to Models:
 QC3R -01
 QC6R -01
Special Tools:
 Tube of Silicon Caulking

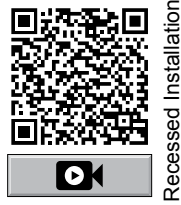Reccessed Installation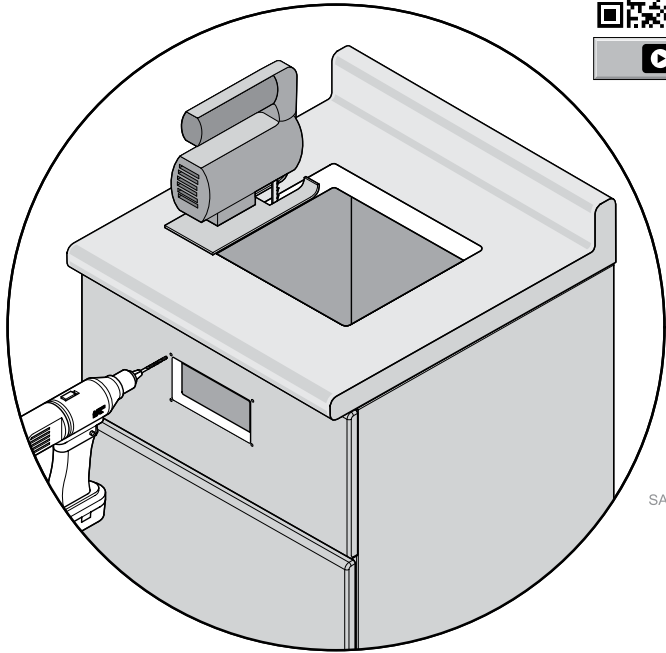

SA1882i

Step 2: Install retaining clips into tab on ultrasonic unit flange.

- A) Use the chart below to select proper tab height to match counter top thickness.
- B) Install one clip in each corner of the ultrasonic, choose only one of the four possible tabs in each corner.
- C) Un-thread screw almost all the way & pull down bottom of the clip.
- D) Rotate bottom of clip towards the side of unit.

Tab Location	Counter Top Thickness
A	Min 3/4" - Max 1 1/2"
B	Min 1" - Max 1 3/4"


Note
 If counter top is less than 3/4" thick a filler strip will have to be added to the bottom of the cutout.

Step 1: Cut out recess openings in countertop and cabinet front.

- A) Use appropriate templates to create outline, cut openings, & pre drill mounting holes on worksurface & cabinet front.


Note: Templates are included with each unit or can be found at Midmark.com.

Equipment Alert
 Be sure to position openings so there is enough clearance inside cabinet for communication cable and power cord connections (front of ultrasonic unit and back of remote panel).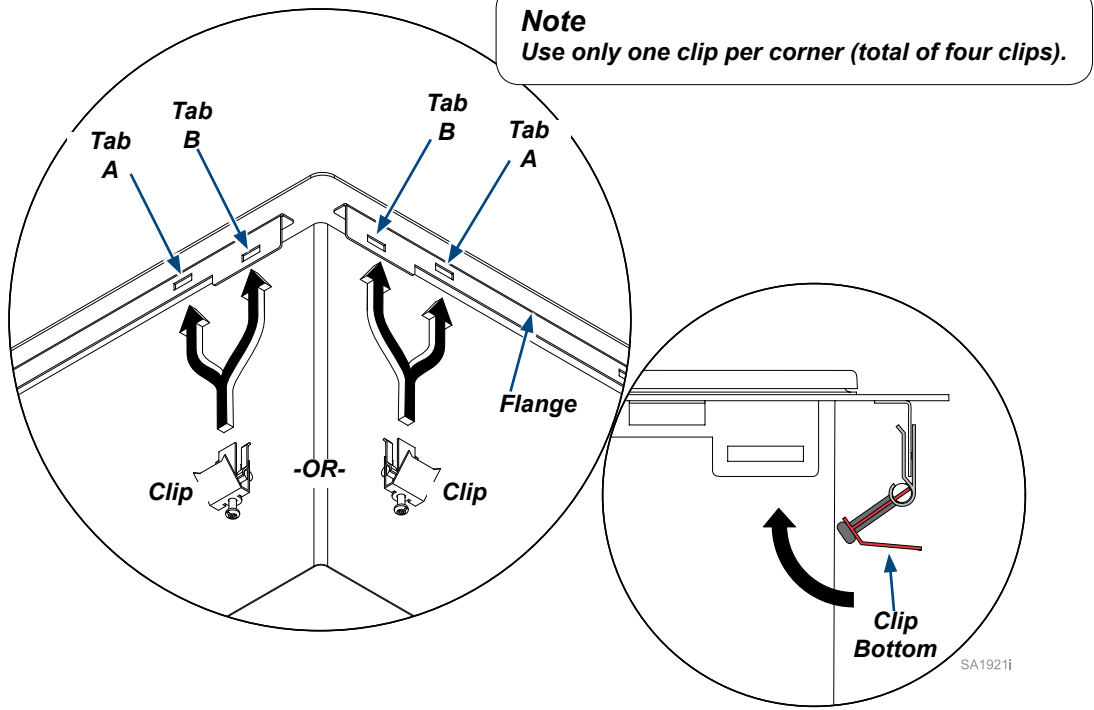

Note
 Use only one clip per corner (total of four clips).

SA1921i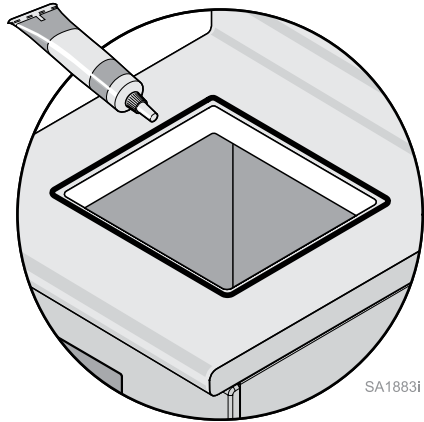


SA1883i


Step 3: Install ultrasonic unit into counter top.

- A) Place a 1/4 bead of silicon caulking around the top edge of the counter top opening.
- B) Position ultrasonic unit with on/off switch, power connection & drain valve located to the front of the cabinet and lower into opening .
- C) Center in counter top, secure ultrasonic unit by rotating bottom of retaining clips down, tighten retaining clip screws.
- D) Clean up excess silicon on top edge.


Note: Be careful not to over tighten screws.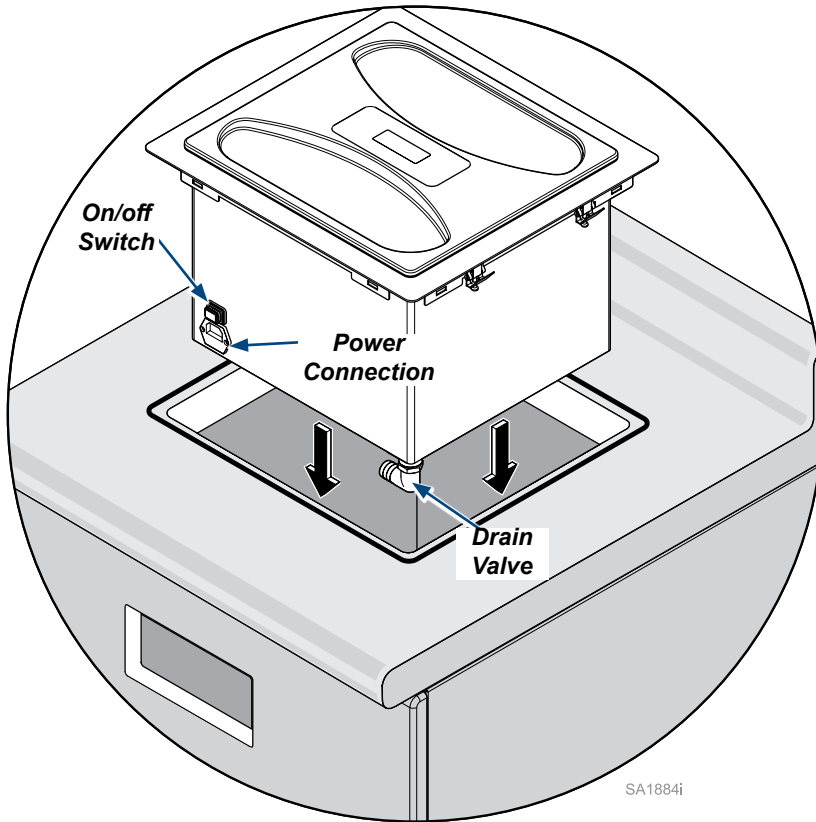

SA1884i

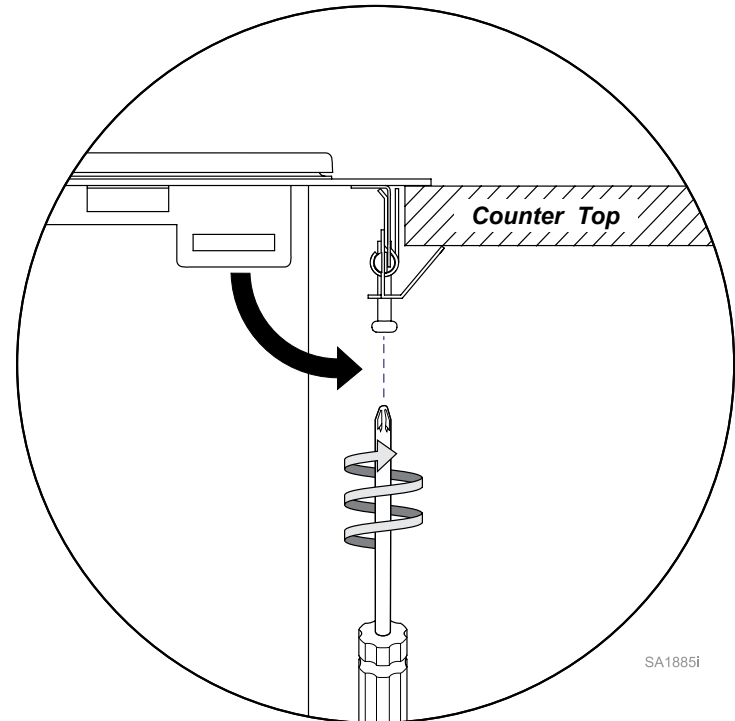SA1885i

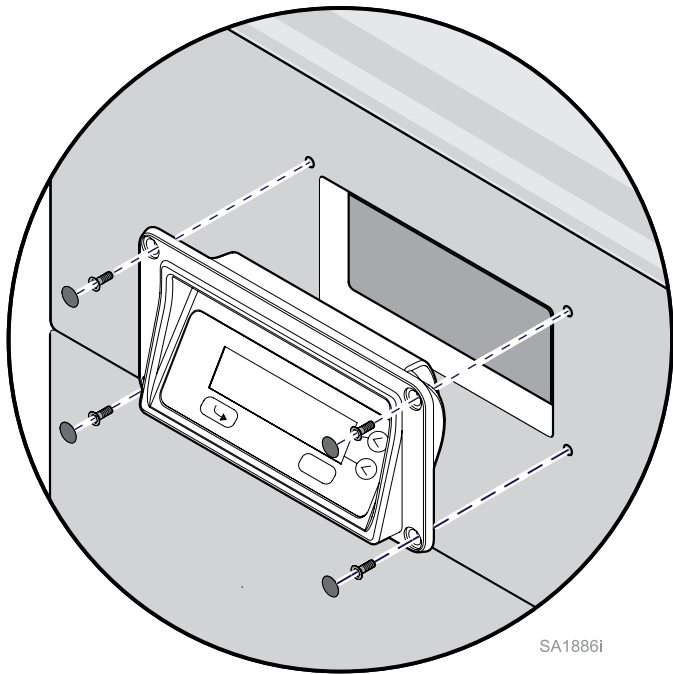SA1886i


Step 4: Install ultrasonic remote panel into cabinet front.

A) Attach remote panel into cabinet using four screws and insert caps provided.

Note: Source fasteners locally if cabinet front is metal.

Note


Remote panel to unit cable connector plug is keyed so it only fits / threads in the sockets one way.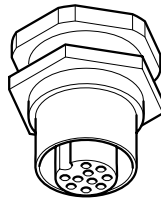

Step 5: Connect the remote panel cable and power cord.

- A) Insert the remote cable plug into socket at the bottom of the unit, turn clockwise direction until fully tightened.
- B) Plug in power cord to unit and electrical outlet.
- C) Thread elbow into drain valve. Push hose onto barb end.

Note: Position any extra cable(s) & hose out of the way to prevent damage.

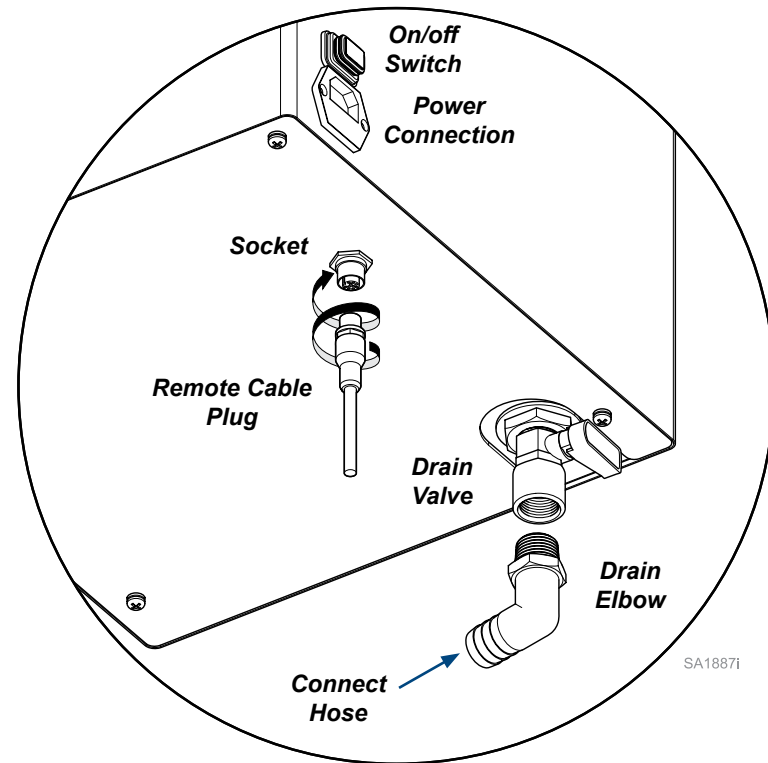SA1887i

Note

If connecting the QuickClean(TM) directly to the drain, it is recommended that the drain connection is installed by a plumber in compliance with local building codes.

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