


To elevate seat section...

Lift seat until pelvic support drops into place.

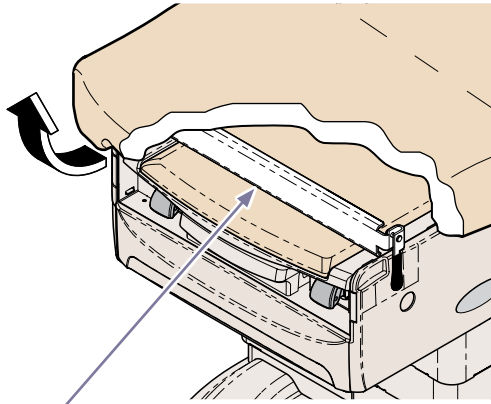


Pelvic Support

MA103071

To return seat section to flat position...

- A) Lift seat section slightly.
- B) Pivot pelvic support handle upward.
- C) Lower seat section.

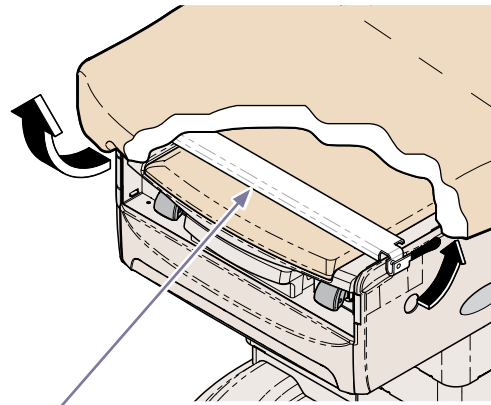

Pelvic Support

MA103081

Authorized Representatives

EC	REP
----	-----

Customers in the EU should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Newmed Srl.
Via Lenin 79/A
Quattro Castella (RE), 42020, Italy
Phone: +39.0522.875.166
Fax: +39.0522.243.096

Customers in Australia should direct all questions, incidents, and complaints to Midmark's Sponsor listed below.

MIDMED PTY LTD
Unit 4/62 Borthwick Avenue
Murarrie, QLD, 4172
Phone: +61 7 3348 9155 / Fax: +61 7 3348 9950

Customers in India should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Midmark India
Kalpataru Point, Unit No. 12, 1st Floor Kamani road
Sion (E), Mumbai, 400022
Tel:+91 22 4915 3000 / Fax:+91 22 4915 3100

Customers in Saudi Arabia should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Attieh Medico Ltd
Abdullah M. Al Khalifi Street
Al Nakheel Dist. II
P.O. Box 116105
Jeddah (East), 21391
Saudi Arabia
Tel: + 966 2 286 4707 / Fax: + 966 2 286 4744

Customers in the United Arab Emirates should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

GULF DRUG LLC.
Al Barsha,
Dubai, UAE
Phone: +971 4 501 4000 / Fax: +971 4 501 4100

Maintenance

Periodically inspect the following:

- Power cord must be free of visible damage.
- Foot control cord must be free of visible damage.
- All fasteners must be present and secure.
- All mechanical functions should operate

Cleaning

Upholstery

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners (see current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities). Follow this with a clear water rinse and thorough drying of material.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on www.midmark.com in the Technical library under the User Information tab for your product.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner. Use of chlorine based disinfectant can cause damage to the hand control.

Calling for Service

If service is required, contact your Midmark dealer. To contact Midmark Medical Technical Support: US 844-856-1230 / Canada 937-526-8585 Monday-Friday 8:00 am until 7:00 pm (EST)

Midmark Corporation
60 Vista Drive
Versailles, OH 45380 USA
1-800-643-6275
1-937-526-3662



Because we care.



Quick Reference Guide 627 (-011) Barrier-Free® Exam Chair

Model Number:
Serial Number:


Dealer Information:

Purchase Date:

Upholstery Color:

003-10032-00 Rev AA3 (7/18/18)

This guide outlines basic operation of the chair. The enclosed User Guide provides additional instructions on operation and care, as well as regulatory compliance information. If service is required, contact your authorized Midmark dealer.


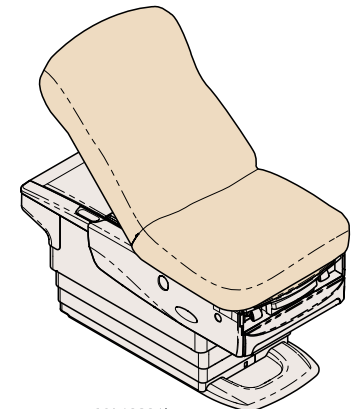


MA102911

Note

The chair automatically stops at its maximum and minimum height.


To move the chair Up or Down...
Press and hold the appropriate button.

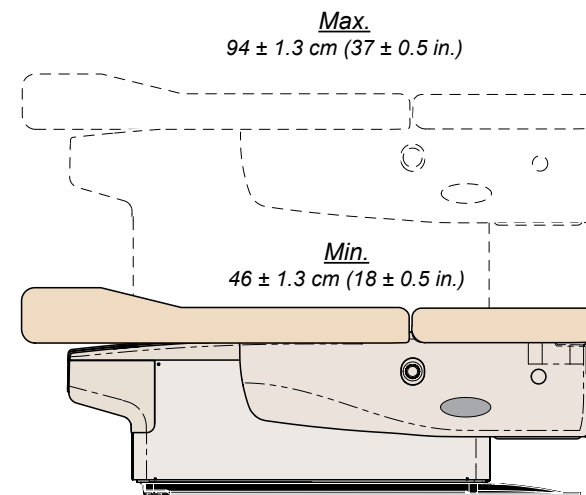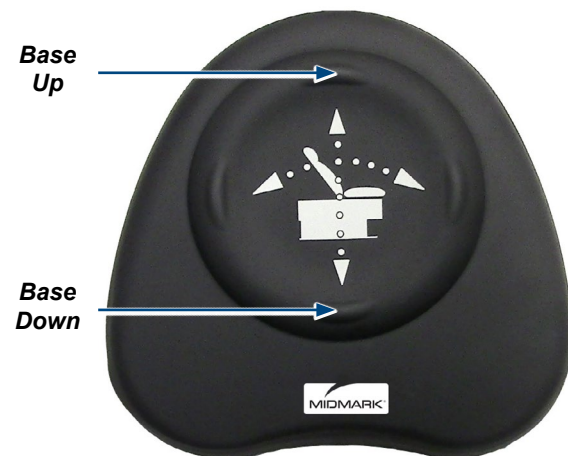

Note

The back section automatically stops at the top and bottom of its travel.

To position the back section...
Press and hold the appropriate button.

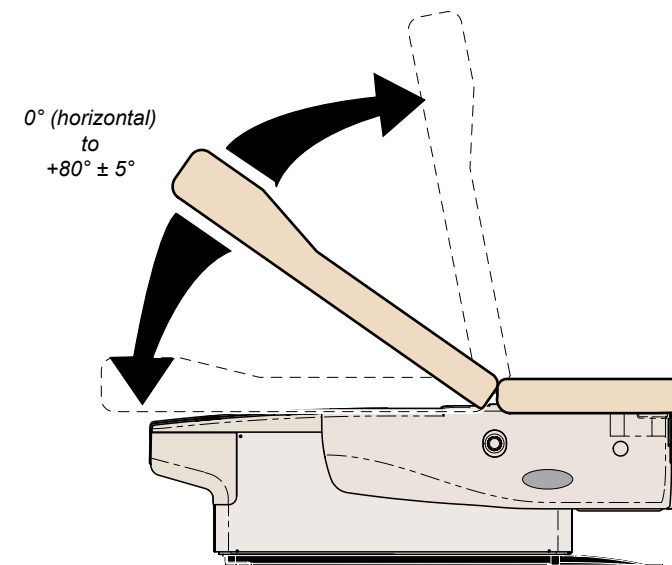

MA10293i