


Connectivity Wiring Setup Kit [002-10063-00]

Applies to Models:
625 (-004)
Language of origin: English

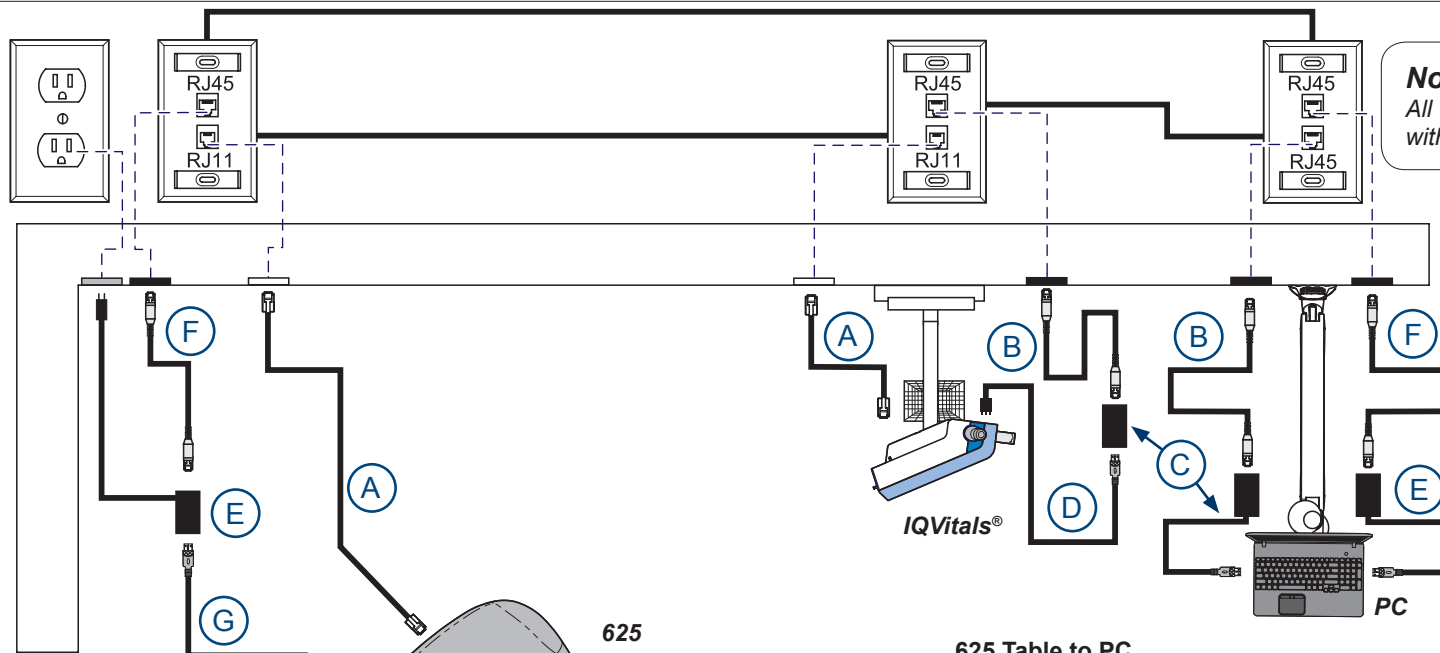

Note
All wall plates and wiring connections within wall are provided by facility.

Note
Connecting adapters E with cables F and G to table and PC are optional, and only required to activate the IQhub®(s) on side of table and fully control the table from the PC.

625 Table connections to IQvitals® and PC


Item	Description	Part Number
A	6-ft Serial Cable (straight)	015-3074-04
B	4-ft Ethernet Patch Cable	015-10124-00
C	USB/Ethernet Adapter 1.1	015-10122-00
D	IQvitals® USB Cable (comes w/ Vitals)	3-009-0016
E	USB/Ethernet Adapter 2.0 w/ power plug	015-10121-00
F	4-ft Ethernet Patch Cable	015-10124-00
G	USB A to B	015-2945-00

625 Table to PC

- For 015-10121-00: After the USB and Ethernet cables are connected to each of the two adaptors, ensure the adaptor with the power source is plugged into the power source before connecting the other adaptor into the PC. If done incorrectly unplug adaptor from computer and replug it in for proper functionality.



Midmark IQvitals® on PC Setup

- Open Midmark IQvitals® Control Panel
- Press the "CONNECT" button to connect the Vitals machine through USB port
- Go to Device Manager on PC
- Under Ports (COM & LPT) note the COM # of the 625 Table
- Exit Device Manager
- In Midmark IQvitals® Control Panel click "Settings"
- Ensure USB is selected in the "Connection Method" box
- In the Scale Settings box click "Settings"
- Under Device ensure "Midmark 625 Scale" is selected in the Scale drop down
- Under Com Port, select the COM # that matches the COM # listed in the device manager
- Go back to the Vitals screen by pressing the "Measurement" Tab
- Click on the image of the Table and your Scale and Table will be connected and ready to use

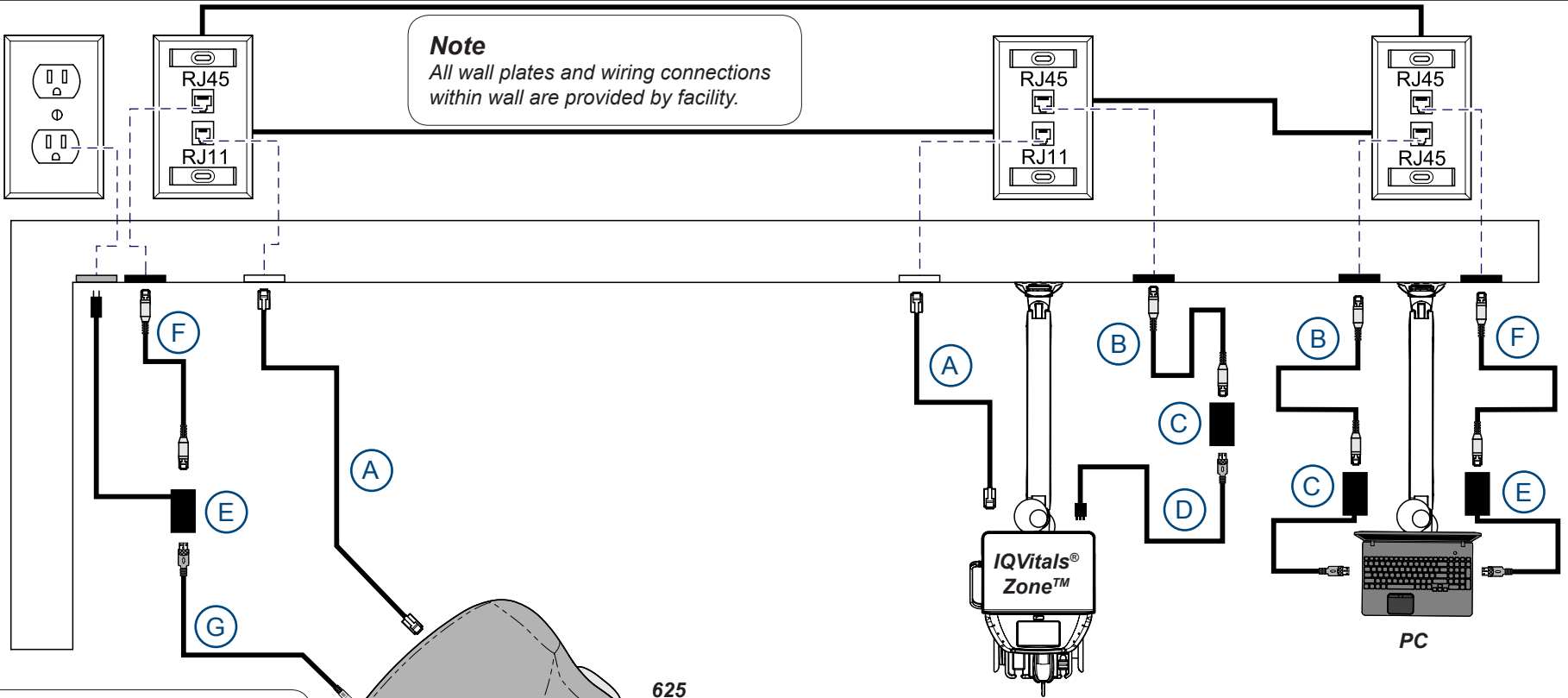

Note
All wall plates and wiring connections within wall are provided by facility.

Note
Connecting adapters E with cables F and G to table and PC are optional, and only required to activate the IQhub® (s) on side of table.

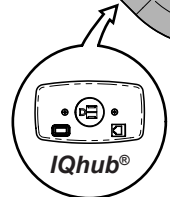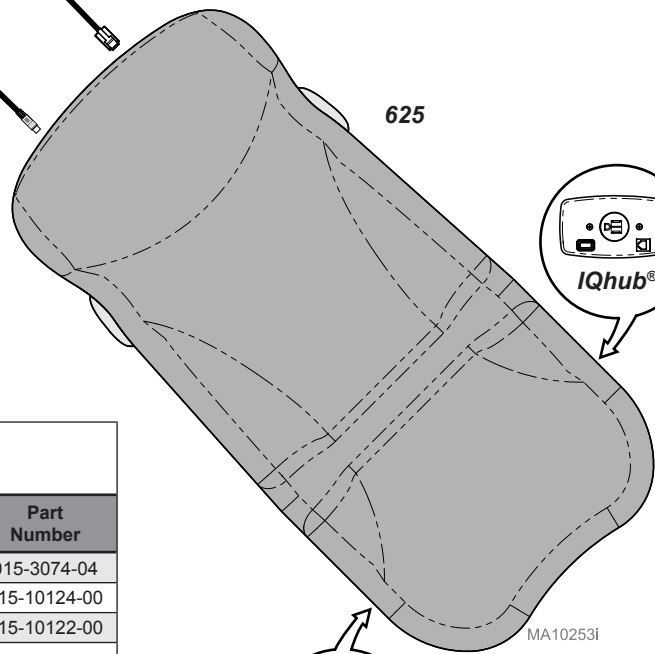

625 Table connections to IQvitals® Zone™ and PC

Item	Description	Part Number
A	6-ft Serial Cable (straight)	015-3074-04
B	4-ft Ethernet Patch Cable	015-10124-00
C	USB/Ethernet Adapter 1.1	015-10122-00
D	IQvitals® Zone™ USB Cable (comes w/ Vitals)	2-200-0004
E	USB/Ethernet Adapter 2.0 w/ power plug	015-10121-00
F	4-ft Ethernet Patch Cable	015-10124-00
G	USB A to B	015-2945-00

Midmark IQvitals® Zone™ on PC Setup

- Open the Midmark IQvitals® Control Panel
- A loading sign should appear over the screen if no device is connected
- Once the USB's are plugged into the computer the loading sign disappears and the device should be connected