


Connectivity Wiring Setup Kit [002-10064-00]


Applies to Models:
626 (-003, -004, -005, & -006)
Language of origin: English

Note
All wall plates and wiring connections within wall are provided by facility.

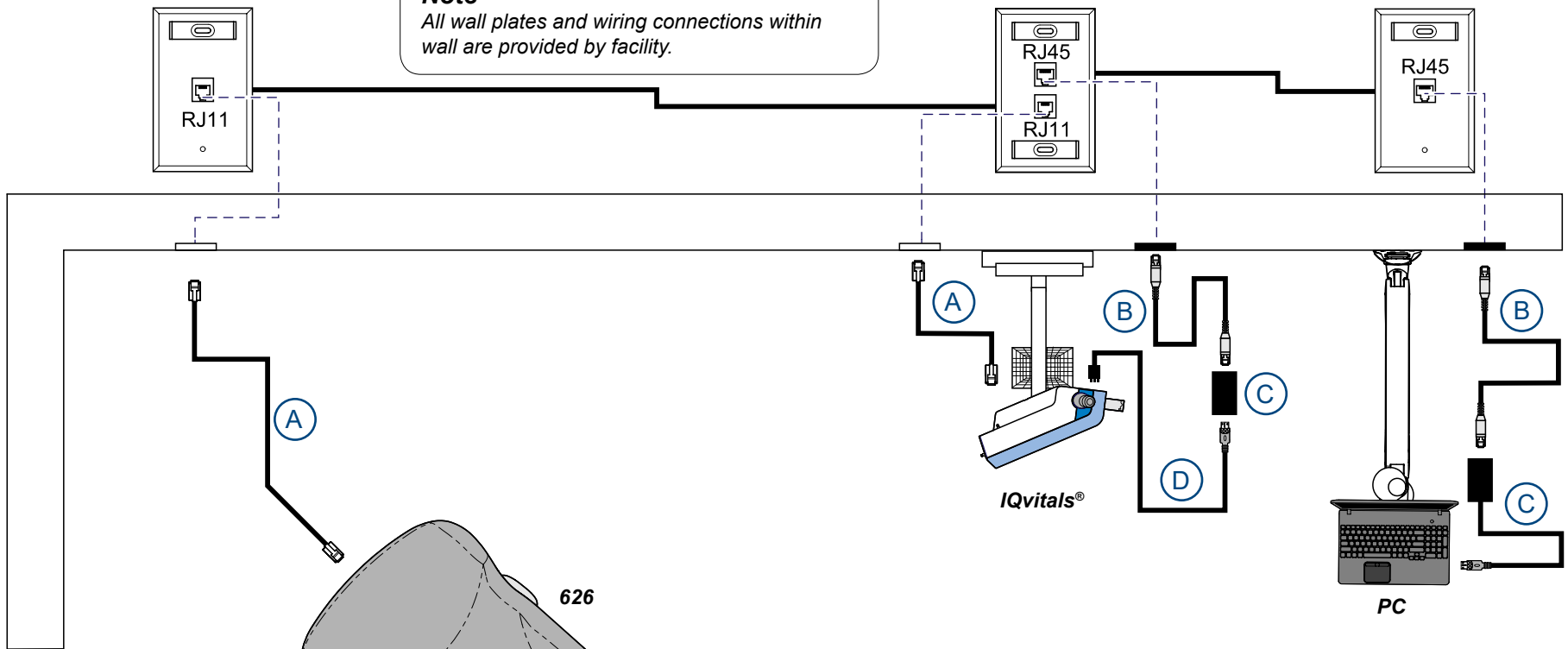

626 Table connections to IQvitals® and PC


Item	Description	Part Number
A	6-ft Serial Cable (straight)	015-3074-04
B	4-ft Ethernet Patch Cable	015-10124-00
C	USB/Ethernet Adapter 1.1	015-10122-00
D	IQvitals® USB Cable (comes w/ Vitals)	3-009-0016

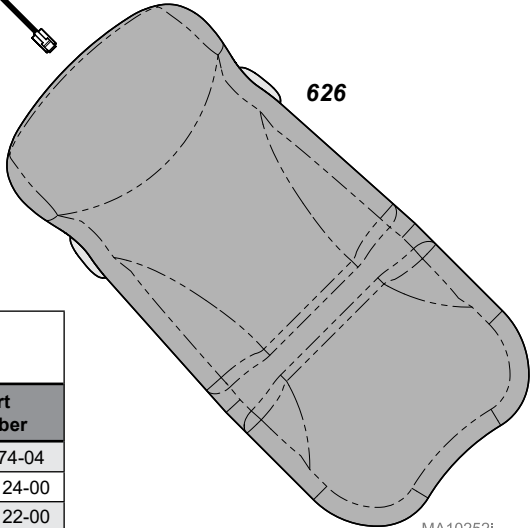

Midmark IQvitals on PC Setup

- Open Midmark IQvitals® Control Panel
- Press the "CONNECT" button to connect the Vitals machine through USB port
- Vitals will then be connected and ready to use

Note
All wall plates and wiring connections within wall are provided by facility.

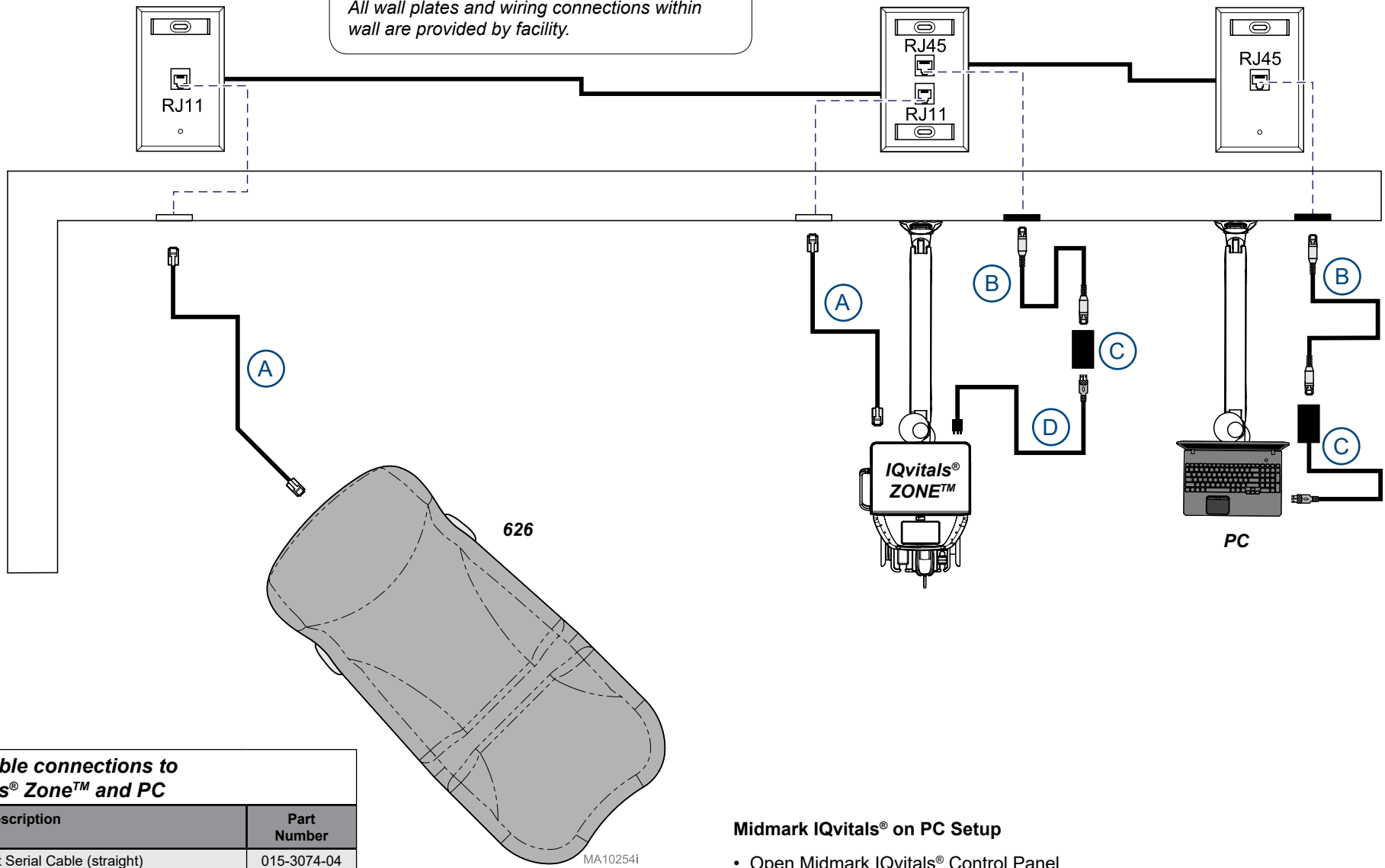

626 Table connections to IQvitals® Zone™ and PC

Item	Description	Part Number
A	6-ft Serial Cable (straight)	015-3074-04
B	4-ft Ethernet Patch Cable	015-10124-00
C	USB/Ethernet Adapter 1.1	015-10122-00
D	IQvitals® Zone™ USB Cable (comes w/ Vitals)	2-200-0004

Midmark IQvitals® on PC Setup

- Open Midmark IQvitals® Control Panel
- A loading sign should appear over the screen if no device is connected
- Once the USB's are plugged into the computer the loading sign disappears and the device should be connected