Important:




- This update applies to all IQvitals Zone models with Main Board firmware version 0.10.9 or higher

- Before using the device, refer to its User Guide (22-78-0002) for complete instruction and safety warnings.

Purpose: Following is the instruction for users to setup and operate the “SPRINT” and “Custom” blood pressure measurement protocols.

A. Settings for SPRINT / Custom protocols:

- Enter Settings menu.
- Enter NIBP tab.
- Turn on “SPRINT” and/or “Custom” options.
- For “Custom” protocol:
 - “Delay start time” (1 min – 5 min): specify time delay between start of protocol to start of cuff inflation.
 - “Interval time” (30, 45, 60 sec): specify time between successive BP readings.
 - “Number of readings” (3, 4, 5): specify number of BP readings acquired prior to completion.
 - “Discard 1st reading”: exclude 1st reading from averaging calculation if checked.
- Select “Apply”, then select “Close”.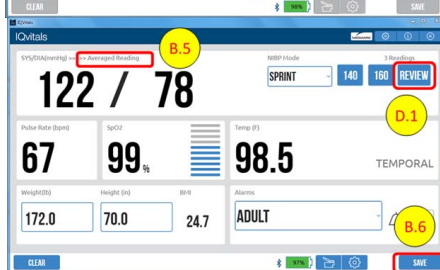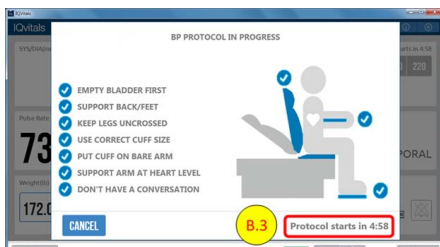
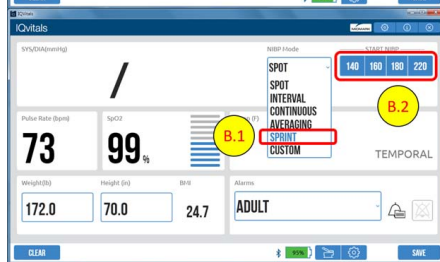


B. Operating SPRINT BP protocol:

- Under NIBP pull-down menu, select “SPRINT”.
- Select one of four inflation pressure buttons to begin BP protocol.
- Protocol begins with a 5-min countdown timer.
- Three BP measurements will be taken at a one-minute interval between readings.
- BP Result is displayed based on the average of all three readings.
- Select “Save” to save result into patient record (only the averaged reading will be saved to patient record).


C. Operating Custom BP protocol:

- Under NIBP pull-down menu, select “Custom”.
- Select one of four inflation pressure buttons to begin BP protocol.
- Protocol begins with a pre-defined countdown timer.
- BP measurements will be taken at a pre-defined interval between readings.
- BP Result is displayed after the pre-defined number of readings has been acquired and based on the average of all those readings.
- The average of all readings might exclude the first reading based on pre-defined settings.
- Select “Save” to save result into patient record (only the averaged reading will be saved to patient record).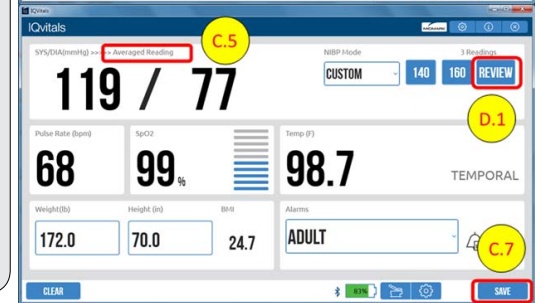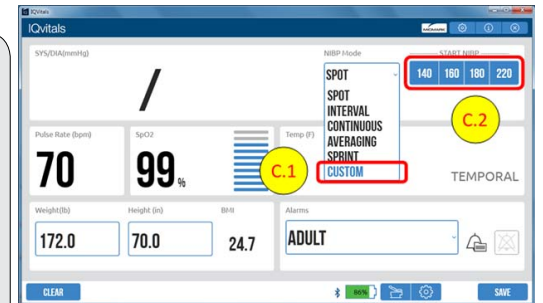

D. Review of SPRINT / Custom results:

- Upon completion of the acquisition protocol, select “Review” to review the list of all acquired BP readings.
- All individual attempts of BP measurement from the current BP protocol will be shown (averaged reading is only shown on the main screen).
- Select “Cancel” to exit review screen.
- Individual readings on the Review Screen will be discarded when any of the following selections is made on the main screen: “Clear”, “Save”, any other BP modes, or device power off.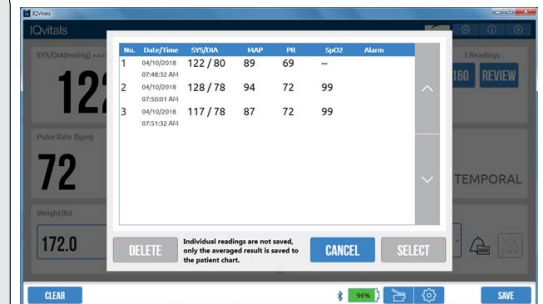


Product Bulletin IQvitals® Zone™ Product Update


Purpose: Following provides an explanation of selected software update.

A. Linear Deflation method update:

1. When the Linear Deflation method is unable to obtain a valid blood pressure reading, a prompt is displayed and user action is required.
2. Select “Confirm” to continue to the Step Deflation method (Note: The device will automatically returns to Linear Deflation method when the user select to “Save” or “Clear” data).
3. Select “Cancel” to continue using Linear Deflation method.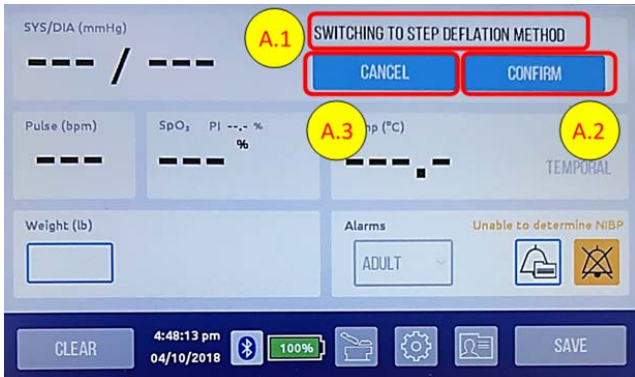


On IQvitals Zone Screen

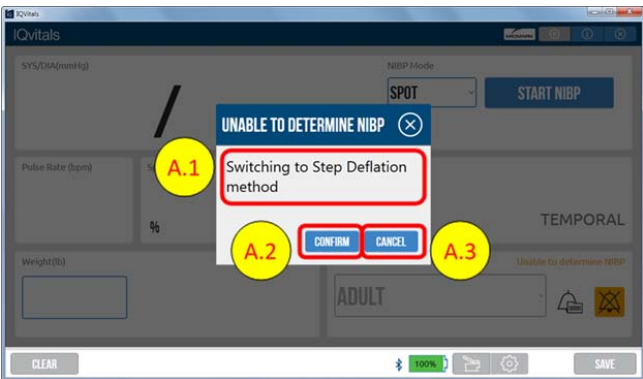
On PC Screen

B. Using Step Deflation method in Continuous, Interval and Averaging NIBP modes:

Note: With Main Board firmware version 0.10.9, only Step Deflation method is available for Continuous, Interval and Averaging NIBP modes. Following depicts the main screen with default settings in these modes.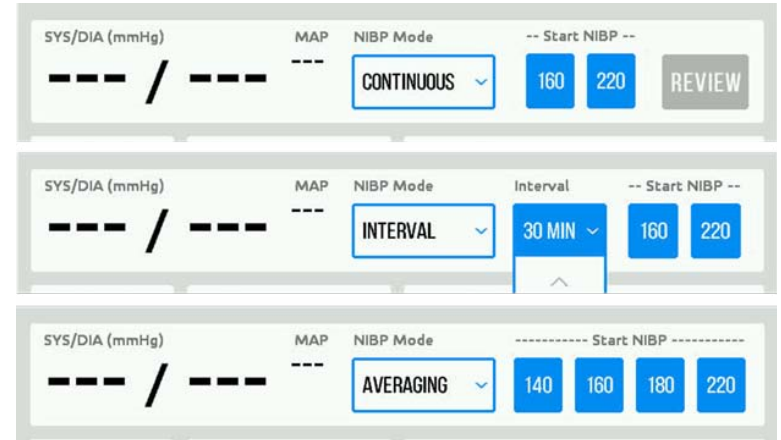

1. When ready to start a blood pressure measurement using step deflation mode, press the appropriate inflation pressure button.
2. Once the blood pressure measurement is started, the selected inflation pressure button changes to a red Stop button.

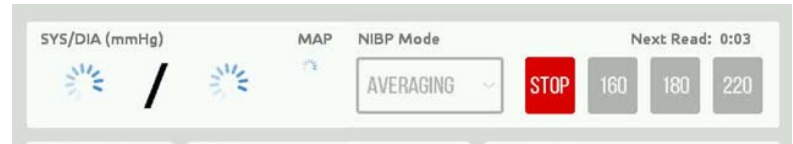3. To stop a blood pressure measurement at any time, press “Stop”.

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