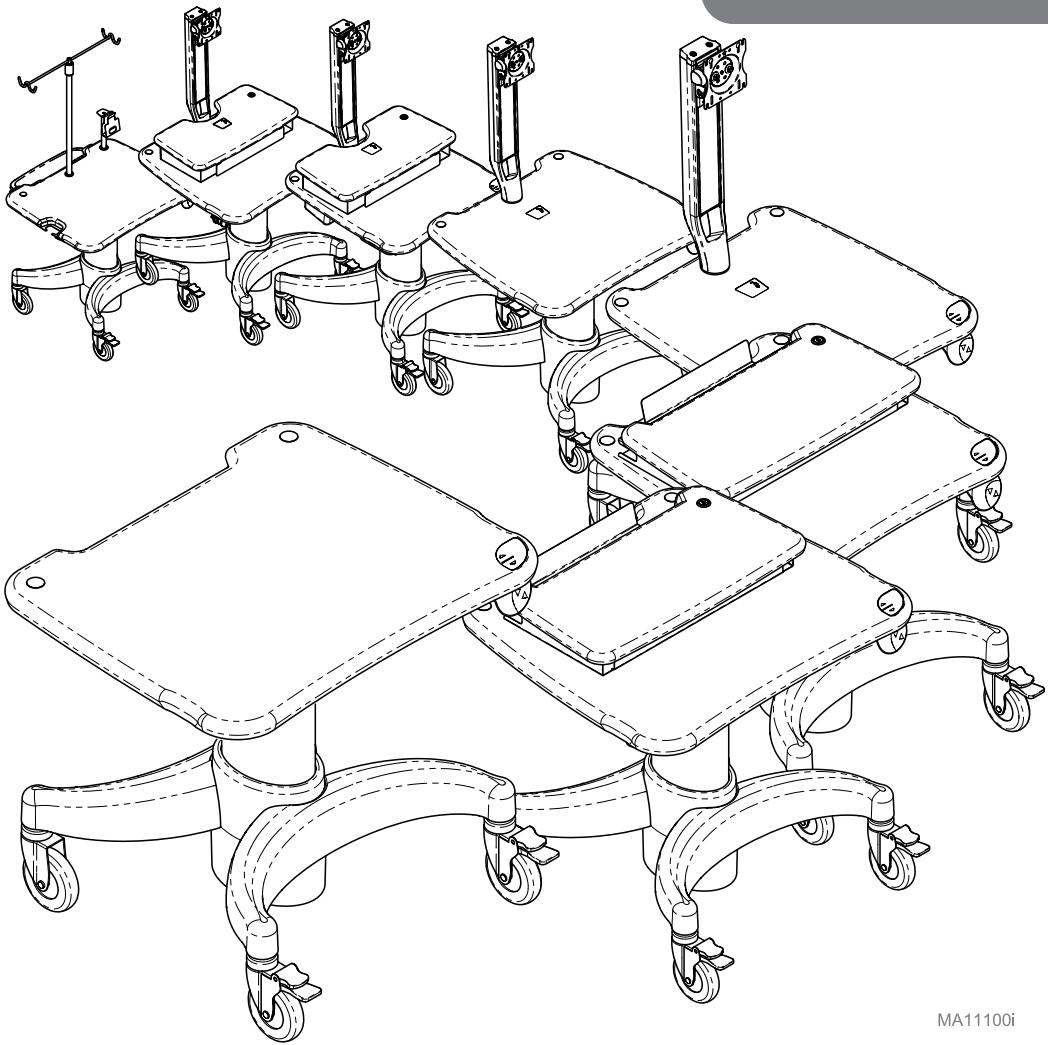
Mobile Workstations

For Models:
6211, 6212, 6213, 6214, 6215,
6216, 6217, 6218 & 6219

MA11100i

User Guide

003-10179-99 Rev. AA4 (8/17/2021)
TP202 20-42-FO-00014 Rev A1 C2169

Style P

Product Information

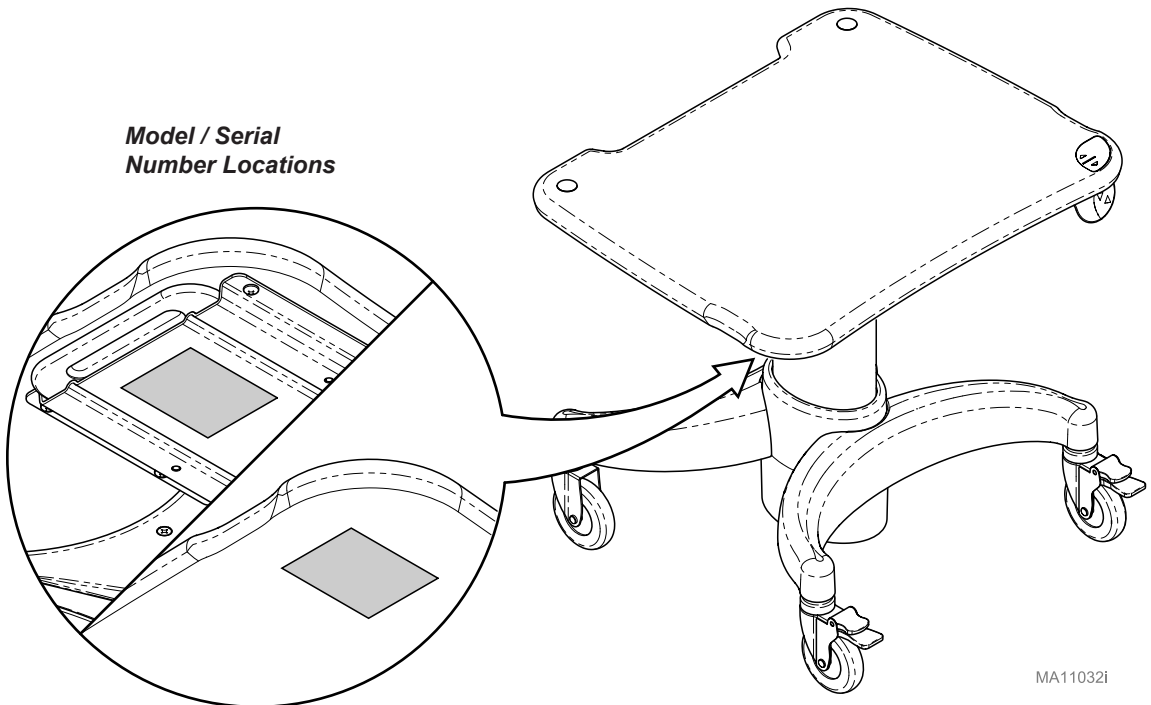
Dealer:

Date of Purchase:

Model / Serial Number:

Midmark Authorized Service Company:

**Model / Serial
Number Locations**

MA11032I

Table of Contents

<u>Product Information</u>	2
Important Information	
<u>Product Registration</u>	4
<u>Transportation / Storage Conditions</u>	4
<u>Intended Use</u>	4
<u>Disposal of Equipment</u>	4
<u>Authorized Representatives</u>	5
<u>Safety Symbols</u>	6
Operation	
<u>Cutting Zip Tie</u>	7
<u>Raise and Lower Workstation Top</u>	8
<u>Caster Break Pedal</u>	9
<u>Adjusting Monitor Tension</u>	10
<u>Adjusting Monitor Float Tower Tension</u>	10
<u>Removing Countertop</u>	11
<u>Adjusting Extension Front Form Factor</u>	11
<u>Moving Mouse Tray</u>	12
<u>Adjusting Mobile Printer Shelf</u>	12
<u>Battery Indicator Display</u>	13
<u>Changing Height of CPU Holder or Column Basket</u>	13
<u>Placing CPU in Holder</u>	14
<u>Removing Extender Plate</u>	14
<u>VOIP Bracket Adjustment and Changing Location</u>	15
Maintenance	
<u>Calling for Service</u>	16
<u>Cleaning</u>	16
<u>Preventative Maintenance</u>	16
Specifications	
<u>Workstations</u>	17
<u>Accessories</u>	18
Warranty	
<u>Midmark Workstation Limited Warranty</u>	19

Important Information

Product Registration

To register your product, go to www.midmark.com

Transportation / Storage Conditions

Ambient Temperature Range:..... 50°F to 120°F (+10°C to +48.9°C)

Relative Humidity..... 10% to 85% (non-condensing)

Intended Use

Provide a mobile and height adjustable workstation that provides support for multiple technology form factors. The work surface is intended to provide ergonomic positioning of supported equipment for users ranging from a seated 5th percentile female to a standing 95th percentile male.

Disposal of Equipment

At the end of product life, the workstation, accessories, and other consumable goods may become contaminated from normal use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

Note:

The battery for the Midmark Mobile Workstation is a lithium ion battery and must be recycled or disposed of properly according to national or local regulations.

Warning

Do not move workstation if there are obstructions in front of or under the caster. Failure to remove obstructions may cause tipping hazard.

Warning

Use of this equipment with damaged or excessive wear on casters may result in a tipping hazard.

Warning

Do not move workstation while extension cord or other cords are plugged in or not secured into proper storage bins or management systems.

Warning

Cylinder force and option selections are specific to their configuration. Please contact Midmark Technical Service before making any modifications to the cart.

Authorized Representatives

Customers in the US and Canada should direct all questions, incidents, complaints, and service requests to Midmark's Corporate directly.

1-800-MIDMARK (1-800-643-6275)
8:00 am until 5:00 pm Monday - Friday (EST)
www.midmark.com

Customers in the EU should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Promotal Srl.
Via Lenin 79/A
Quattro Castella (RE), 42020, Italy
Phone: +39.0522.875.166
Fax: +39.0522.243.096

Customers in India should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Midmark India
Kalpataru Point, Unit No. 12, 1st Floor Kamani road, Sion (E)
Mumbai, 400022
Tel:+91 22 4915 3000
Fax:+91 22 4915 3100

Customers in Saudi Arabia should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

Attieh Medico Ltd
Abdullah M. Al Khalifi Street
Al Nakheel Dist. II
P.O. Box 116105
Jeddah (East)
21391
Saudi Arabia
Tel: + 966 2 286 4707
Fax: + 966 2 286 4744

Customers in the United Arab Emirates should direct all questions, incidents, and complaints to Midmark's Authorized Representative listed below.

GULF DRUG LLC.
Al Barsha,
Dubai, UAE
Phone: +971 4 501 4000
Fax: +971 4 501 4100

Important Information - Safety Symbols

WARNING

Indicates a potentially hazardous situation which could result in serious injury.

Caution

Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices

Equipment Alert

Indicates a potentially hazardous situation which could result in equipment damage.

Note

Amplifies a procedure, practice, or condition.

Symbol Glossary

Pressure Limit

Temperature Limit

Proper shipping orientation

Keep dry

Fragile

Refer to manual

Manufacturer

Catalogue Number

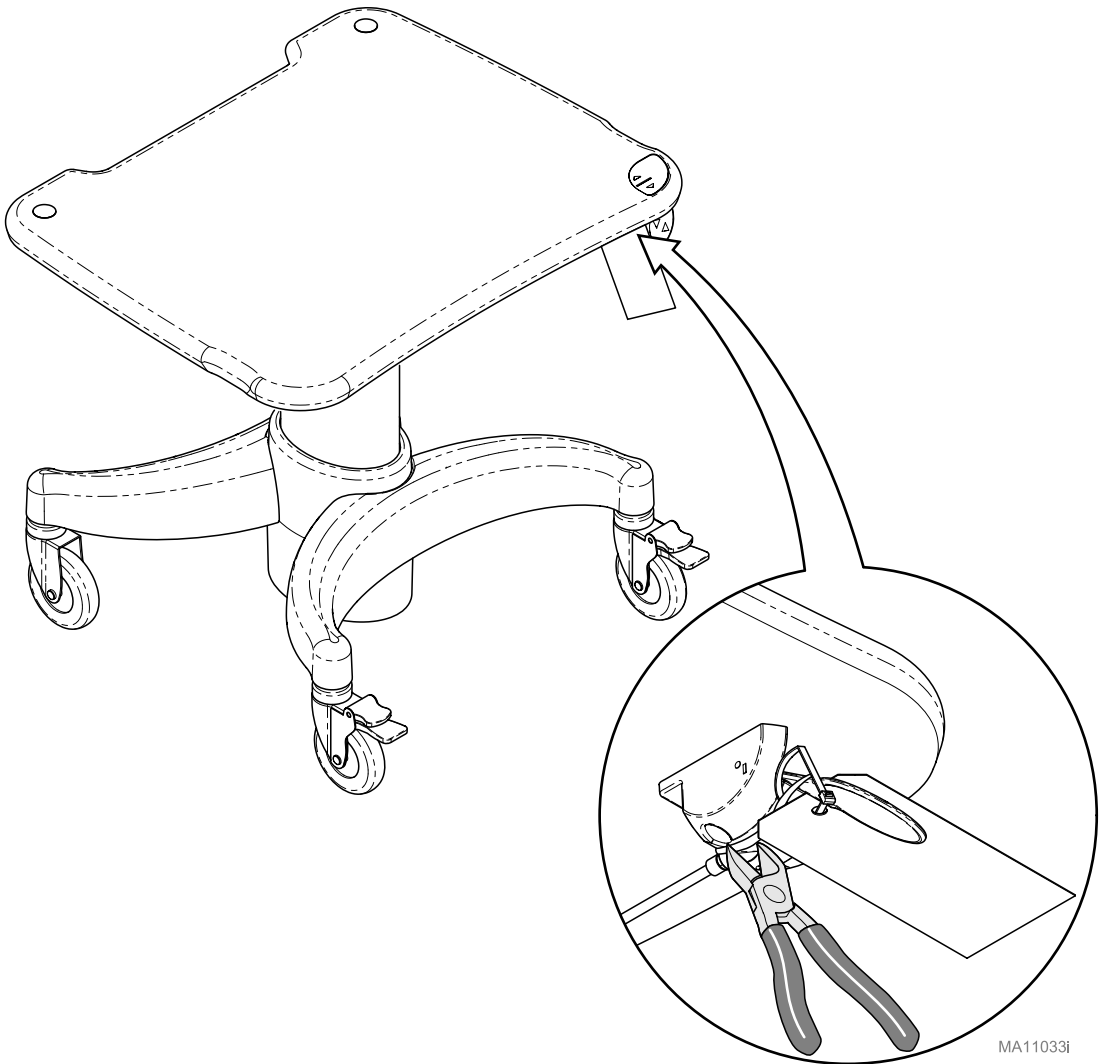
Serial Number

Cutting Zip Tie

WARNING

Pressurized gas cylinder assembled in this workstation. Assemble and attach all work surface mounted options prior to cutting cable tie around brake handle and raising the workstation. To avoid injury, keep your head and body parts away from the work surface.

MA11033i

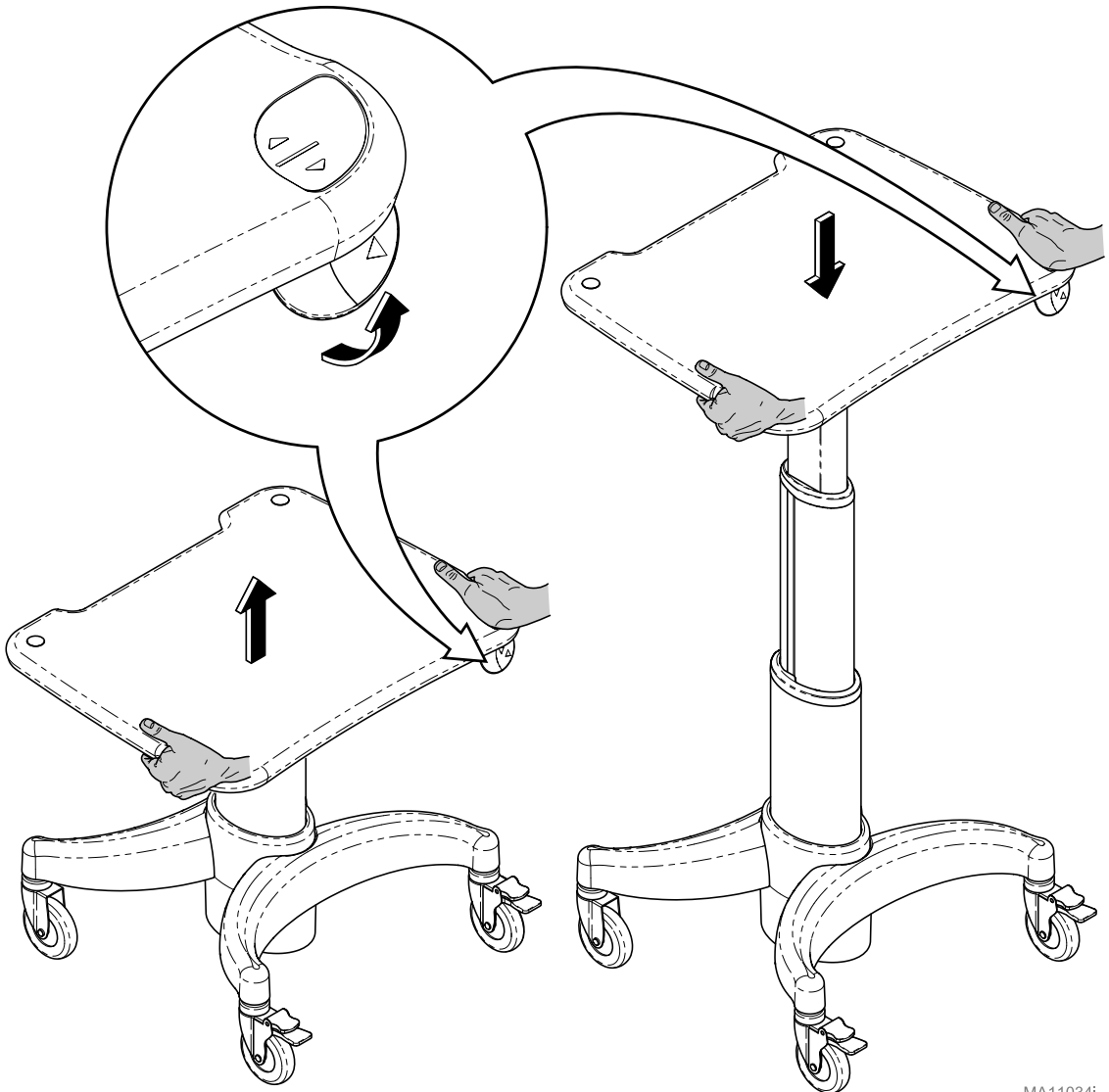
Raise and Lower Workstation Top

Hold workstation top with both hands while pressing handle to move workstation top up or down.

WARNING


Pressurized gas cylinder assembled in this workstation. Assemble and attach all work surface mounted options prior to cutting cable tie around brake handle and raising the workstation. To avoid injury, keep your head and body parts away from the work surface.

MA11034i

Caster Break Pedal

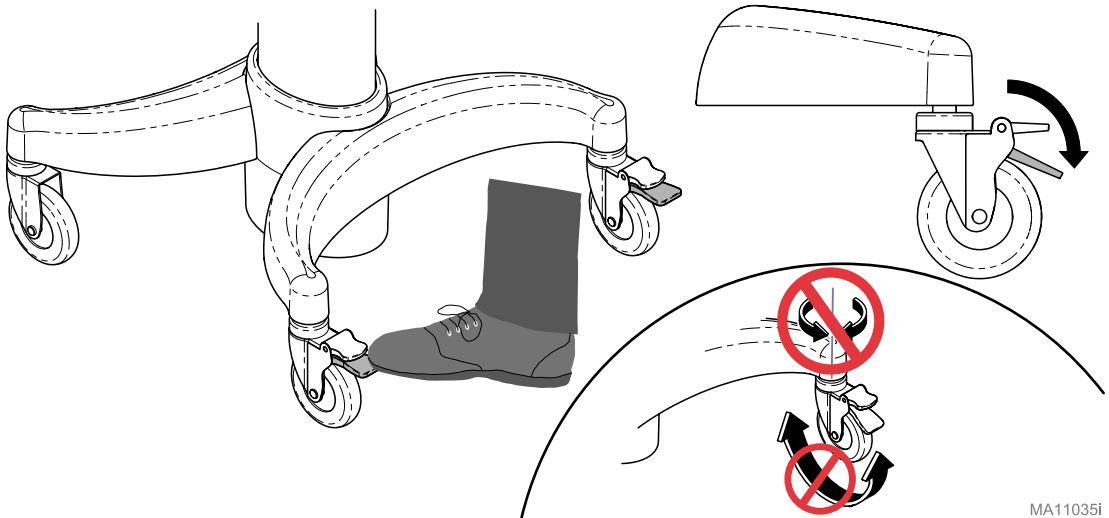
Pressing down on break pedal prevents both wheel and caster from rotating.

MA11035i

Caster Release Pedal

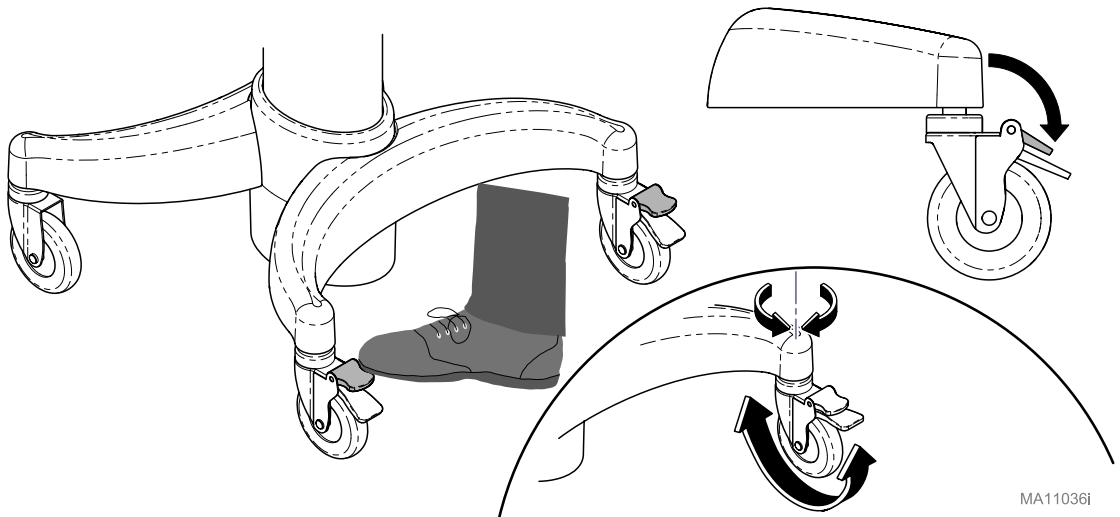
Pressing down on release pedal allows both wheel and caster to rotate.

MA11036i


Adjusting Monitor Tension (6212, 6213, 6218, & 6219)

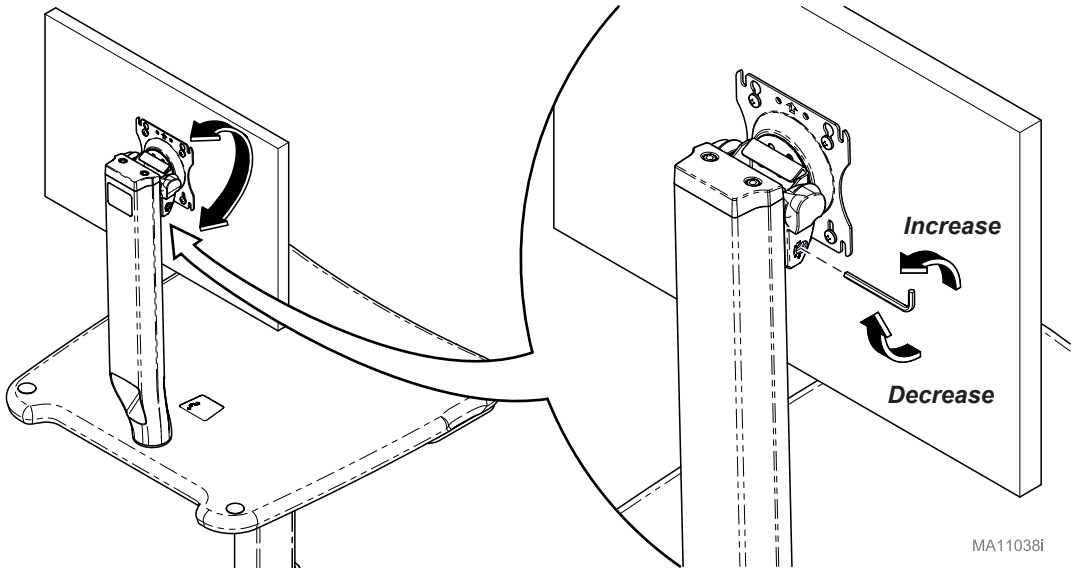Increase or decrease tension on monitor pivot with Allen Wrench on monitor mount.


Adjusting Monitor Float Tower Tension

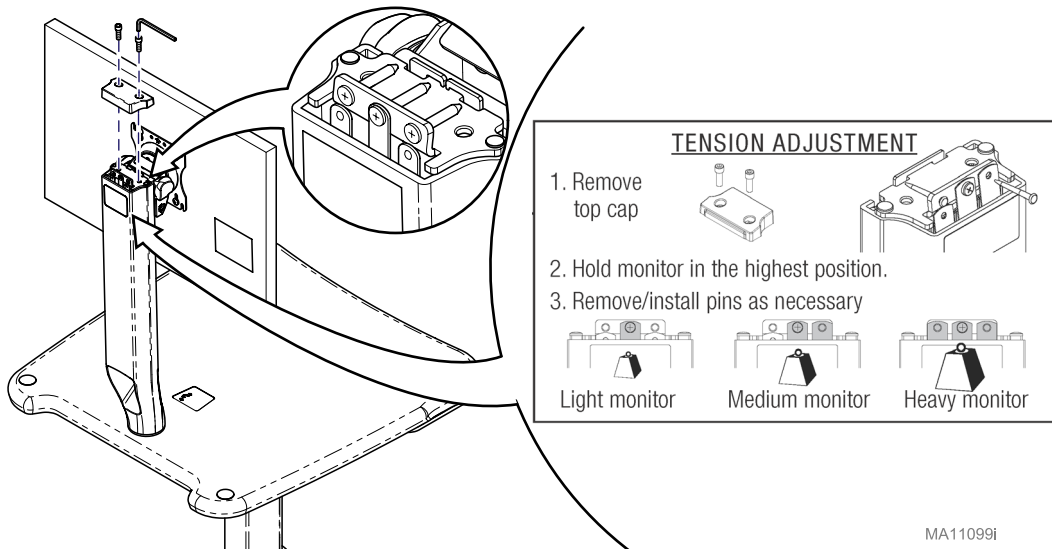- A. Use the 3/16" Allen wrench to remove screws (2) from plastic top cap of float tower.
- B. Remove and install pins as necessary per the label on tower. Keep unused pins.
- C. Replace plastic cap and secure with screws (2).

Removing Countertop (6211, 6213, 6217 & 6219)

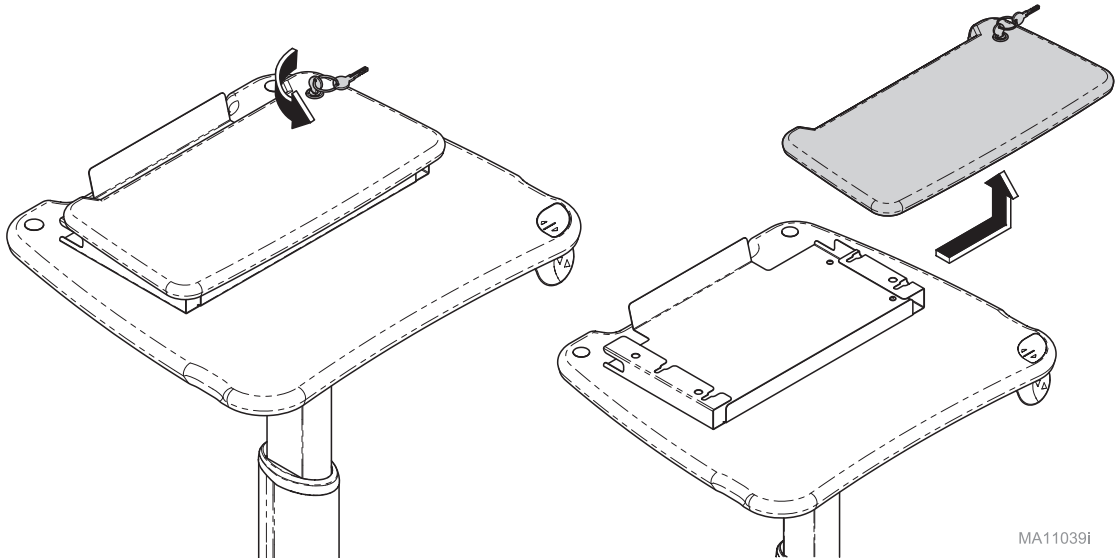
Turn key to unlock countertop, slide to right and lift to remove from worksurface.

MA11039i

Adjusting Extension Front Form Factor (6213 & 6219)

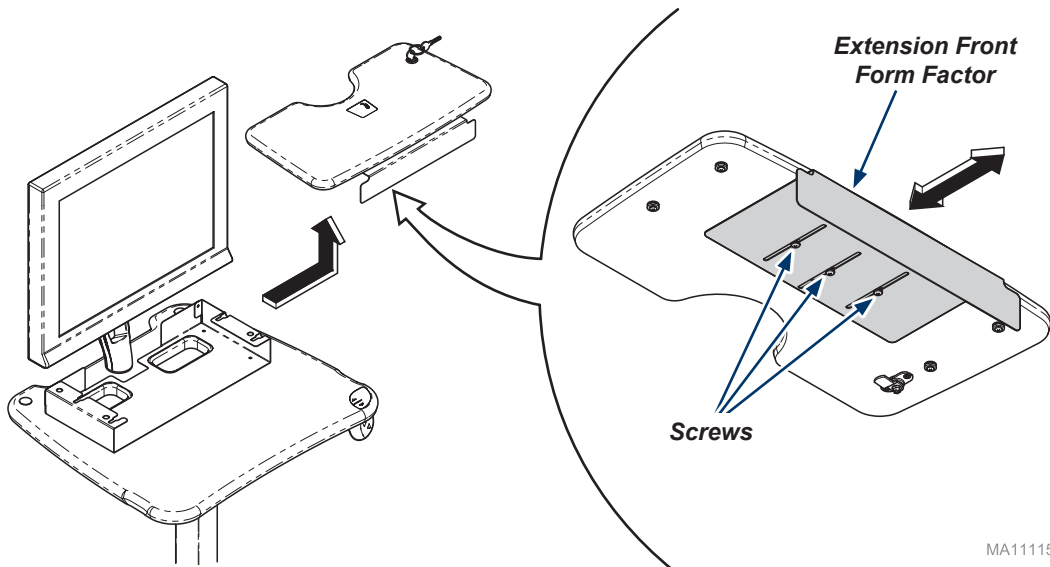
Remove countertop from worksurface. Loosen screws (3) and adjust extension front form factor to desired position and tighten screws (3)

MA11115i

Moving Mouse Tray (6217, 6218, & 6219)

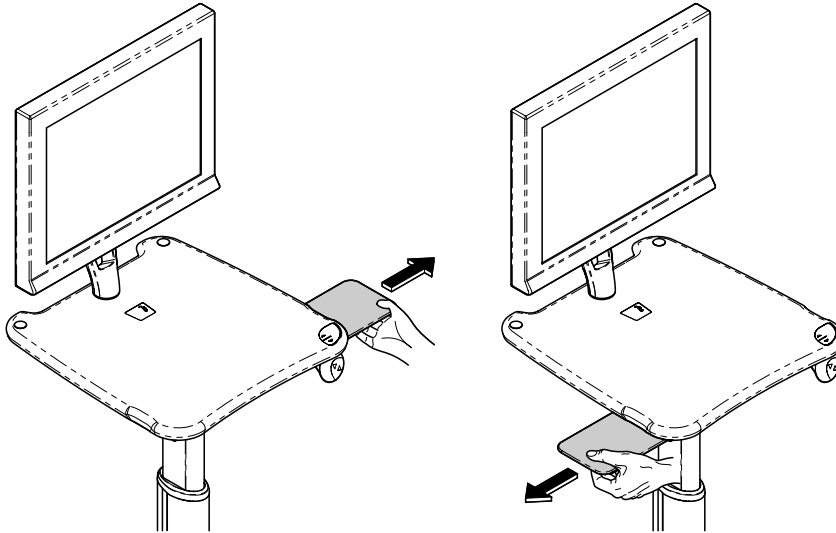
Pull mouse tray out from under countertop either side as shown.

MA11037I

Adjusting Mobile Printer Shelf

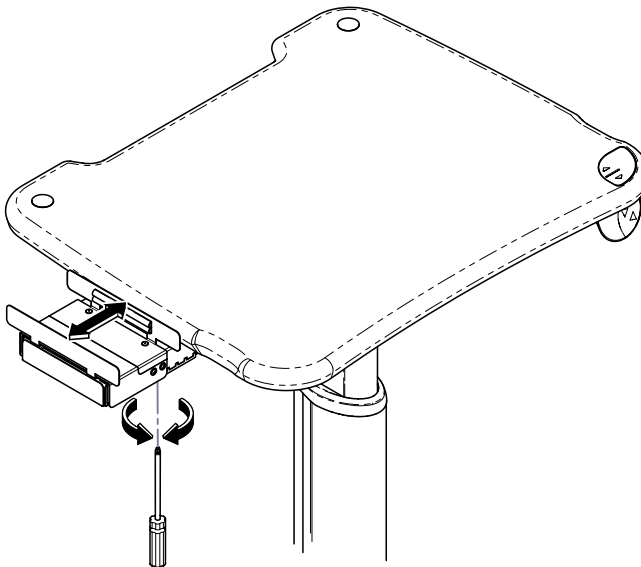Loosen screws (2) on bottom of mobile printer shelf so teeth as loosened and slide printer tray out or in aligning teeth and re-tighten screws (2).


MA11076I

Battery Indicator Display

A. Power button allows user to turn on and off.

B. Battery life indicator allows user to know how much battery life is left for use.

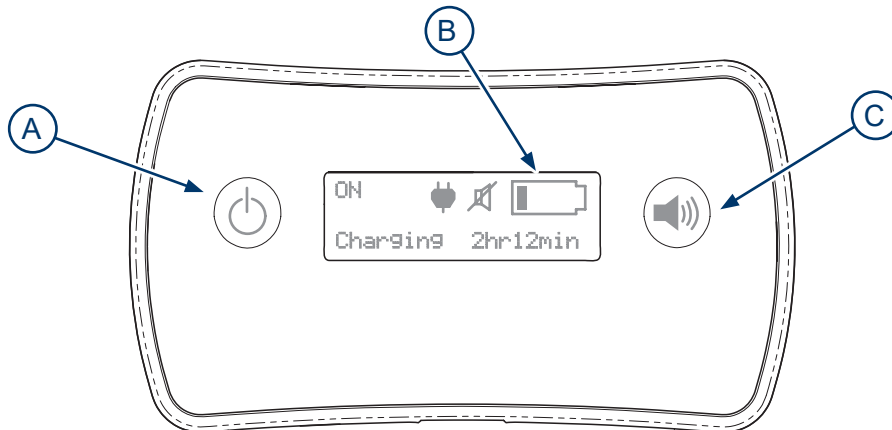
C. Sound button allows user to turn lower battery indicator sound off and on.

MA11054i

Changing Height of CPU Holder or Column Basket

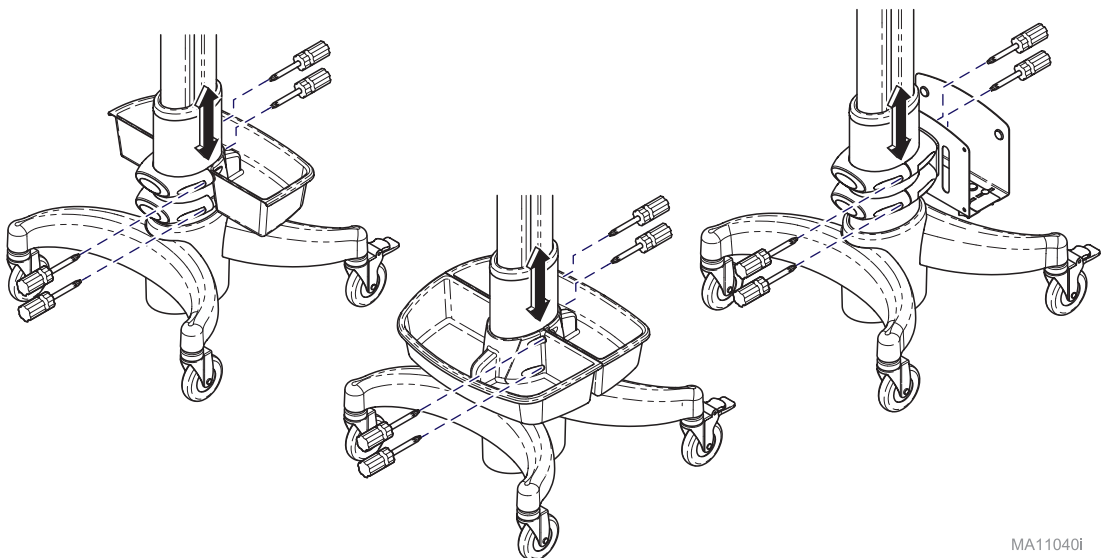
Loosen screws (4) on both sides of clamps or basket to adjust up or down then retighten screws.

MA11040i

Placing CPU in Holder

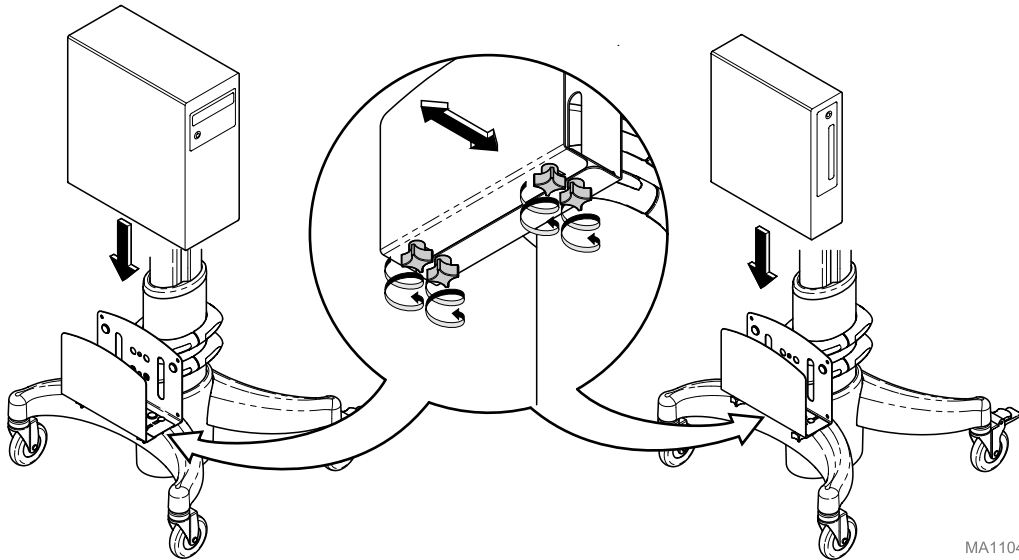
Loosen knobs, adjust holder as needed, install CPU and retighten knobs.

MA11041i

Removing Extender Plate

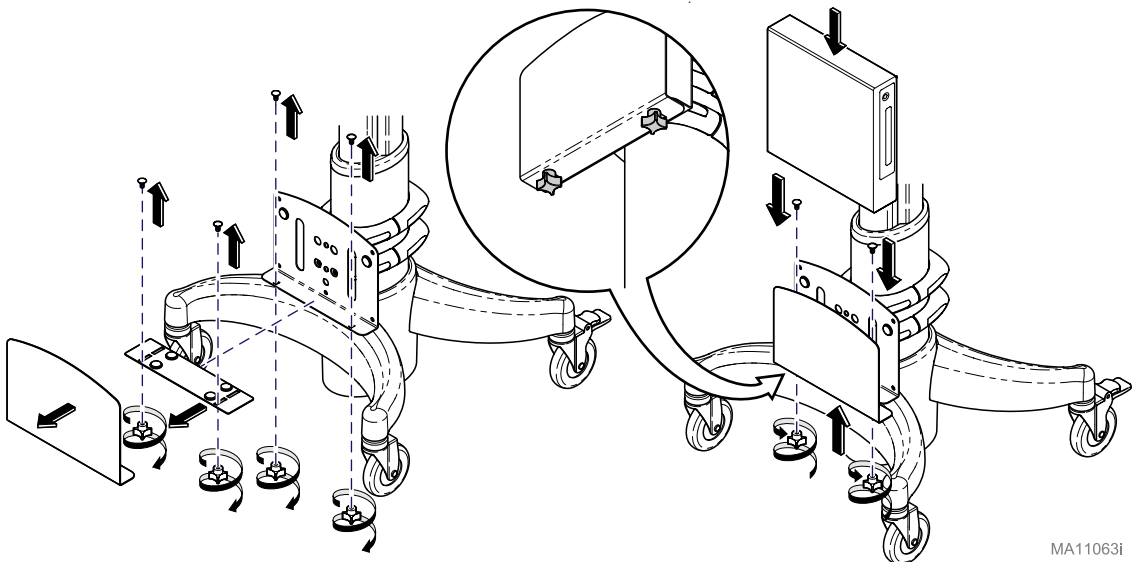
Remove knobs (4), bolts (4) and outer shield of CPU Holder. Install outer shield onto inner shield loosely with bolts (2) and knobs (2), adjust knobs (2) for CPU width and tighten knobs (2)

MA11063i


Adjusting Mounting Plate

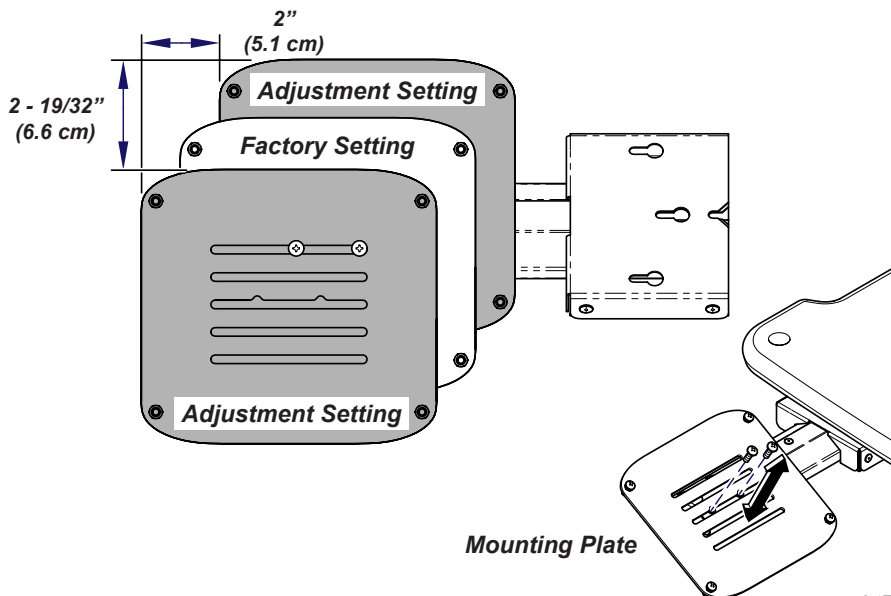Remove screws (2) from mounting plate adjust into position and resuce with screws (2).

Changing VOIP Bracket from one side to other (LH to RH shown)

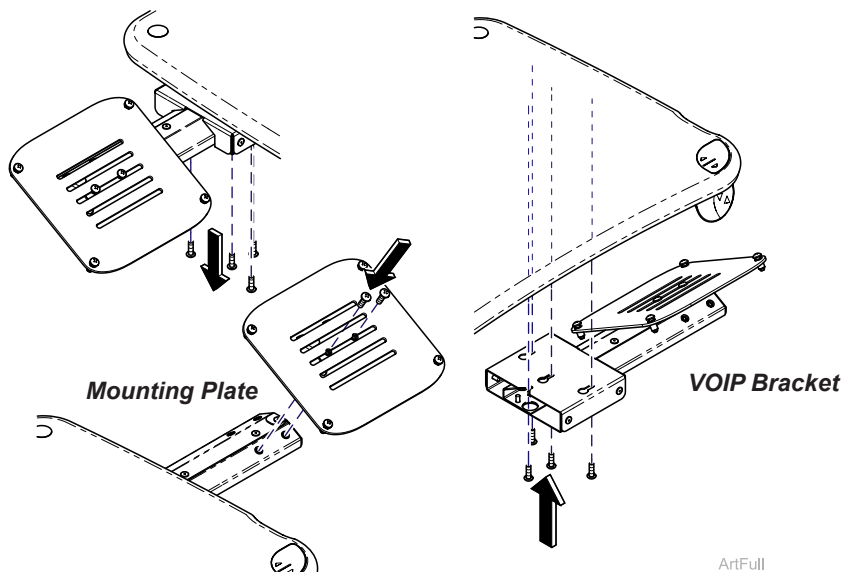Remove screws (4), from under VOIP Bracket from workstation. Install to other side of workstation with screws (4). Remove mounting plate from VOIP Bracket screws (2), reinstall mounting plate with screws (2) to face front of workstation as shown.

Maintenance

Calling for Service

Note

Model / serial number information is required when calling for service.

If service is required, contact your authorized Midmark dealer. (See Authorized Representatives)

Cleaning

Spot cleaning	Wash workstation weekly with a mild soap and water rinse with clear water and dry completely to remove disinfection cleaner build-up.
Difficult stains	Wash workstation with a mild solvent.
Disinfecting	Disinfect workstation with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities.

Detailed care and maintenance instructions are included with your product. This information is also available on www.midmark.com in the Technical library under the User Information tab for your product.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner.

Preventative Maintenance

Periodically inspect the following areas:

- Power cord(s) should be free of cuts or other visible damage.
- All fasteners should be in place and tightened securely.
- All mechanical functions should operate properly.

Specifications - Workstation

Workstation Specifications Chart			
Models:	Worksurface Dimensions Width x Depth	Worksurface Shelf Dimensions (W x D)	Range of Motion Height from Worksurface Top
6211 Secure Laptop Workstation:	26" x 23"	20.5" x 11"	Min 23" ± 1" Max 49" ± 1"
6212 Flat Panel PC Workstation:	26" x 23"	N/A	Min 23" ± 1" Max 49" ± 1"
6213 Flat Panel Secure PC Workstation:	26" x 23"	20.5" x 11"	Min 23" ± 1" Max 49" ± 1"
6214 Procedure Workstation:	26" x 21"	N/A	Min 23" ± 1" Max 49" ± 1"
6215 [-001] Compact (Rectangle Top) Workstation:	26" x 21"	N/A	Min 23" ± 1" Max 49" ± 1"
6215 [-002] Compact (Rectangle Top) Workstation:	23.5" x 17.5"	N/A	Min 23" ± 1" Max 49" ± 1"
6215 [-003] Compact (Kidney Top) Workstation:	25" x 16.8"	N/A	Min 23" ± 1" Max 49" ± 1"
6216 IQvitals Zone Workstation:	N/A	N/A	Min 23" ± 1" Max 49" ± 1"
6217 Secure Laptop Workstation w/Mouse Tray:	22" x 23"	20.5" x 11"	Min 23" ± 1" Max 49" ± 1"
6218 Flat Panel PC Workstation w/Mouse Tray:	22" x 23"	N/A	Min 23" ± 1" Max 49" ± 1"
6219 Flat Panel Secure PC Workstation w/Mouse Tray:	22" x 23"	20.5" x 11"	Min 23" ± 1" Max 49" ± 1"

Note:

6216 Workstation maximum working load, including, IQvitals Zone device and accessories, is 34 pounds.

Specifications - Accessories

Accessory Specifications Chart		
	Dimensions (H x L x W)	Weight capacity
Drawer Assembly (9A646001 / 9A647001):	5.25" x 10" x 14"	3 lbs
Mobile Printer Shelf (9A648001):	3" x 8.75" x 8"	5 lbs
VOIP Shelf (9A649001 / 9A649002):	1.5" x 8.75" x 15"	5 lbs
Rear Basket (9A650001):	4" x 6.5" x 16.5"	2 lbs
Equipment Pole (9A651001):	22.5" x 4" x 19"	3 lbs
Single Column Basket (9A652001):	4" x 12" x 17.5"	2 lbs
Double Column Basket (9A653001):	4" x 15.75" x 17.5"	4 lbs
Scale Holder (9A654001):	12.75" x 12.25" x 14.5"	8 lbs
Power Mount (9A655001 / 9A655002):	11.5" x 17.25" x 16.5"	N/A
Utility Shelf (9A656001):	8.25" x 12.5" x 17.75"	30 lbs
Blood Pressure Cuff Holder (9A658001):	8.75" x 3" x 3"	2 lbs
Bar Code Holder (9A659001):	3" x 4.25" x 3"	2 lbs
CPU Holder (9A663001)	7.25" x 12.5" x 10"	20 lbs
VOIP Bracket (9A695001 / 9A695002):	3.38" x 13" x 6.06"	6 lbs

Midmark Workstation Limited Warranty

Extension of Warranty

Midmark™ Corporation (“Midmark”) warrants to the original purchaser its Workstation, manufactured by Midmark, to be free from defects in material and workmanship under normal interior use and service. Midmark’s obligation under this warranty is limited to the repair or replacement, at Midmark’s option, of the parts or the products the defects of which are reported to Midmark within the applicable warranty period and which upon examination by Midmark prove to be defective.

Applicable Warranty Period

The applicable warranty period, measured from the date of delivery to the original user, shall be as follows:

- (a) Five (5) years for all products and components including worksurfaces and drawer fronts, casters and slides, except for the item in (b).
- (b) One (1) years for electrical components such as, cords, controls and accessories..

Exclusions

Midmark’s warranty does not cover sinks, faucets, and plumbing accessories. Any warranties on these items are extended directly by the manufacturer of these items to the original purchaser. Information on these manufacturers’ warranties will be enclosed with the applicable products. Also, Midmark will furnish copies of any of the warranties extended by any such manufacturers upon request.

This warranty does not cover and Midmark shall not be liable for: (1) repairs and replacements required because of misuse, abuse, negligence, alteration, accident, freight damage, or tampering; (2) matching of color, grain, or texture except to commercially acceptable standards; (3) changes in color caused by natural or artificial light; (4) products which are not installed, used, and properly cleaned as required in Midmark’s written Installation Manual; (5) specially manufactured products; (6) products considered to be of consumable nature such as light lamps and surge suppression product; (7) accessories or parts not manufactured by Midmark; (8) charges by anyone (including Midmark’s authorized Midmark Dental Casework dealers) for adjustments, repairs, replacement parts, installation, or other work performed upon or in connection with such products which is not expressly authorized in writing in advance by Midmark.

Exclusive Remedy

Midmark’s only obligation under this warranty is the repair or replacement of defective parts. Midmark shall not be liable for any direct, special, indirect, incidental, exemplary, or consequential damages or delay.

No Authorization

No person or firm is authorized to create for Midmark any other obligation or liability in connection with the products.

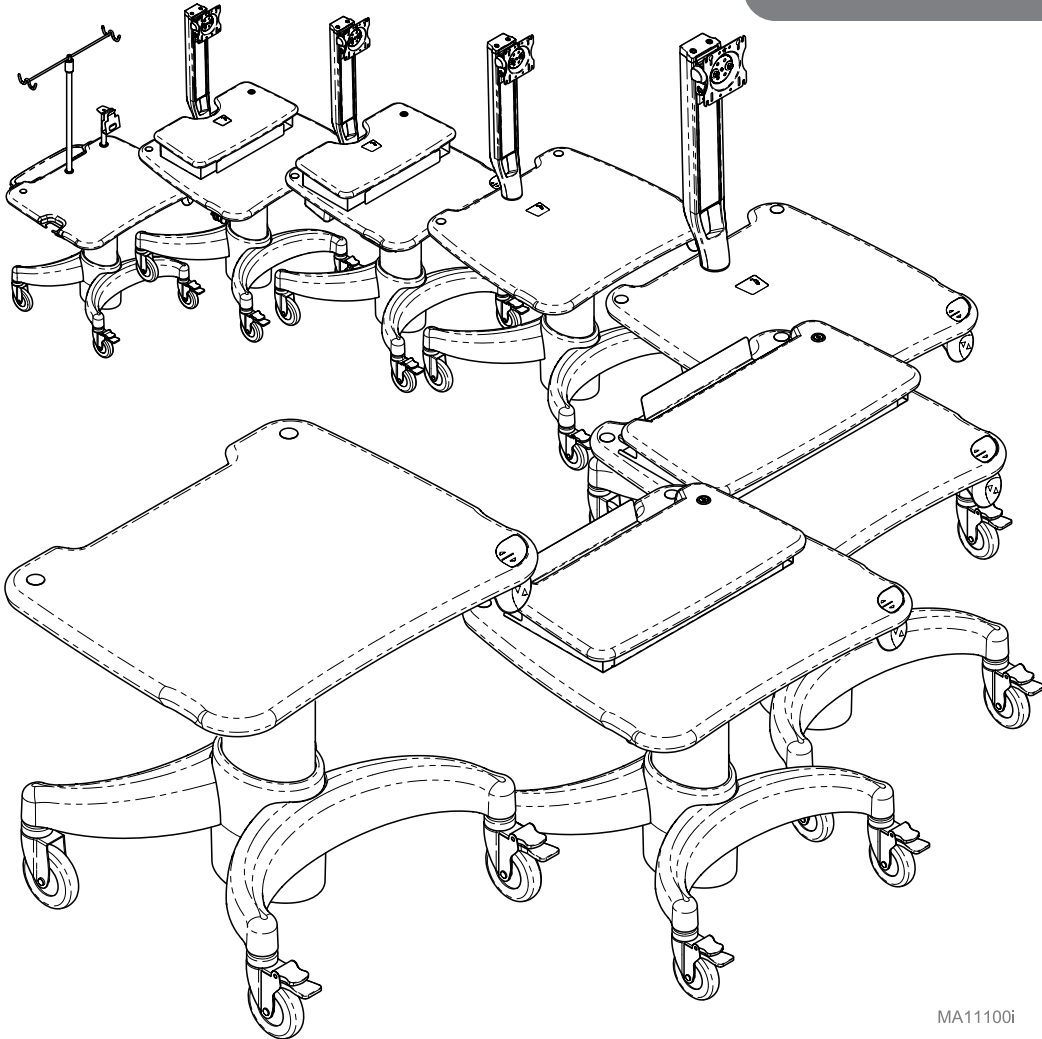
THIS WARRANTY IS MIDMARK’S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. MIDMARK MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY WARRANTIES OF MERCHANT-ABILITY OR FITNESS FOR ANY PARTICULAR PURPOSE. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS.

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Midmark Corporation