

M9/M11 Fill / Vent Valve Repair

Vent	002-10342-00	115vac
	002-10342-01	230vac
Fill	002-10330-00	115vac
	002-10330-01	230vac

Components:
 Valve 1
Special Tools:
 None



WARNING


To prevent risk of shock always disconnect power before removing covers or performing any service procedure.

Step 1: Drain water from reservoir.


Note: For fill valve replacement only.

Step 2: Disconnect compression fitting from top of vent valve.

Note: For vent valve replacement only.

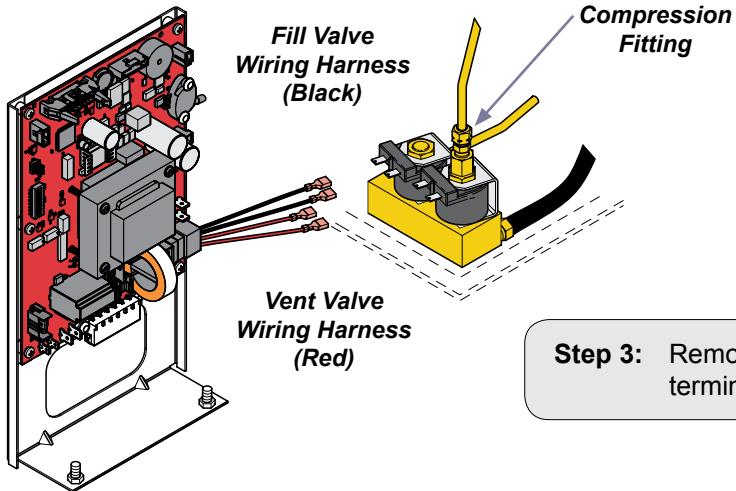

Step 3: Remove wires from terminals of faulty valve.

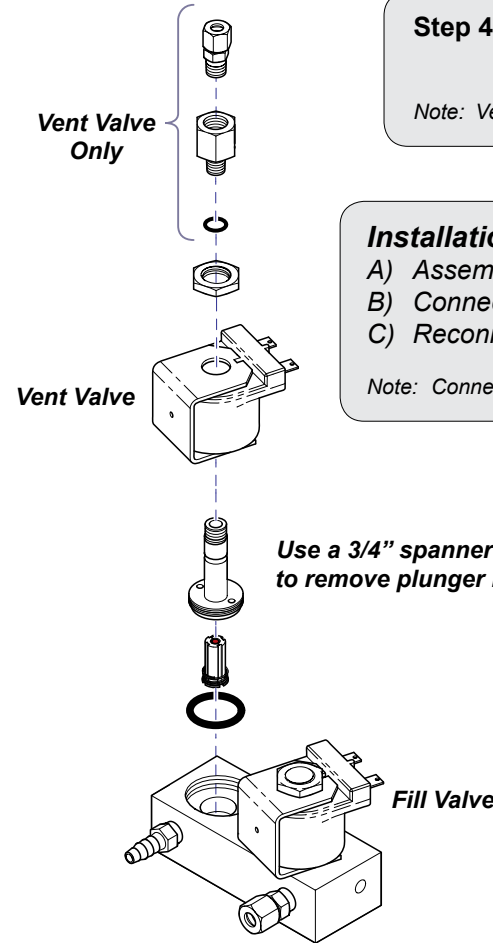

Step 4: Disassemble valve and remove any debris.

Note: Vent valve shown.

Installation...

- A) Assemble valve.
- B) Connect compression fitting. (Vent Valve)
- C) Reconnect Fill / Vent valve wiring harness.

Note: Connect Red to Vent Valve - Black to Fill Valve.

Test...

- A) Refill reservoir.
- B) Run test cycle.
- C) Check for leaks.

SA1984i