

Pressure Relief Valve [002-10372-00]

Components:
 Pressure relief valve w/ washer..... 1
 Enclosure (RH) for Pressure relief valve 1
 Enclosure (LH) for Pressure relief valve..... 1


Special Tools:
 none



WARNING
 To prevent risk of shock always disconnect power before removing covers or performing any service procedure.




Caution
 Allow to cool to ambient / room temperature prior to removing cover. Release pressure by pulling ring of pressure relief valve if necessary.

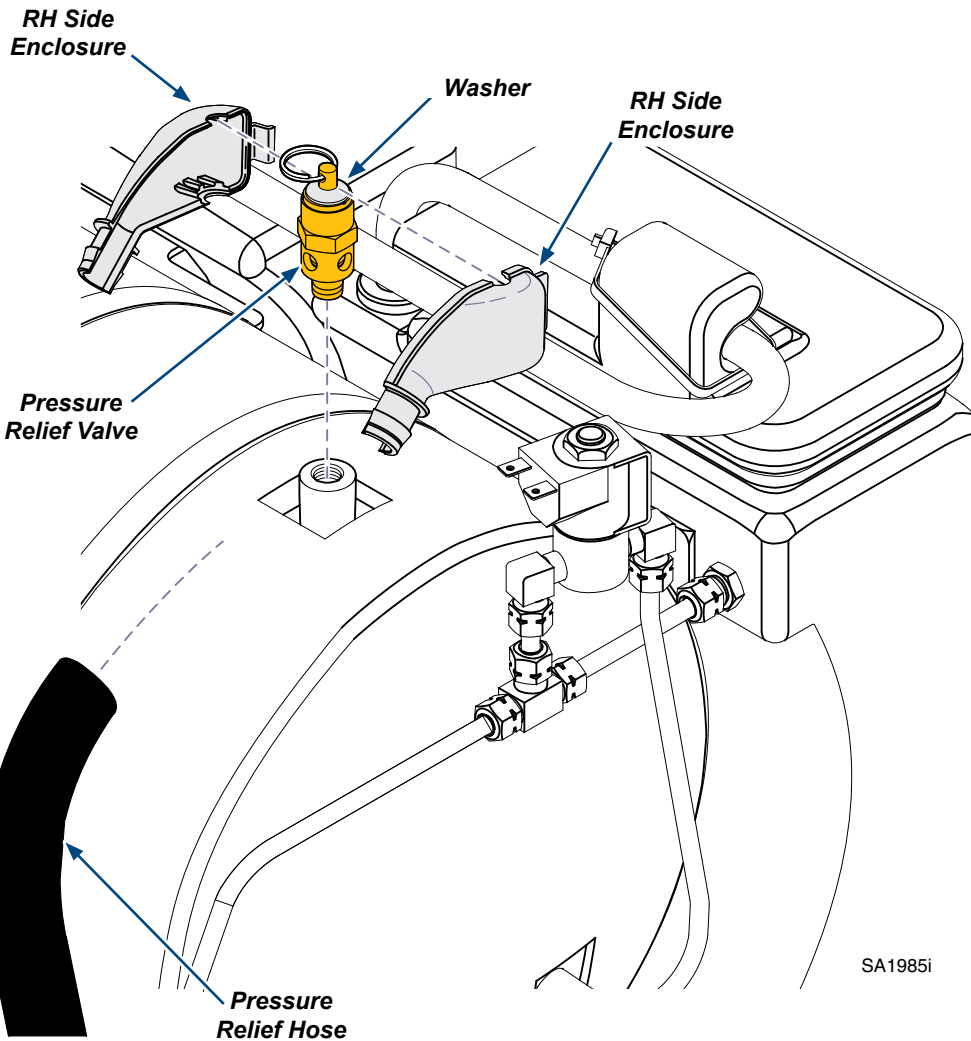

Step 1: Remove Pressure Relief Valve.

- A) Remove Pressure Relief Hose.
- B) Remove Pressure Relief Valve Enclosures.
- C) Remove Pressure Relief Valve.

Note
 Assure the washer is located between the top of the enclosure and the ring of the PRV.

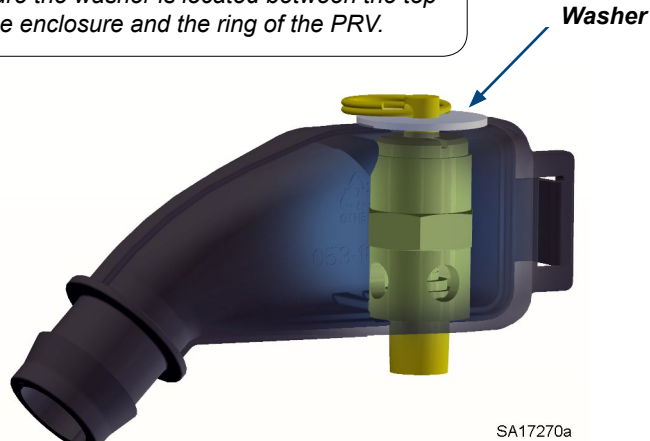

Note
 The Pressure Relief Valve has a seal on the threads to prevent leakage.

Step 2: Install Pressure Relief Valve.

- A) Install Pressure Relief Valve.
- B) Assemble Pressure Relief Valve Enclosures.
- C) Connect Pressure Relief Hose.

Note: If necessary, trim the Hose to prevent it from touching the counter top.