
Applies to Models:

- M9® (-020 /-021 /-022)
(-033 /-034)
(-040 /-041 /-042 /-043)
- M9D (-020 /-022 /-042)
- M11® (-020 /-021 /-022)
(-033 /-034)
(-040 /-041 /-042 /-043)
- M11D (-020 /-022)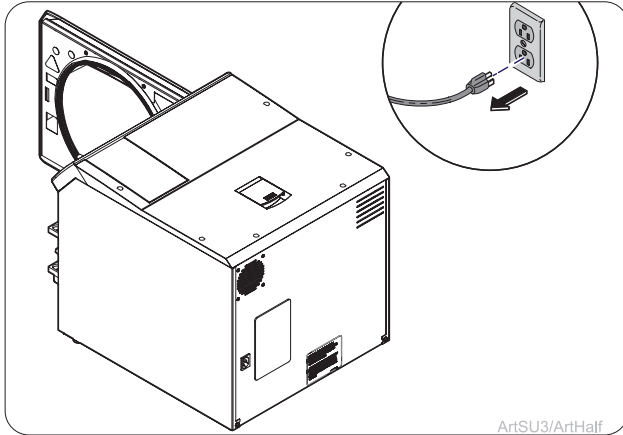

ArtSU3/ArtHalf


Step 1: Unplug Sterilizer.

Unplug power cord from wall outlet or power inlet on back of the sterilizer.

WARNING

To prevent risk of shock always disconnect power before removing covers or performing any service procedure.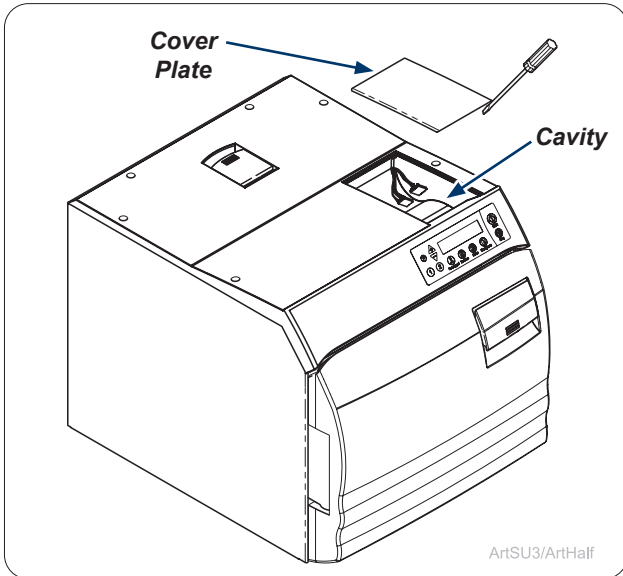

ArtSU3/ArtHalf


Step 2: Remove Cover Plate.

Remove Cover Plate from top right of the sterilizer. Remove any double-sided tape remaining on the sterilizer.

-02X and -03X Model Sterilizer


-04X Model Sterilizer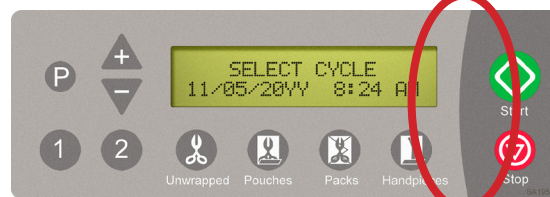

Step 3: Identify Sterilizer Model.

Use images to identify model of sterilizer where Data Logger will be installed before completing following steps. If you have an -04X model sterilizer, proceed to step 5.

-02X models only!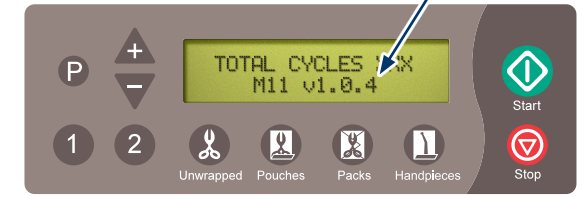

Note


The -02X, model sterilizers must have version 1.0.3 to 1.0.4 operating software for the Data Logger to be compatible. Please confirm your sterilizer has compatible software. If your sterilizer has different operating software, contact your Midmark representative for options.

Skip this step if you have a -03X or -04X model.

Step 4: Verify Software Version. (-02X models only)

To determine the software version of your -02X model sterilizer, cycle the power on the unit. The software version will be displayed during the start up cycle.

-02X/-03X models only!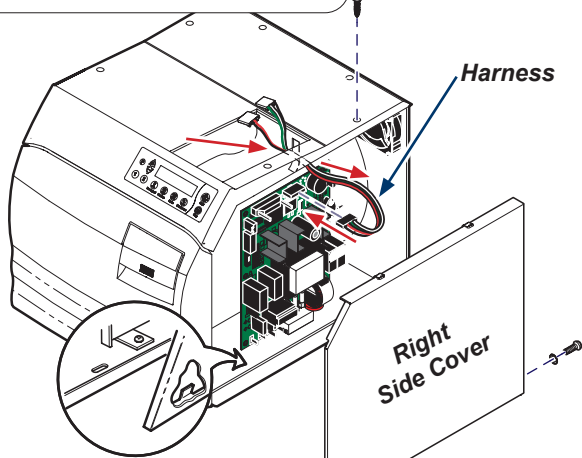


Skip this step if you have a -04X model.

ArtSU3/ArtHalf

Step 5: Install Harness. (-02X/-03X models only)

Remove right side cover. Remove existing harness if present. Install new harness by routing black connector of harness down through hole in top cover. Attach black connector to top of main PC board. Install right side cover.

Test Port
Unused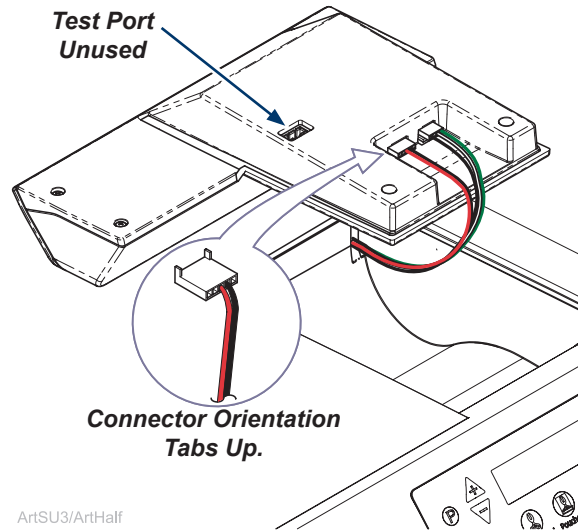


ArtSU3/ArtHalf

Step 6: Connect wires to Data Logger.

Locate wires in the sterilizer cavity and connect them to the Data Logger.

Remove
film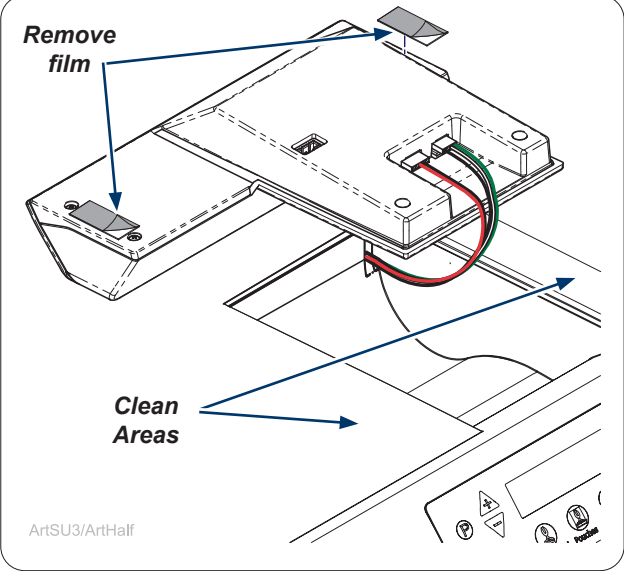


ArtSU3/ArtHalf

Step 7: Remove film from tape.

Remove film from double sided tape.

NOTE: Thoroughly clean top cover of sterilizer where Data Logger adheres.

Back Edge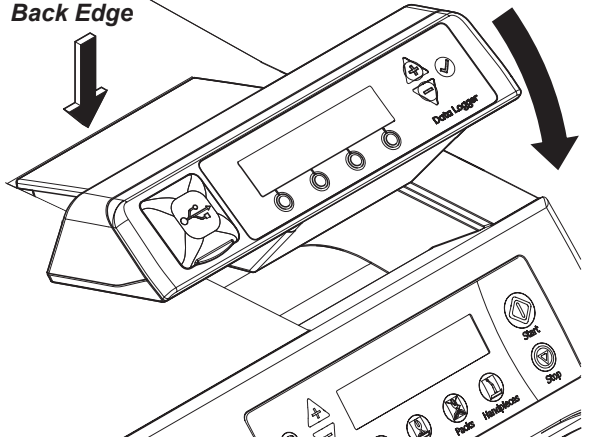

Step 8: Install Data Logger.

Line up back edge of Data Logger into back edge of sterilizer cavity. Make sure the wires do not interfere with placing the Data Logger and rotate front edge of Data Logger down into place. Press down firmly on the Data Logger to secure.

Front
View

Back
View

Serial Number

ArtSU3/ArtHalf

Step 9: Plug in Sterilizer and Record SN's.

Plug power cord into wall outlet and power inlet on sterilizer back. Record serial numbers of sterilizer and Data Logger.

midmark

Solutions • Contact Us • Online Store • Products • Support • About Us •

Data Logger Registration

TO REGISTER YOUR DATA LOGGER, PLEASE FILL OUT THE FORM BELOW.

First Name*

Last Name*

Email Address (for unlock code)*

Address Line 1

Note

Check to make sure information is entered correctly prior to hitting submit.


Registration URL:

<https://www.midmark.com/data-logger-registration>

Step 10: Obtain Unlock Code.

Access the Data Logger Registration web page at the on-line address above. Complete the Data Logger registration form on the web page to obtain an Unlock Code for the Data Logger that will be sent to you by e-mail. Once you have the Unlock Code, you can choose to continue the Data Logger set-up using either a manual or on-line method.

ONLINE METHOD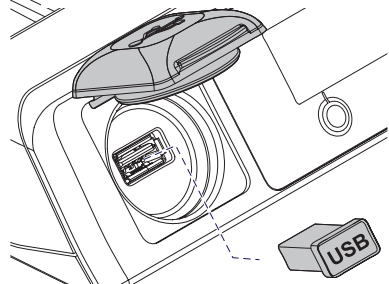

Configuration URL:
<https://www.midmark.com/data-logger-configuration>

Online Method Step 1: Setup.

Remove the USB device from the Data Logger. Transfer the USB device to an internet enabled computer. Navigate to <https://midmark.com/data-logger-configuration> using the computer. Complete the form on the page. Once the form is completed, a file named "config.ini" will be downloaded onto the computer. Locate the file in the computer's "Downloads" folder. Save this file to the Data Logger's USB device installed on the computer. Make sure the "config.ini" file has been successfully copied to the USB device. Do not rename the file. Consult with an appropriate IT professional for support with downloading the config.ini file if required.

Data Logger Touchpad


Choose Setup Method

ArtSU3/ArtHalf

Online Method Step 2: Data Logger Setup.


Once you have obtained the Unlock Code. Use the Data Logger Touchpad to press ENTER (✓). Then select the Online method of setting up the Data Logger.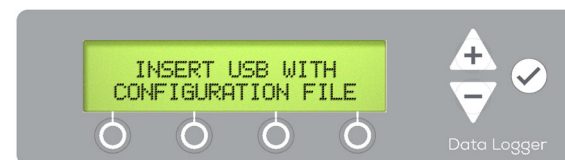

ArtSU3/ArtHalf

Online Method Step 3: Install Files.

Reinstall the USB device containing the downloaded "config.ini" file into Data Logger and close cover. Wait for the "config.ini" file to automatically install on the Data Logger.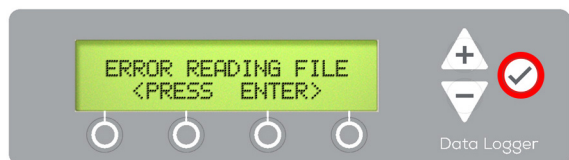

ArtSU3/ArtHalf

Online Method Step 4: Press Enter (✓).

Press ENTER (✓) to proceed with normal operation. If an error occurs press ENTER (✓) and restart the online method.

-02X/-03X models only!


Set Date / Time

Set Date / Time.

- Navigate to the Set Date / Time Menu using PLUS or MINUS buttons then press the ENTER button.
- Use PLUS / MINUS buttons to adjust the values for each menu then press NEXT.
- When finished setting, AM / PM press the DONE button to complete the Date and Time setup.

Note

Setting Date/Time is not required for any -4x model sterilizers. These models use Date/Time from sterilizer not the Data Logger for sterilization records.

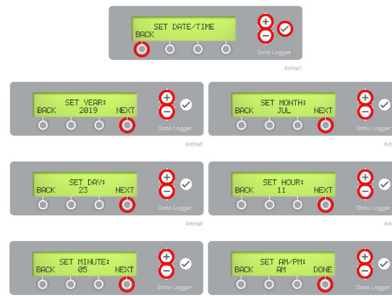003-10278-99

English - 20

© Midmark Corporation 2019

Online Method Step 5: Set Date/Time.

Sterilizer models -02X and -03X require navigating to the SET DATE/TIME menus and entering Date and Time parameters. Refer to user guide for details.

Navigating Setup Menus.

- **PLUS, MINUS** changes characters.
 - Press and hold to scroll through available characters when entering data.
- **NEXT** advances to next field.
- **DONE** advances to next menu when entering data is complete.
- **BACK** deletes the previous character/returns to the previous menu.

Blank Spaces

The blank space character is located between A and 9 when scrolling through characters on the screen.

Pressing ENTER (✓)

DO NOT press ENTER (✓) in any of the following steps until you have finished entering all information for that step. Refer to the User Guide for details on updating incomplete entries.

Data Logger Touchpad

Choose Setup Method

ArtSU3/ArtHalf

Manual Method Step 1 Data Logger Setup.

Once you have obtained the Unlock Code. Use the Data Logger Touchpad to press ENTER (✓). Then select the Manual method of setting up the Data Logger.

MANUAL METHOD

Manual Method Step 2: Enter Unlock Code.

Using the Data Logger Touchpad, enter the Unlock Code obtained in Step 9.

ArtSU3/ArtHalf

Manual Method Step 3: Code Acceptance.

Press ENTER (✓) when the Unlock Code entry is accepted. If an error occurs, press BACK and return to the main menu.

ArtSU3/ArtHalf

Equipment Alert

Ensure sterilizer serial number being assigned to Data Logger is correct so sterilization data records will be maintained accurately.

Manual Method Step 4: Enter Serial Number.

Enter sterilizer serial number by using PLUS, MINUS, NEXT/DONE and BACK buttons. Serial Number can be up to 9 characters. If Serial Number is shorter than 9 characters navigate to the blank space (between A and 9) and press NEXT/DONE, or press ENTER (✓) after the last character is entered.


ArtSU3/ArtHalf

Note

Sterilizer Name will be part of electronic record on receipt. Setting up sterilizer name is optional.


Manual Method Step 5: Set Sterilizer Name.

Press YES to name the sterilizer. Enter sterilizer name by using PLUS, MINUS, NEXT/DONE and BACK buttons. Or press NO to proceed to SETUP USER menu.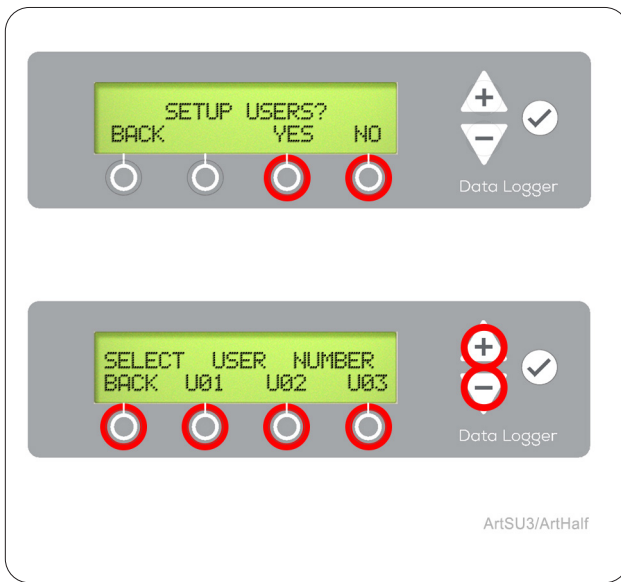


Manual Method Step 6: Setup Users.

From 1 to 12 users (designated U01-U12) can be set up in the Data Logger. Use PLUS, MINUS to scroll through the user identifiers and select them for set up.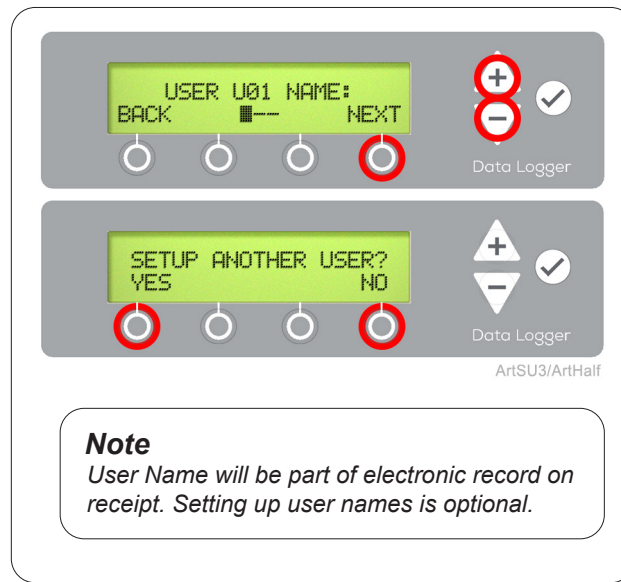


Manual Method Step 7: Setup User Name.

Enter a unique 1 to 3 character identifier for each user of the sterilizer. When all users have been entered and identified, select "No" when prompted to set up another user.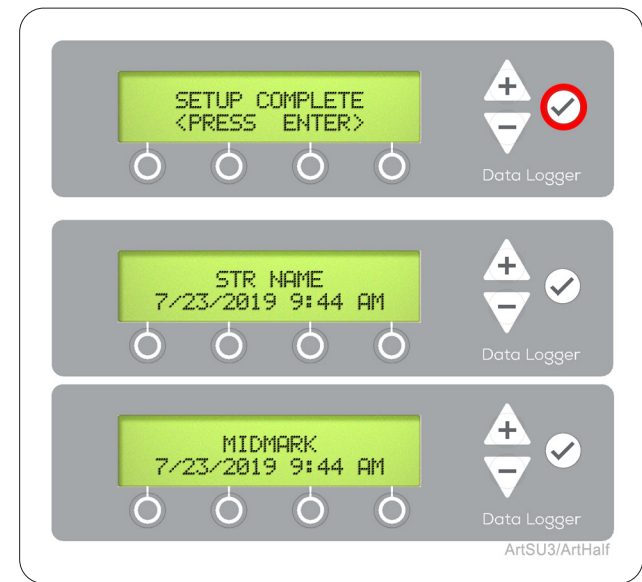


Manual Method Step 8: Setup Complete.

Press ENTER (✓) to complete setup. Main menu will appear with either sterilizer name if you entered one, or MIDMARK will appear as default. -04X models will not show Date/Time on Data Logger.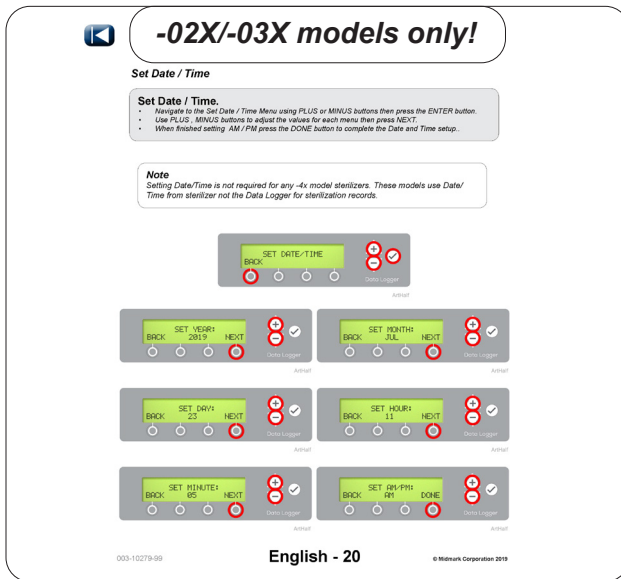

Manual Method Step 9: Set Date/Time.

Sterilizer Models -02X and -03X require navigating to the Additional Menus and setting up the Date / Time parameters. Refer to user guide for details.

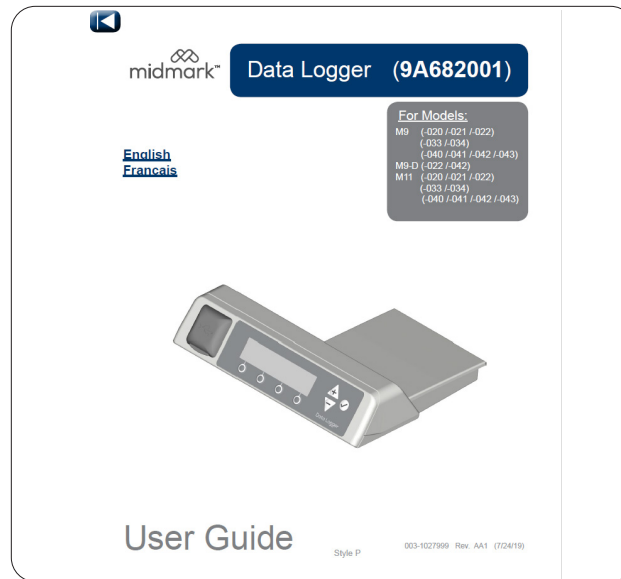Refer to User Guide.

Refer to user guide for additional menus and operation.

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003-10279-99 English - 20 © Midmark Corporation 2019