Data Logger (9A682001)

[English](#)
[Français](#)

Applies to Models:

M9[®] (-020 /-021 /-022)
(-033 /-034)
(-040 /-041 /-042 /-043)
M9D (-020 /-022 /-042)
M11[®] (-020 /-021 /-022)
(-033 /-034)
(-040 /-041 /-042 /-043)
M11D (-020 /-022)

User Guide

Style P

003-10279-99 Rev. AA8 (6/20/22)

Table of Contents

Important Information

<i>Safety Instructions</i>	3
<i>Electromagnetic Interference</i>	3
<i>Safety Symbols</i>	4
<i>Symbol Glossary</i>	4
<i>Transportation / Storage Conditions</i>	4
<i>Intended Use</i>	5
<i>Capabilities</i>	5
<i>Identify Compatible Sterilizer Models</i> ...	5
<i>Controls and Indicators</i>	6
<i>USB Drive</i>	7

Operation

<i>Home Screen</i>	8
<i>During the Cycle</i>	8
<i>Cycle Complete</i>	9
<i>Cycle Accepted by Operator</i>	9
<i>Cycle Rejected by Operator</i>	10

Additional Menus

<i>Menu Navigation</i>	11
<i>Product Info Menus</i>	11
<i>View Data Logger Serial Number</i>	11
<i>View Sterilizer Serial Number</i>	11
<i>View Data Logger Firmware Version</i>	11
<i>Set Date / Time</i>	12
<i>Set User Setup</i>	13
<i>Set Sterilizer Name</i>	14
<i>Set Sterilizer Serial Number</i>	15
<i>Sample Sterilization Cycle Log (PDF)</i> .16	

Specifications

<i>Operating Environment</i>	18
------------------------------------	----

Maintenance / Troubleshooting

<i>Maintaining Your Data Logger</i>	18
<i>Calling for Service</i>	18
<i>Troubleshooting / Error Codes</i>	19

Warranty

<i>Scope of Warranty</i>	21
--------------------------------	----

Important Information

Safety Instructions

The primary concern of Midmark is that this equipment is operated and maintained with the safety of the patient and staff in mind. To assure safe and reliable operation:

- Read and understand this manual before attempting to setup or operate the Data Logger.
- Assure that the appropriate personnel are informed on the contents of this manual.

Electromagnetic Interference

The Midmark® Data Logger is intended for use in the electromagnetic environment as specified in the Midmark Sterilizer User Guide. The customer or the user of the Midmark® Data Logger should assure that it is used in such an environment.

This product is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this product:

- Remove interfering device from room.
- Plug device into isolated circuit.
- Increase separation between Data Logger and interfering device.
- Contact Midmark Corporation if interference persists.

WARNING

Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

WARNING

Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Equipment Alert

Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

Safety Symbols

WARNING

Indicates a potentially hazardous situation which could result in serious injury.

Caution

Indicates a potentially hazardous situation which may result in minor or moderate injury. It may also be used to alert against unsafe practices

Equipment Alert

Indicates a potentially hazardous situation which could result in equipment damage.

Symbol Glossary

Proper shipping orientation

Keep dry

Fragile

Catalogue Number

Date of Manufacture

Serial Number

Manufacture

Transportation / Storage Conditions

Ambient Temperature Range:	-22°F to 140°F (-30°C to +60°C)
Relative Humidity:	10% to 90% (non-condensing)
Atmospheric Pressure:	7.2 psia to 15.4 psia (49.6 kPa to 106.4 kPa)

Intended Use

The Midmark® Data Logger is designed to connect with a sterilizer and electronically store sterilizer cycle data such as: sterilizer serial number, date and time the cycle was run, sterilization cycle parameters, errors and the cycle approving authority.

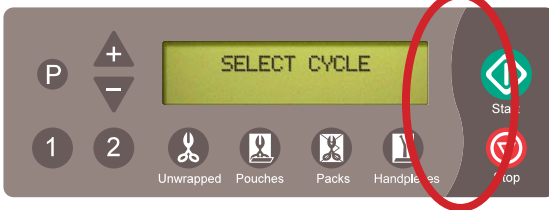
Capabilities

The Data Logger accessory is compatible with Midmark®/Ritter® M9®/M9®D-02X, M11®/M11®D-02X sterilizers with version 1.0.3 or later operating software and all Midmark®/Ritter® M9®/M9®D-03X/-04X, M11®/M11D/-03X/-04X sterilizers. The Data Logger creates and stores an electronic record of the sterilization cycle data with date and time, sterilizer identification (8 character maximum) and the approving operator (3 character maximum). The Data Logger stores the data and automatically downloads a pdf file to the included USB for easy transfer to a PC for access and backup. Additionally, the Data Logger stores up to 12 operator IDs for easy access by multiple operators.


NOTE: The Data Logger cannot be used in conjunction with printers 9A259001 or 9A599001.

Identify Compatible Sterilizer Models.

-02X and -03X Model Sterilizer


-04X Model Sterilizer

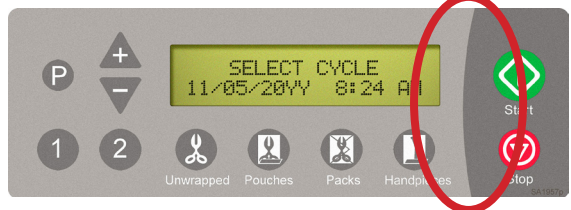
The Data Logger is compatible with most -02X and all -03X, -04X model sterilizers. Use the images to identify sterilizer models when needed.

Note

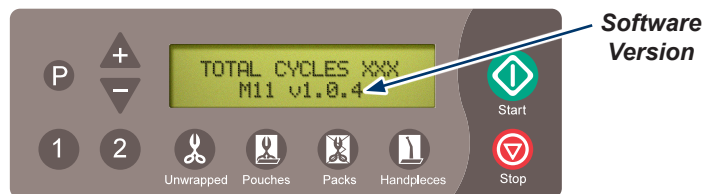The -02X, model sterilizers must have version 1.0.3 to 1.0.4 operating software for the Data Logger to be compatible. Please confirm your sterilizer has compatible software. To determine the software version of your sterilizer, cycle the power on the unit. The software version will be displayed during the start up cycle. If your sterilizer has different operating software, contact your Midmark representative for options.


Controls and Indicators

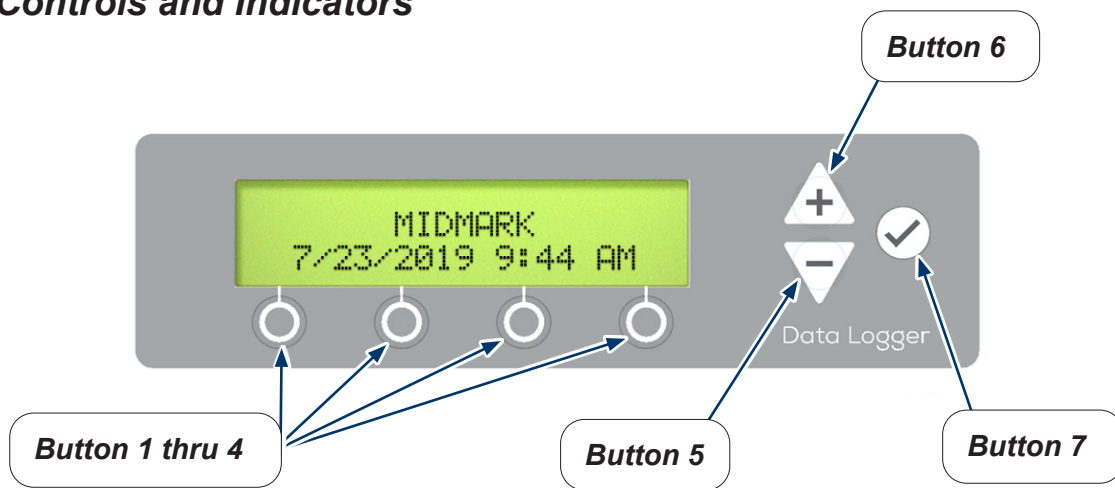The different keys perform the following functions:

- Button 1 thru 4:** Selects the function directly above it on the display when available.
- Button 5: (Minus)** – Scrolls down through the menus or values.
- Button 6: (Plus)** – Scrolls up through the menus or values.
- Button 7: (Enter)** – Selects value or advances menu.

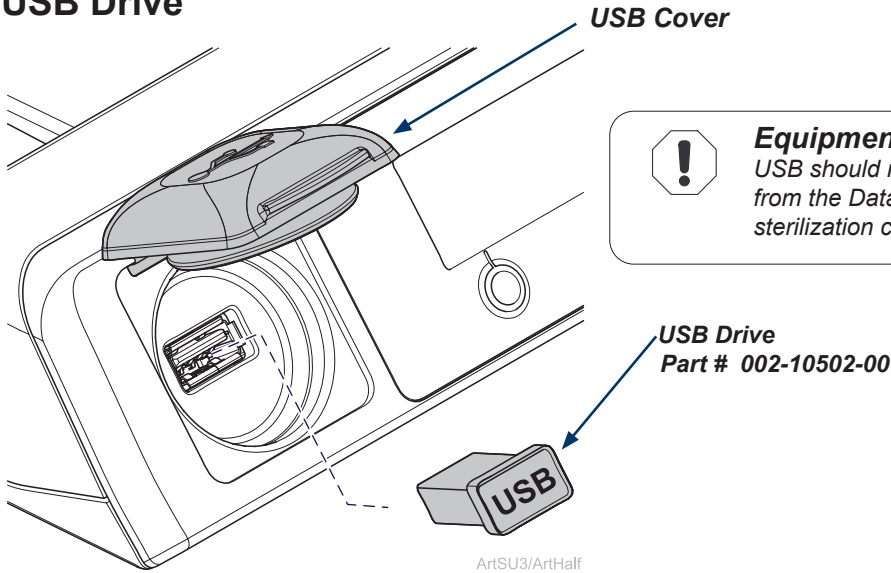USB Drive

Equipment Alert

USB should not be removed from the Data Logger while a sterilization cycle is in progress.

USB Drive.

- To ensure compatibility only use USB drives provided by Midmark for the Data Logger.
- Midmark recommends backing up contents of the Data Logger to a PC daily, or at least weekly at a minimum.
- The USB drive is used to transfer data from Data Logger to PC for backup.
- Ensure USB Cover is closed while Sterilizer and Data Logger are in use.
- If utilizing a brand new USB Drive it may take more time to transfer all the files (depending on number of cycles ran). This is a one time event for each new USB drive.

Transferring data from Data Logger to PC for backup.

- Remove USB from the Data Logger.
- Plug USB into a PC.
- Navigate to the File Explorer on the PC.
- Open up the USB Drive Folder on the PC.
- Navigate to the correct folder labeled as the sterilizer serial number.
- Transfer data to the PC.
- Consult with an appropriate IT professional for support with file transfer if required.

Equipment Alert

After reinstalling USB into Data Logger allow all files to copy prior to using sterilizer.

Operation

Home Screen.

- One of these main menus will appear on the Data Logger when it is ready for a cycle to be run. The default screen will show MIDMARK unless the sterilizer name was setup.
- -04X models do not show the date and time on the Data Logger display.

-02X and -03X Models

ArtHalf

ArtHalf

-04X Models

ArtHalf

ArtHalf

During the Cycle.

- Data Logger indicates sterilization cycle progress by flashing the bar under the active cycle phase.

ArtHalf

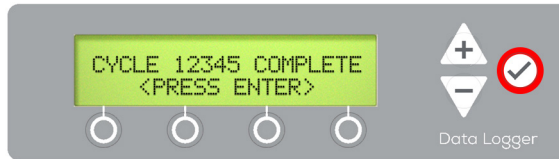
Caution

If user stops a cycle during the dry cycle phase, the instruments will need to be inspected to ensure they are dry prior to accepting load.

Cycle Complete.

- When sterilization cycle is complete press ENTER (✓) then select your operator sign off button.

Note

The cycle complete screen displays the current completed sterilization cycle count. The sterilization cycle count is not included on this screen for early versions of the Data Logger.

Cycle Accepted by Operator.

- To accept cycle press ACCEPT button then press DONE button. Cycle information will be recorded by Data Logger, Sterilizer is ready to run the next cycle.

Note

After cycle completes, open the door and examine the condition of the instruments. Accept or Reject the sterilization load based on your sterilization protocols. Qualified personnel responsible for infection control should prepare a protocol for handling sterilized items. This protocol should be followed by all personnel responsible for handling sterilized items.

Cycle Rejected by Operator.

- To reject cycle press **REJECT** button then press **PLUS** or **MINUS** button to navigate to the failure reason and **ENTER** (✓). Cycle information will be recorded by **Data Logger**, Sterilizer is ready to run the next cycle.

ArtHalf

ArtHalf

ArtHalf

ArtHalf

ArtHalf

Additional Menus

Menu Navigation.

- PLUS, MINUS changes characters.
 - Press and hold to scroll through available characters when entering data.
- NEXT advances to next field.
- DONE advances to next menu when entering data is complete.
- BACK deletes the previous character/returns to the previous menu.

Blank Spaces

To enter a blank space in a character string on Data Logger, navigate to blank character located between letter A and number 9 then press NEXT/DONE.

Pressing ENTER (✓)

Pressing ENTER (✓) at any point accepts all characters left of flashing cursor completing that step. This could result in unwanted blank fields.

Product Info Menus.

- Navigate to the Product Info menu using the MINUS button then press ENTER (✓).
- Use PLUS, MINUS buttons to view Data Logger Serial Number, Sterilizer Serial Number and current Firmware Version.
- Press BACK button once to return to the previous menu or twice to return to the main menu.

ArtHalf

ArtHalf

ArtHalf

ArtHalf

Set Date / Time.

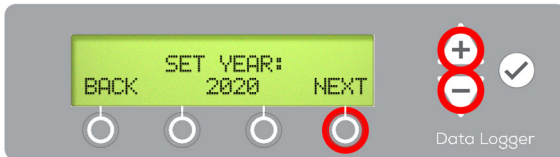
- Navigate to the Set Date / Time menu using the MINUS button then press ENTER (✓).
- Use PLUS, MINUS buttons to adjust the values for each menu then press NEXT.
- When finished setting AM / PM press the DONE button to complete the Date and Time setup.

Note

Setting Date/Time is not required for any -04X model sterilizers. These models use Date/Time from sterilizer not the Data Logger for sterilization records.

ArtHalf


ArtHalf


ArtHalf

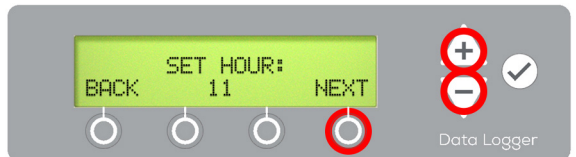
ArtHalf

ArtHalf

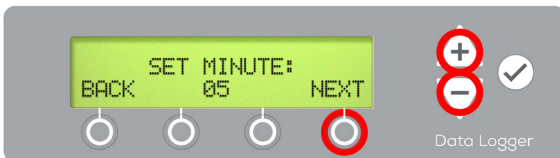ArtHalf

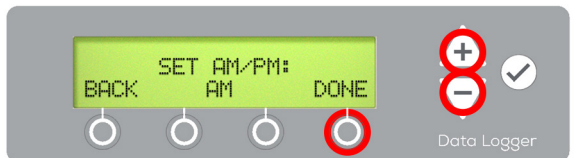ArtHalf

User Setup.

- Navigate to the User Setup menu using MINUS button then press ENTER (✓).
- From 1 to 12 users (originally designated U01-U12) can be set up in the Data Logger. Use PLUS, MINUS to scroll through the user identifiers to select one for setup.
- Enter a unique 1 to 3 character identifier for each user of the sterilizer using PLUS, MINUS, NEXT, DONE. When all users have been entered and identified, select "No" when prompted to set up another user.

ArtHalf

ArtHalf

Data Logger

ArtSU3/ArtHalf

Note

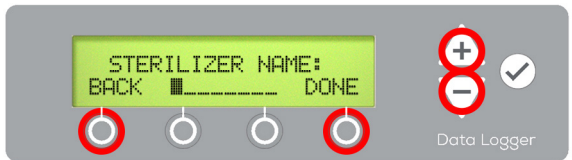
User Name will be part of electronic record on receipt. Setting up user names is optional.

Set Sterilizer Name.

- Navigate to the Set Sterilizer Name menu using MINUS button then press ENTER (✓).
- Enter sterilizer name by using PLUS, MINUS, NEXT/DONE and BACK buttons.
- Sterilizer name can be from 1 to 8 characters.

ArtHalf

ArtHalf

Note

Sterilizer Name will be part of electronic record on receipt. Setting up sterilizer name is optional.

Set Sterilizer Serial Number.

- Navigate to the Set Sterilizer Serial Number menu using MINUS button then press ENTER (✓).
- Enter sterilizer serial number by using PLUS, MINUS, NEXT/DONE and BACK buttons.
- Serial Number can be up to 9 characters. If Serial Number is shorter than 9 characters navigate to the blank space (between A and 9) and press NEXT/DONE, or press ENTER (✓) after the last character is entered.

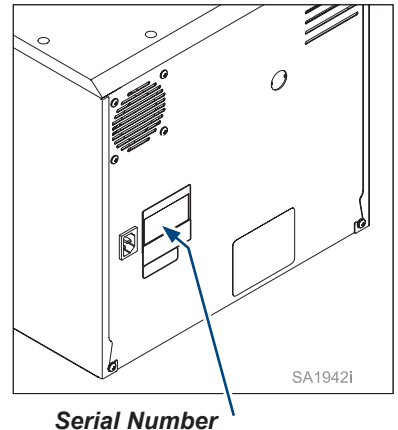
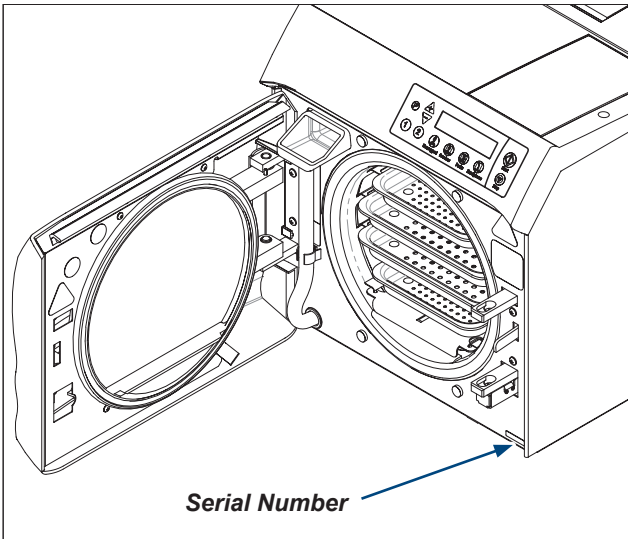
ArtHalf

ArtHalf

Note

Ensure sterilizer serial number being assigned to Data Logger is correct so sterilization data records will be maintained accurately.

Sterilization Cycle Log (PDF)

Normal Cycle Log.

- This is an example of a completed and approved sterilization cycle.

Sterilizer Name → Midmark M11 - v1.0.4
Model # and Software Version → Midmark M11 - v1.0.4
Sterilizer ID (Serial Number) → Name: MIDMARK
 Sterilizer ID: V2072130
Total # of cycles run on unit → Total Cycles: 41
Date and Time of cycle start → 01/16/202X 08:54 AM
Selected Cycle → BEGIN UNWRAPPED CYCLE
Sterilization temperature, time and dry time settings → Temp: 270 Degrees F
 Time: 3 Minutes
 Dry: 1 Minutes
Indicates Filling phase initiated → FILLING CHAMBER

During Heating phase, the Data Logger records chamber temperature and pressure in 30 second increments

Note
 The Data Logger stores detailed descriptions of each sterilization cycle as printable PDF's. The Data Logger stores these files in automatically created folders by date.

mm:ss	Degrees	PSI
0:00	66.2 F	0.0
0:30	66.3 F	0.0
1:00	66.6 F	0.0
1:30	68.1 F	0.0
2:00	71.4 F	0.0
2:30	77.2 F	0.0
3:00	87.8 F	0.0
3:30	89.4 F	0.0
4:00	90.7 F	0.0
4:30	92.6 F	0.0
5:00	93.3 F	0.0
5:30	100.7 F	0.0
6:00	110.1 F	0.0
6:30	120.4 F	0.0
7:00	132.5 F	0.0
7:30	145.9 F	0.0
8:00	155.3 F	0.4
8:30	161.7 F	1.2
9:00	167.9 F	2.0
9:30	173.8 F	3.0
10:00	178.9 F	4.0
10:30	182.5 F	4.9
11:00	187.1 F	5.9
11:30	191.7 F	6.9
12:00	195.9 F	7.9
12:30	199.7 F	8.9
13:00	203.4 F	9.8
13:30	207.4 F	10.9
14:00	210.4 F	11.9
14:30	213.1 F	12.9
15:00	213.3 F	9.8
15:30	216.5 F	6.0
16:00	219.6 F	5.8
16:30	222.9 F	6.8
17:00	225.9 F	7.8
17:30	228.6 F	8.8
18:00	231.4 F	9.8
18:30	233.9 F	10.9
19:00	236.7 F	12.0
19:30	239.3 F	13.2
20:00	236.6 F	10.3
20:30	233.4 F	8.1
21:00	235.9 F	9.5
21:30	239.2 F	11.0
22:00	242.3 F	12.4
22:30	244.0 F	12.5
23:00	239.2 F	10.0

Continued:

23:30	237.9 F	9.6
24:00	241.5 F	11.4
24:30	245.1 F	13.1
25:00	248.4 F	14.7
25:30	251.5 F	16.4
26:00	254.6 F	18.0
26:30	257.4 F	19.6
27:00	260.3 F	21.3
27:30	263.1 F	23.0
28:00	265.7 F	24.7
28:30	268.5 F	26.4
29:00	271.0 F	28.2
29:30	272.6 F	28.9

STERILIZING

mm:ss	Degrees	PSI
0:00	271.6 F	28.6
0:30	271.9 F	28.5
1:00	272.3 F	28.8
1:30	272.6 F	29.1
2:00	272.9 F	29.3
2:30	272.7 F	29.4
3:00	272.2 F	29.0
Min	271.6 F	28.4
Max	272.9 F	29.4

VENTING CHAMBER

DRYING

mm:ss	Event
0:00	Drying Start
1:00	Drying Complete

Cycle summary

FILLING:	0:40
HEATING:	29:44
STERILIZING:	3:00
VENTING:	2:56
DRYING:	1:00
TOTAL CYCLE:	00:37:19

CYCLE COMPLETE

Operator summary load accepted

01/16/202X 09:32 AM
ACCEPTED: X
REJECTED:
Operator: A02

Sterilizer Cycle Log With Error Code (example).

- ITEMS NOT STERILE. When an error code occurs during a cycle the load being sterilized is not sterile.
- Example below shows the stop button was pressed during the sterilization cycle causing an error code C104 and interrupting the cycle.
- Parameters at the time of the error are shown which could potentially help with troubleshooting processes. Date and time error occurred is also shown.

Continued:

STERILIZING		
mm:ss	Degrees	PSI
0:00	271.6 F	28.6
0:30	271.9 F	28.5
1:00	272.3 F	28.8
1:30	272.6 F	29.1
2:00	272.9 F	29.3
2:30	272.7 F	29.4
C104: STERILIZE MODE STOP PRESSED		
CHAMBER TEMP 273F		
CHAMBER PRESS 29.0PSI		
ELAPSED TIME 02:38		
** ITEMS NOT STERILE **		
10/20/202X 03:52 PM		

During Sterilization phase, the printer records chamber temperature and pressure in 30 second increments

Error code and description

Cycle parameters at time of error

Items Not Sterile

Date and time of error

Sterilizer Cycle Log Rejected By Operator.

- Example below shows the sterilization cycle was rejected by the operator due to a failed indicator.
- The parameters at the time of the error are shown which could potentially help with troubleshooting processes. The date and time the error occurred is also shown.

Continued:

FILLING:	0:40
HEATING:	29:44
STERILIZING:	3:00
VENTING:	2:56
DRYING:	1:00
TOTAL CYCLE:	00:37:19
CYCLE COMPLETE	
01/16/202X 09:32 AM	
ACCEPTED:	
REJECTED: X	
Reason: Fail Indicator	
Operator: A02	

Cycle summary

Operator summary
load rejected due to failed indicator.

Specifications

Operating Environment

Ambient Temperature	68°F to 104°F (+20°C to +40°C)
Relative Humidity	< 80% (non-condensing)
Normal Operating Altitude:	< 9842 ft. (3000 m) above sea level
Operating Environment	Indoor use only

Maintenance / Troubleshooting

Maintaining Your Data Logger

Clean External Surfaces

- A) *Clean the exterior of the Data Logger each day according to your facility's procedure for clinical contact surfaces, noting the following: (Use only quaternary disinfectants to disinfect unit. Staining, pitting, discoloration, or softening could occur if phenolic, iodophor, or glutaraldehyde-based disinfectant is used on plastic surfaces of the unit. Also, use of alcohol or aerosol spray cleaner / disinfectant containing substantial amounts of alcohol in the formula can damage the faceplate).*
- B) *Wring excess solution from the cloth.*
- C) *Using soft cloth, wipe all external surfaces.*
- D) *Follow the instructions provided with the cleaner / disinfectant used regarding rinsing and drying of the external surfaces.*

Equipment Alert

Make sure the USB cover is closed when cleaning to prevent entry of cleaners/moisture into the USB cavity.

Calling for Service

Contact your Midmark Authorized Dealer, or log on to www.midmark.com. Model and serial number information will be required when calling for service and can be found on the back of the unit.

To contact Midmark directly:

1.800.Midmark (1.800.643.6275) or 937.526.3662
8:00 am to 5:00 pm ET (Monday thru Friday)
(Excluding standard US holidays)

Troubleshooting / Error Codes

Display / Symptom	Cause	Check/ Action
Memory Error	Error trying to read or write configuration data internal to Data Logger.	Cycle power to Data Logger.
		If error persists, replace Data Logger.
SD Card Error	Failure to read or write to SD card or SD card has become disconnected.	Cycle power to Data Logger.
		If error persists, replace Data Logger.
USB Drive Error	Failure to read or write to USB drive.	Make sure USB drive is securely inserted.
		Cycle power to Data Logger.
		Replace USB Drive
		If error persists, replace Data Logger.
File Transfer Error	Failure to copy one or more files from SD card to USB drive.	Remove USB drive from Data Logger and re-insert. This will initiate the file transfer from SD card to USB drive.
		If error persists, replace USB drive.
	USB drive is disconnected during file transfer process.	Remove USB drive from Data Logger and re-insert. This will initiate the file transfer from SD card to USB drive.
		If error persists, replace Data Logger.
Clock Battery Error	Internal clock battery is either low on power or dead.	Replace Data Logger.
PDF Creation Error	Error creating pdf of sterilization cycle or saving file to SD card.	Run another sterilization cycle, if error persists, replace Data Logger.
Config File Copy Error	Data Logger is unable to read configuration parameters from config file on SD card.	Remove USB drive from Data Logger. Delete config file from USB drive and download a new config file onto the USB drive using online registration process. Insert USB drive into Data Logger and update config file.
		If error persists, replace USB drive with new Config file downloaded using online registration process.
		If error persists, replace Data Logger.
Firmware Update Error	Data Logger encountered an error while updating firmware.	Restart Data Logger with a USB drive that contains firmware update file to re-initiate firmware update.
		If error persists, get a new USB drive with a new firmware update file and retry.
		If error persists, replace Data Logger.

Troubleshooting / Error Codes - Continued...

<i>Display / Symptom</i>	<i>Cause</i>	<i>Check/ Action</i>
Button Stuck Error	Data Logger detects a button on membrane is stuck.	Try to manually unstick button and restart Data Logger.
		If the error persists, replace Data Logger.
Data Logger Display Does Not Go Past Fill	Sterilizer software version to Prior to 1.0.3	Sterilizer software version must be version 1.0.3 or newer (-02X models).
		Sterilizer serial numbers must be programed into all replacement sterilizer main PC the boards

Midmark® + Ritter® Medical Equipment (Tables, Chairs, Physician Seating, Lights, Workstations and Sterilizers) Limited Warranty

SCOPE OF WARRANTY Midmark Corporation ("Midmark") warrants to the original retail purchaser that it will, at Midmark's option, repair or replace components of the domestic and international medical products manufactured by Midmark (except for components not warranted under "Exclusions") that are defective in material or workmanship under normal use and service. The sole remedy under this limited warranty is the repair or replacement, at Midmark's option, of the applicable components. This limited warranty shall only apply to defects that are reported to Midmark within the applicable warranty period and which are determined to exist upon examination by Midmark. This warranty extends only to the original retail purchaser of a product and is not transferable or assignable. Replacement components or products may be used and/or refurbished components or products, provided they are of like quality and specifications as new components or products.

Midmark warrants to the original retail purchaser that during the applicable warranty period it will repair or replace software contained within the products manufactured by Midmark (except for those not warranted under "Exclusions") if: (1) the media on which the software is furnished exhibits defects in material or workmanship under normal use; or (2) the software does not substantially conform to its published specifications.

APPLICABLE WARRANTY PERIOD The applicable warranty period, measured from the date of invoice to the original retail purchaser of the product and shall be one (1) year for all warranted products and components. QuickClean® Ultrasonic Cleaners are warranted for a period of three (3) years.

OBTAINING WARRANTY SERVICE Warranty service must be obtained through either Midmark or an authorized dealer in the Midmark product line for which warranty service is requested. Midmark may be contacted for warranty service inquiries or issues via email at midmark.com; by phone at 1.800.MIDMARK or by mail to Midmark Corporation, 60 Vista Drive, Versailles, Ohio 45380. It is the retail purchaser's obligation to arrange for delivery of a product to Midmark or one of its authorized dealers for warranty service, which delivery shall be at retail purchaser's expense. It is also the retail purchaser's obligation to comply with the warranty service instructions provided either by Midmark or its authorized dealer. The retail purchaser must provide Midmark with completed warranty registration information within thirty (30) days after purchase in order to obtain the benefits of this limited warranty.

EXCLUSIONS: This limited warranty does not cover and Midmark shall not be liable for the following:

- (1) Defects, damage or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, negligent storage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- (2) Products which are not installed, used, and properly cleaned and maintained as required or recommended in the Midmark "Installation" and/or "Installation/Operation Manual" for the applicable product, including the specified structural and operational environmental conditions and electrical requirements;
- (3) Products considered to be of a consumable or sterile nature;
- (4) Accessories or parts not manufactured by Midmark;
- (5) Charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing in advance by Midmark;
- (6) Costs and expenses of routine maintenance and cleaning; and
- (7) Representations and warranties made by any person or entity other than Midmark.
- (8) Matching of color, grain or texture except to commercially acceptable standards;
- (9) Changes in color caused by natural or artificial light;
- (10) Custom manufactured products;
- (11) Alterations or modifications to the product by any person or entity other than Midmark; and
- (12) Products that would otherwise be covered under this limited warranty, but are acquired: (i) from a person or entity that is not Midmark or one of its authorized dealers; or (ii) from a Midmark dealer that is not authorized to sell the product at issue in the geographic territory where the purchaser is located, or is not authorized to sell the product at issue within the medical market.

SOFTWARE; WITH RESPECT TO SOFTWARE THAT IS A PRODUCT OR COMPONENT THEREOF, MIDMARK DOES NOT WARRANT THAT THE SOFTWARE: (1) IS ERROR FREE; (2) CAN BE USED WITHOUT PROBLEMS OR INTERRUPTIONS; OR (3) IS FREE FROM VULNERABILITY TO INTRUSION OR ATTACK BY VIRUSES OR OTHER METHODS.

EXCLUSIVE REMEDY; CONSEQUENTIAL DAMAGES DISCLAIMER MIDMARK'S ONLY OBLIGATION UNDER THIS LIMITED WARRANTY IS THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS. MIDMARK SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES OR DELAYS, INCLUDING, BUT NOT LIMITED TO, DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, LOSS OF DATA, DOWNTIME, COVER AND EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS. THIS DISCLAIMER SHALL SURVIVE ANY FAILURE OR ASSERTED FAILURE OF THE ESSENTIAL PURPOSE OF THIS LIMITED WARRANTY OR ITS REMEDIES SPECIFIED HEREIN. WARRANTY DISCLAIMER THIS WARRANTY IS MIDMARK'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. MIDMARK MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS.

STATUTE OF LIMITATIONS No action may be brought against Midmark for breach of this limited warranty, an implied warranty, if any, or for any other claim arising out of or relating to the products, more than ninety (90) days following expiration of the limited warranty period.

NO AUTHORIZATION No person or firm is authorized to create or approve for Midmark any other obligation or liability in connection with the products.

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M9® (-020/-021/-022)
(-033/-034)
(-040/-041/-042/-043)
M9D (-020/-022/-042)
M11® (-020/-021/-022)
(-033/-034)
(-040/-041/-042/-043)
M11D (-020/-022)

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