

# MIDMARK®

## Console Mounting Arm Installation Guide


### INSTALLATION

- (1) Raise chair base as high as it will go; then lift seat upholstery to access components under chair seat.
- (2) Secure mounting arm (1, Figure 1) to casting (2) **[located under chair seat]** with bolt (3) and lockwasher (4) **[from parts bag]**. Do not completely tighten bolt at this time.

### NOTE

Verify desired position of mounting arm (*LH or RH side*) with proper personnel prior to step 3.

- (3) Position mounting arm (1) as desired; then tighten two small hex bolts (5).
- (4) Tighten bolt (3) as specified.  
**[Torque: 65-75 ft/lbs.(88-102 N•m)]**

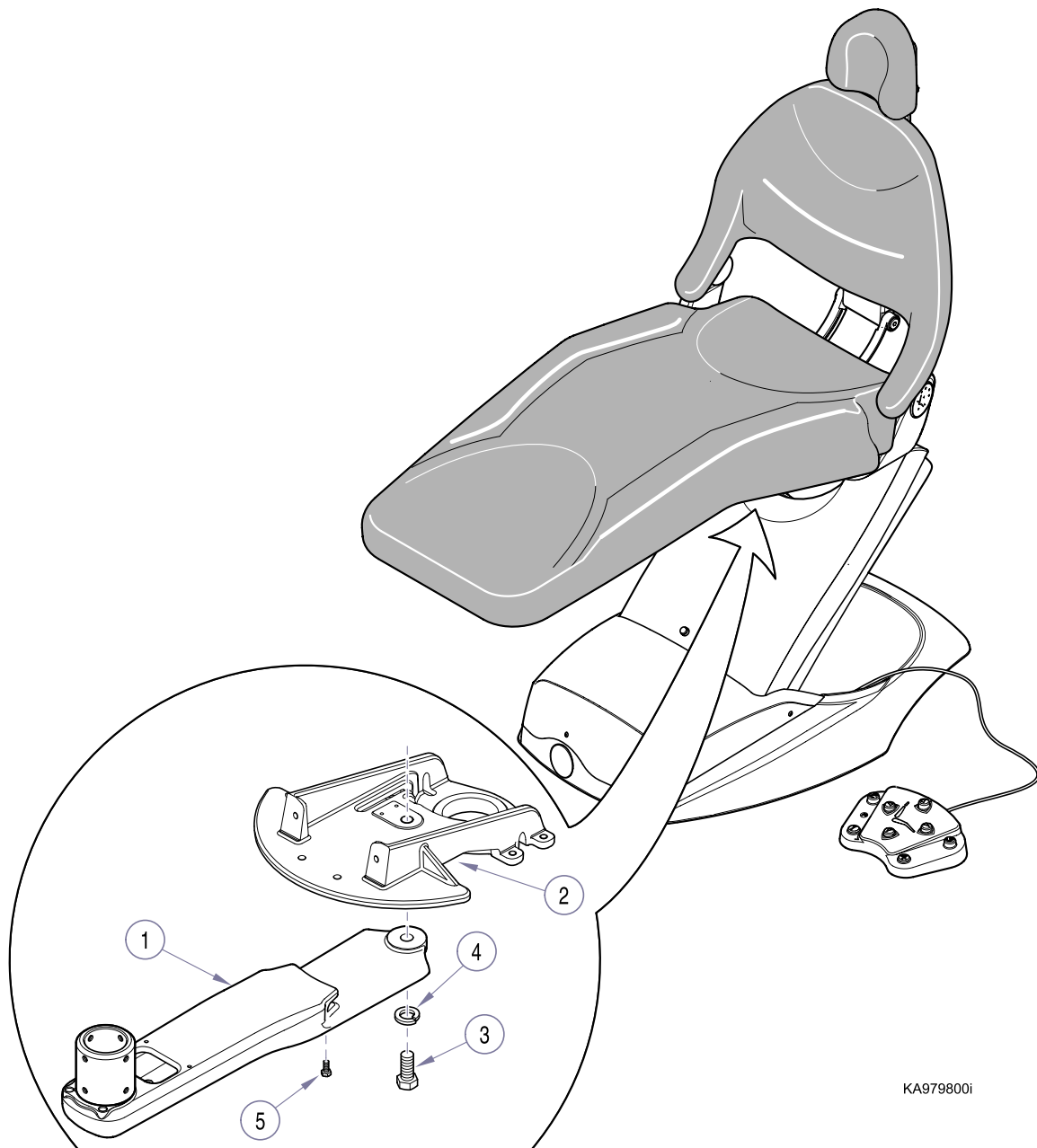

KA979800i

Figure 1. Console Mounting Arm Installation