


Midmark LR Chair Mount

Installation Guide [Midmark Dental Chair]

- (1) Raise chair base as high as it will go; then lift seat upholstery to access components under chair seat.
- (2) Secure mounting arm (1, Figure 1) to casting (2) [located under chair seat] with bolt (3) and washer (4) [from parts bag]. Do not completely tighten bolt at this time.
- (3) Position mounting arm (1) straight out from foot end of chair; then tighten two small hex bolts (5).
- (4) Tighten bolt (3) as specified. [Torque: 65-75 ft/lbs. (88-102 N•m)].
- (5) After removing protective paper from adhesive strip, install bolt cover (6).

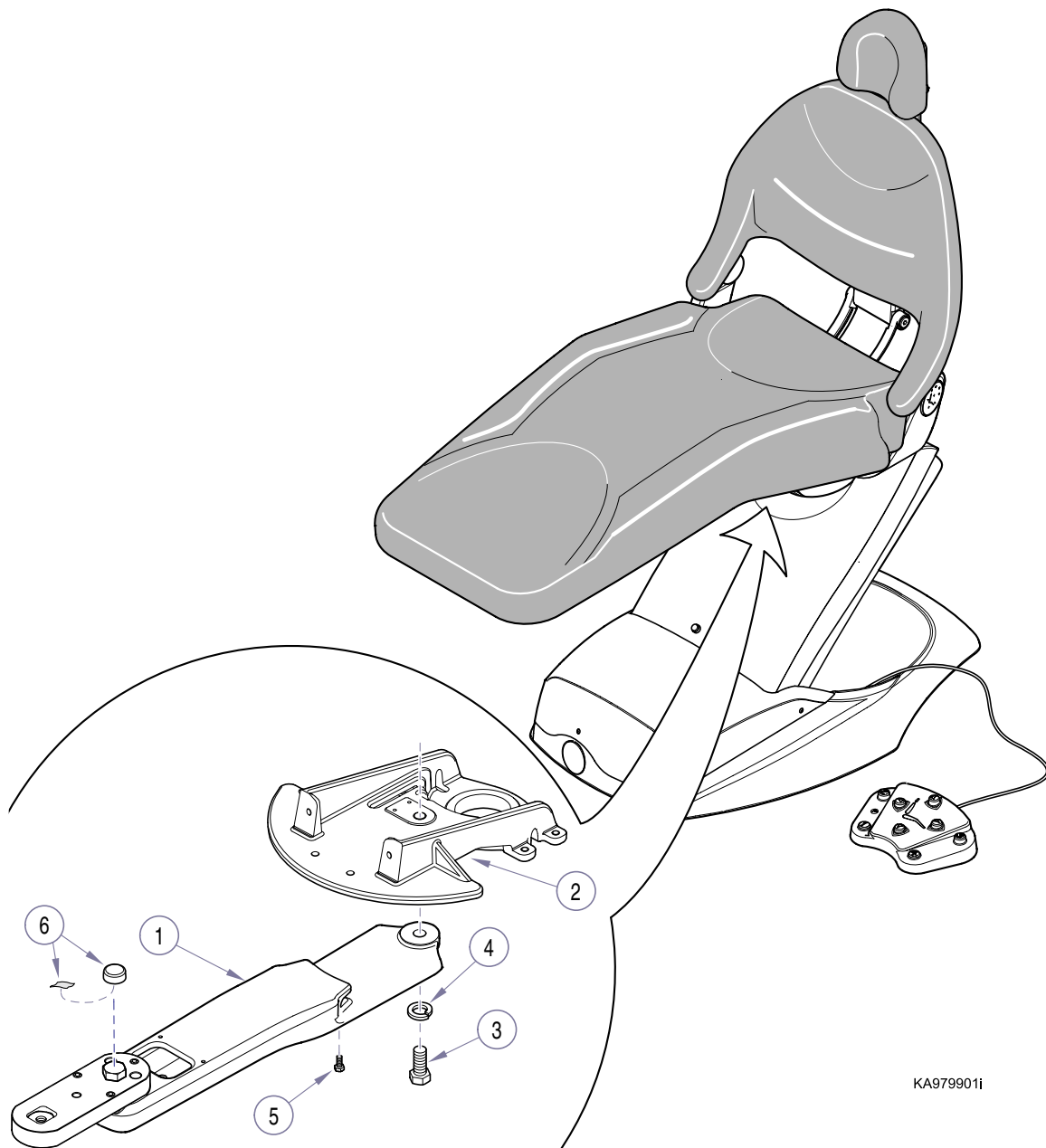

KA979901i

Figure 1. Mounting Arm Installation