




UPT Upholstery Installation - Midmark

Applies to Models:
630 (-001 thru -009)
75 (-020 / -021)
9A332
9A339


Language of origin: English

Note
Contoured Headrest Accessories for Ritter 230 models:
9A332001-xxx Standard upholstery
9A339001-xxx Cal 133 upholstery

To install headrest upholstery...
A) Insert mounting studs into holes in pivot mount.
B) Push three lock levers inward to lock in place.

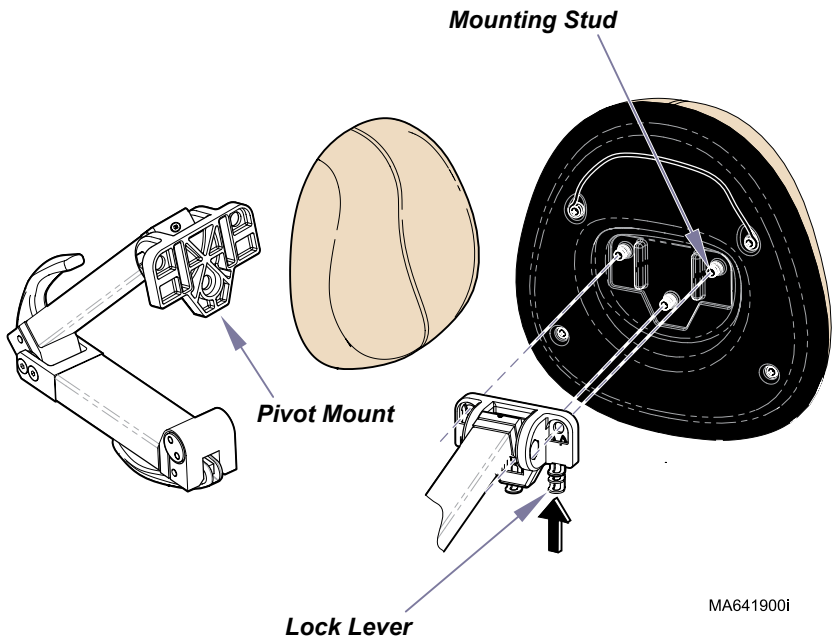

To install seat / back upholstery...
A) Position front of seat section against mounting board.
B) Lay upholstery flat, then press down firmly.

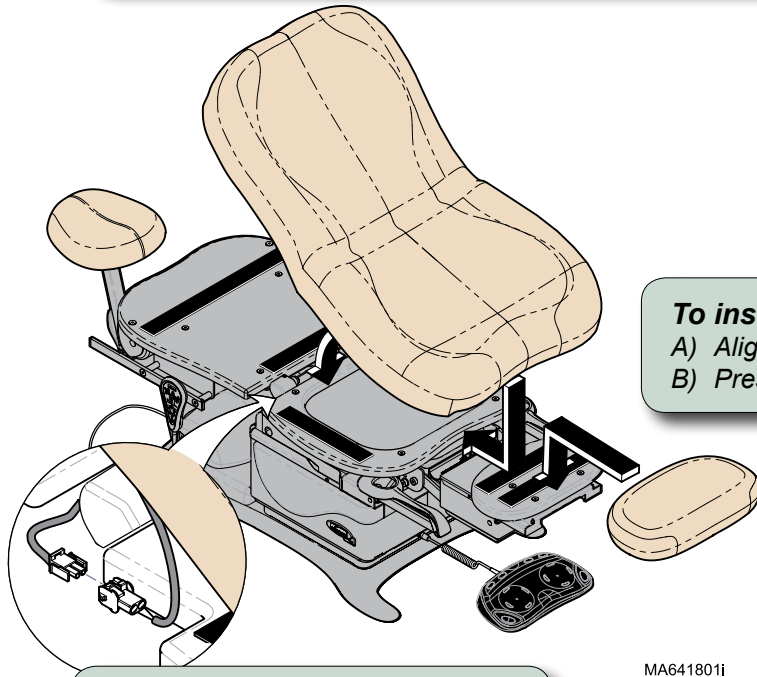

To install foot upholstery...
A) Align velcro strips.
B) Press down firmly.

Heated upholstery only:
Connect upholstery heater harness.