

# M9/M9D & M11/M11D PC Board Kit




<u>Kit Number</u>	<u>Used On</u>
002-1992-00	115 VAC, M9 / M11
002-1992-01	230 VAC, M9 / M11
002-1992-02	115 VAC, M9D / M11D

**Warning**  
 Always disconnect power before removing covers.

**Removal Step 1:** Tag and disconnect all wire harnesses from PC board.

**Installation Step 3:** Connect all wire harnesses to PC board.

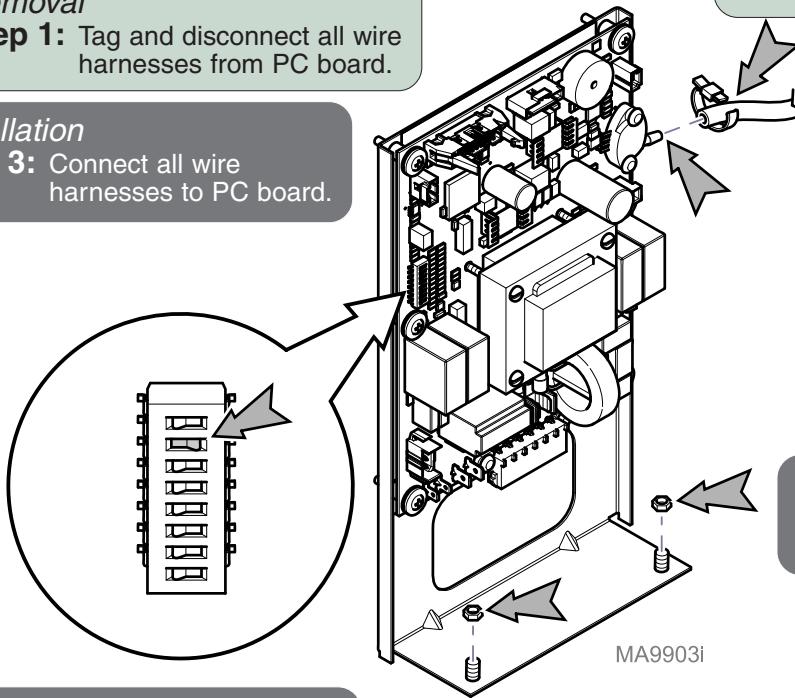

**Installation Step 2:** Connect tubing to pressure sensor. Secure with a high temp. cable tie.

**Removal Step 2:** Cut cable tie, then disconnect tubing from pressure sensor.

**Installation Step 1:** Install PC board / bracket. Secure with two nuts

**Removal Step 3:** Remove two nuts and PC board / bracket.

**Installation Step 4:** Unplug power cord. Move switch #2 to ON. Reconnect power cord.

**Installation Step 5:** Adjust the PC Board configuration by following the prompts on the display panel.  
 Use the <+> <-> buttons to adjust settings. Press the <P> button when finished.

**CHAMBER SIZE:**

M9/M9D .....	9 INCH
M11/M11D .....	11 INCH

**FEATURE SET:**

M9/M11 .....	FULLY FEATURED
M9D/M11D .....	DEFEATURED



CHAMBER DIAMETER:  
9 INCH

FEATURE SET:  
FULLY FEATURED

**Installation Step 6:** Unplug power cord. Move switch #2 to OFF. Reconnect power cord.