


Heating Element Kit M7 Sterilizer

002-0839-00.....120 VAC

002-0839-01.....230 VAC

Kit Components:

	Qty
Heating Element	1
Gaskets	2
Installation Instruction	1

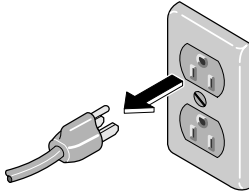

MA463100i



Warning

Disconnect Power and allow unit to cool before removing parts.

NOTE

Heating elements used in the M7-011 thru -016 can be replaced using the M7-020 thru -022 elements with matching voltage ratings.

Removal

- Step 1:**
- Disconnect power to sterilizer.
 - Drain all water from reservoir.
 - Remove Trays & Tray Plate.

Removal

Step 2: Remove inspection cover.

Installation

Step 5: Install inspection cover.

Installation

Step 4: Assure unit is clean then fill reservoir with distilled water and run a cycle. Check for leakage while the unit is under pressure..

Installation

Step 2: Install two flat washers, lockwashers, & nuts.

Note: Hold heating element in place when tightening nuts.

Installation


Step 3: Connect wires to heating element terminals. (Refer to **Section D** for wiring diagrams)

Removal

Step 4: Remove two nuts, lockwashers & flat washers.

Removal


Step 3: Tag & disconnect wires from heating element terminals.

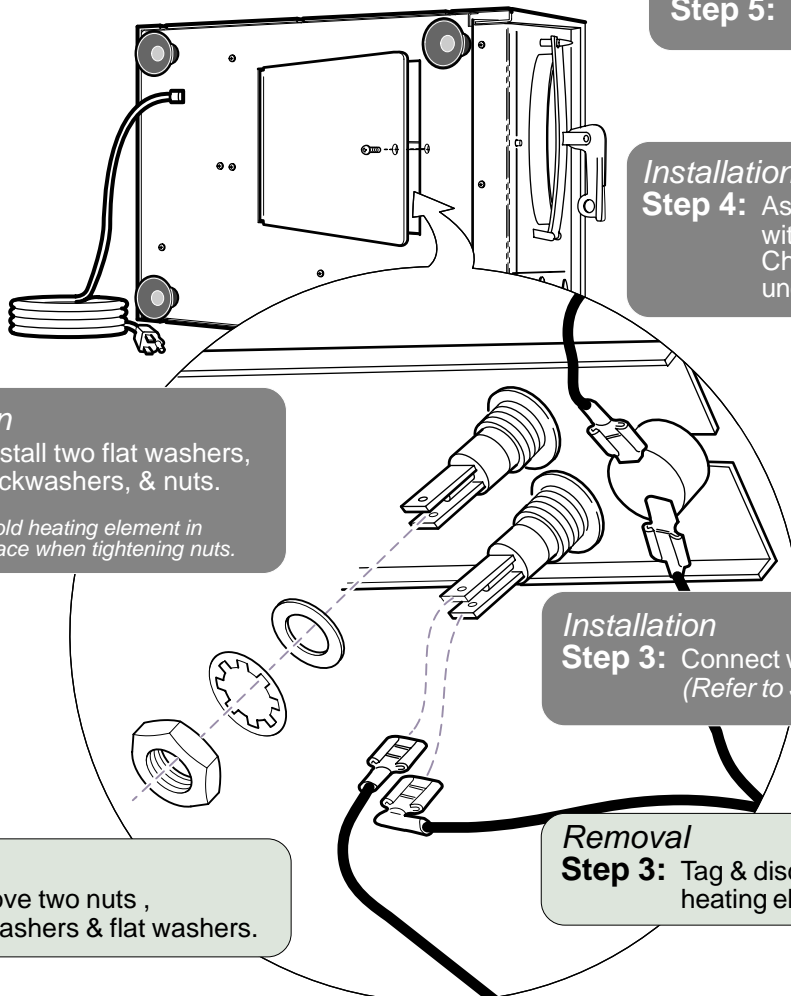

MA675000i


Removal

Step 5: Remove heating element.
Remove gaskets from terminal posts.

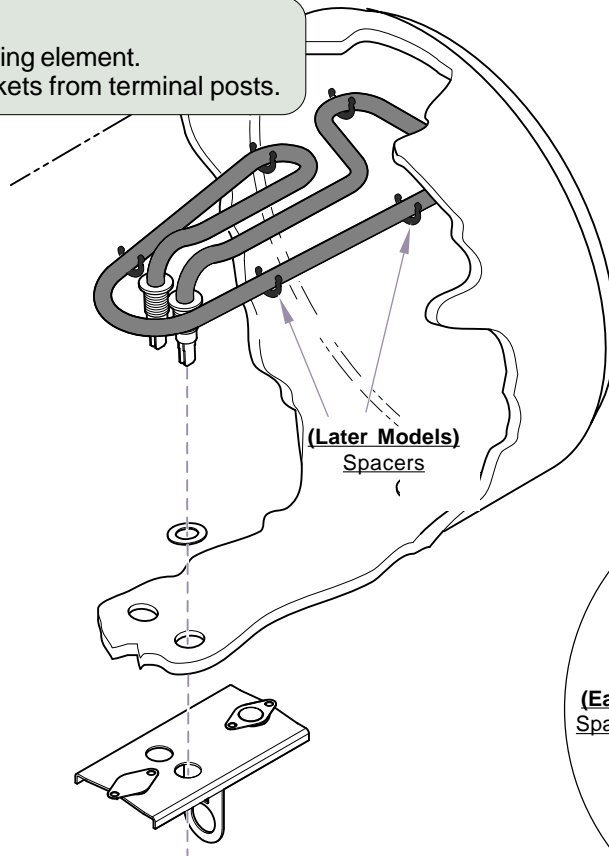

NOTE

When installing newer heating element in an Earlier Model M7, discard older style "T" Spacer Bracket as newer heating element has spacers attached to element.

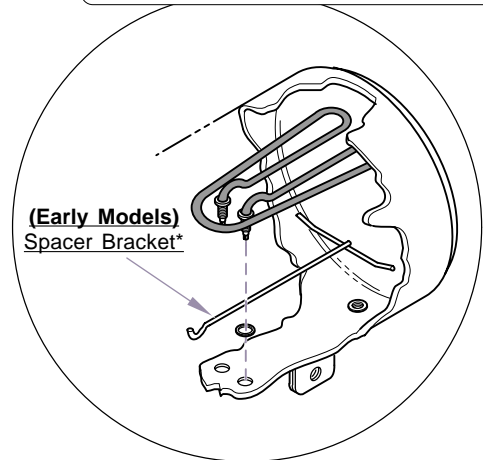

* Discard Spacer Bracket.

Installation

Step 1: Install gaskets onto terminal posts.
Insert terminal posts thru chamber & thermostat bracket.