

PC Board Calibration Procedure



Equipment Alert



Anytime the PC board is removed you **must** perform the Calibration Procedure. Failure to do so will cause the chair to malfunction.

If the chair has a retractable foot section, it **must** be in the stowed position before performing the Calibration Procedure.

Step 3: Calibrate PC board.
 Wait five seconds, then press S2 button.


Note: During calibration, you will hear a steady pattern of "beeps" as each function moves. When successful calibration is complete, the chair will stop moving, and you will hear three "beeps".

Step 1: Put PC board into Calibration Mode.
 A) Move MODE SELECT switch 1 to ON (up).
 B) Move MODE SELECT switch 2 to OFF (down).

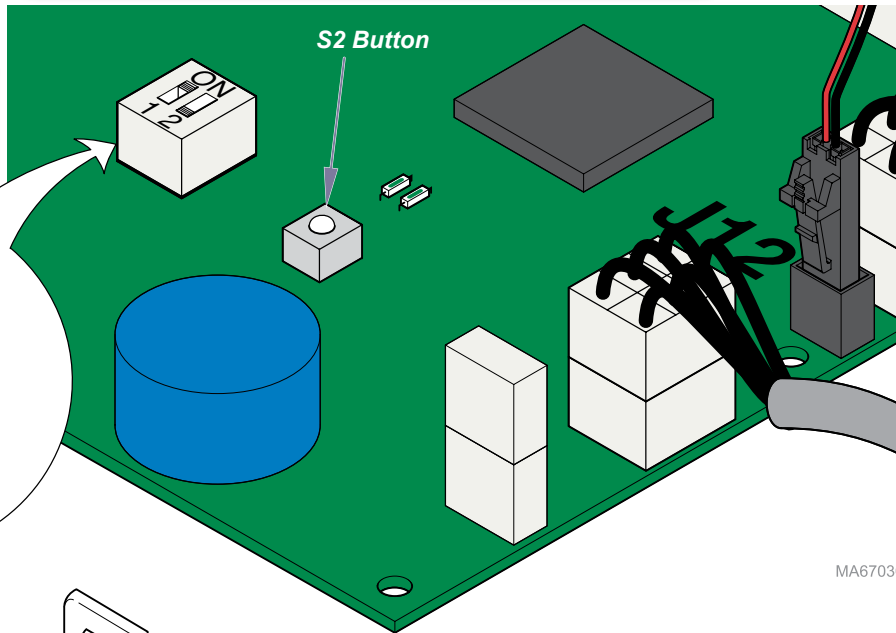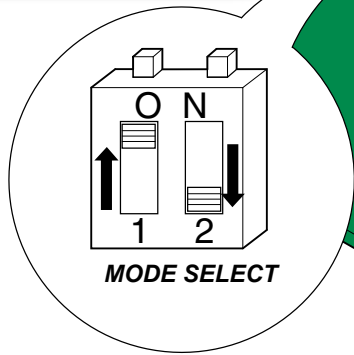

MA670300i

Step 2: Reset PC board.
 A) Unplug chair for a few seconds.
 B) Plug chair back in.

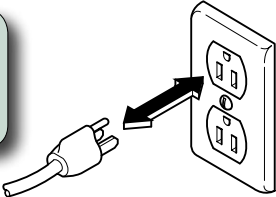

Step 4: Return to normal operating mode.
 A) Move MODE SELECT switches back to original position.
 B) Unplug chair for a few seconds (to reset PC board).
 C) Plug chair back in.