



Bellows / Plumbing Replacement Kit


[002-0375-01]

Components:


- Bellows 1
- Tubing 2
- Comp. Nuts 3

Special Tools:

none

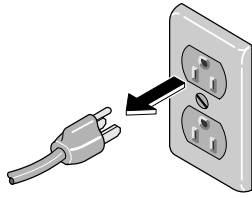


Warning
Disconnect Power
and allow unit to cool before
removing parts.

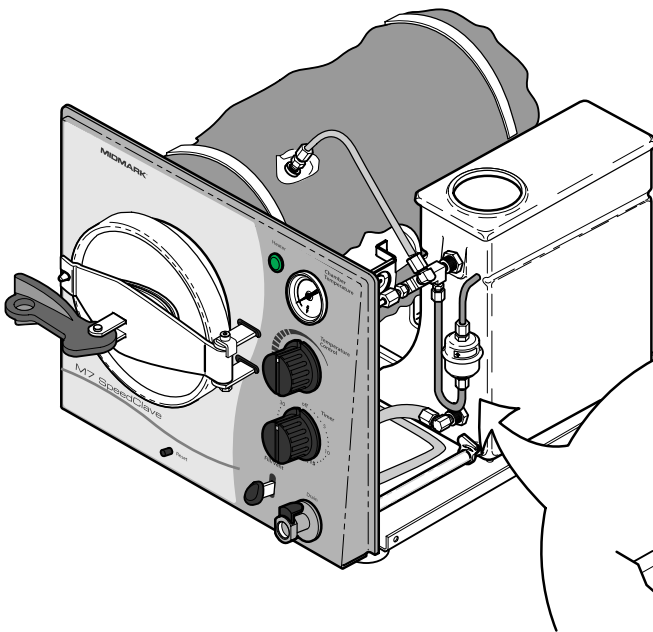

Removal
Step 1: • Disconnect power to sterilizer.

Note: Compression Fitting may spin, you may need to use another wrench to make sure it does not move.

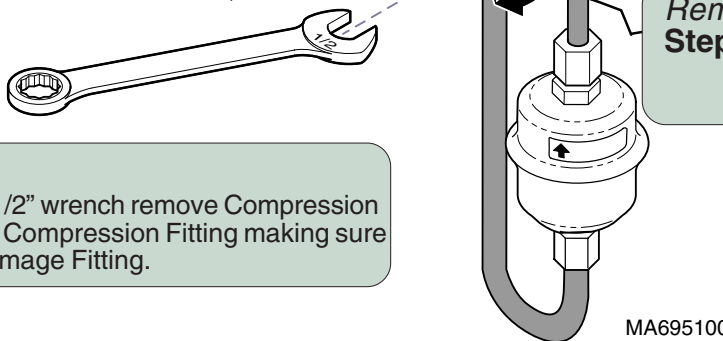


Removal
Step 3: Using a 1/2" wrench remove Compression Nut from Compression Fitting making sure not to damage Fitting.

Removal
Step 2: With covers removed from sterilizer, pull out the tube that is inserted into the tank.




Arrow **must** point toward the reservoir as shown

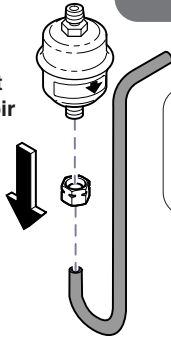



Installation
Step 1: Take Bellows and one Compression Nut and fasten to tubing as shown.

NOTE:
The arrow on the bellows indicates the direction of flow **toward the reservoir**. The bellows must be oriented correctly to ensure proper operation.

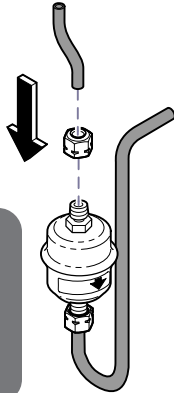

Installation
Step 2: Secure last piece of tubing and the second Compression Nut to other end of bellows making sure not to tighten so you can align with Compression Fitting.

Installation
Step 3: Take last Compression Nut and put on the end of the short tube.

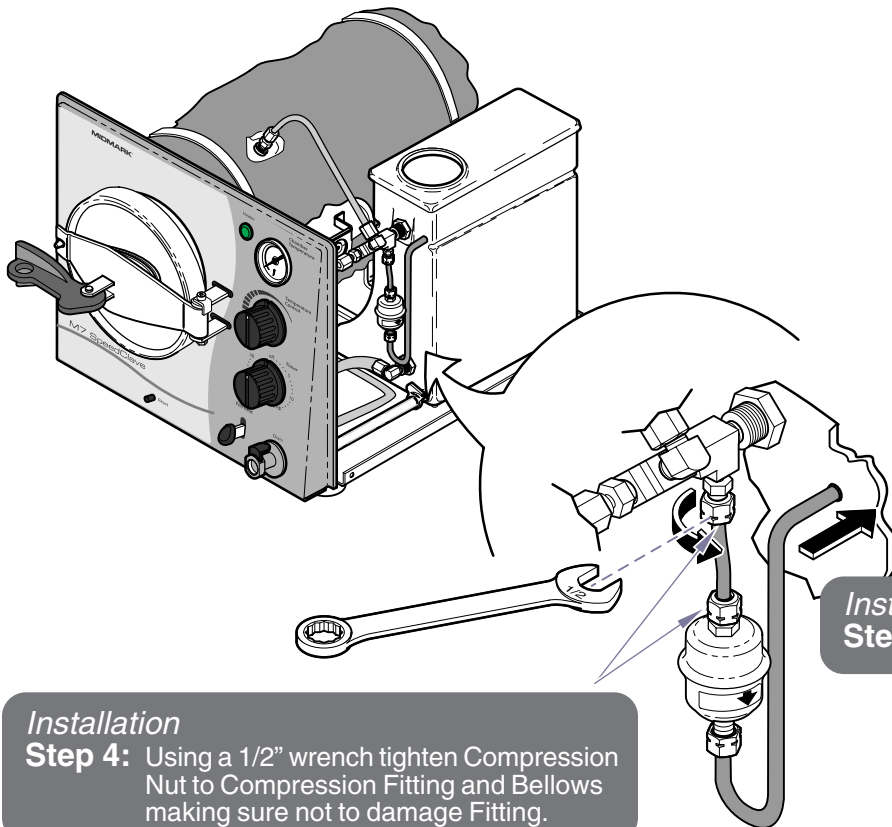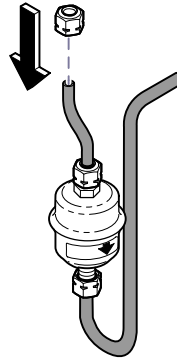

Installation
Step 4: Using a 1/2" wrench tighten Compression Nut to Compression Fitting and Bellows making sure not to damage Fitting.

Installation
Step 5: Inserted tubing into the tank.

MA695200i