



Paper Roll Holder [9A334, 9A355, 9A618]


Components:
Paper Roll Holder ... 1

Special Tools:
none

Intended Use:
To allow table to accomodate European sized paper roll.
50 cm long x 12 cm diameter (19.7 in. long x 4.7 in. diameter)

Restrictions:
Cannot be used with 9A313 Rear Storage Pod.

Step 1: Remove standard paper roll holder.
Note: Save screws & dowel rod for new holder.

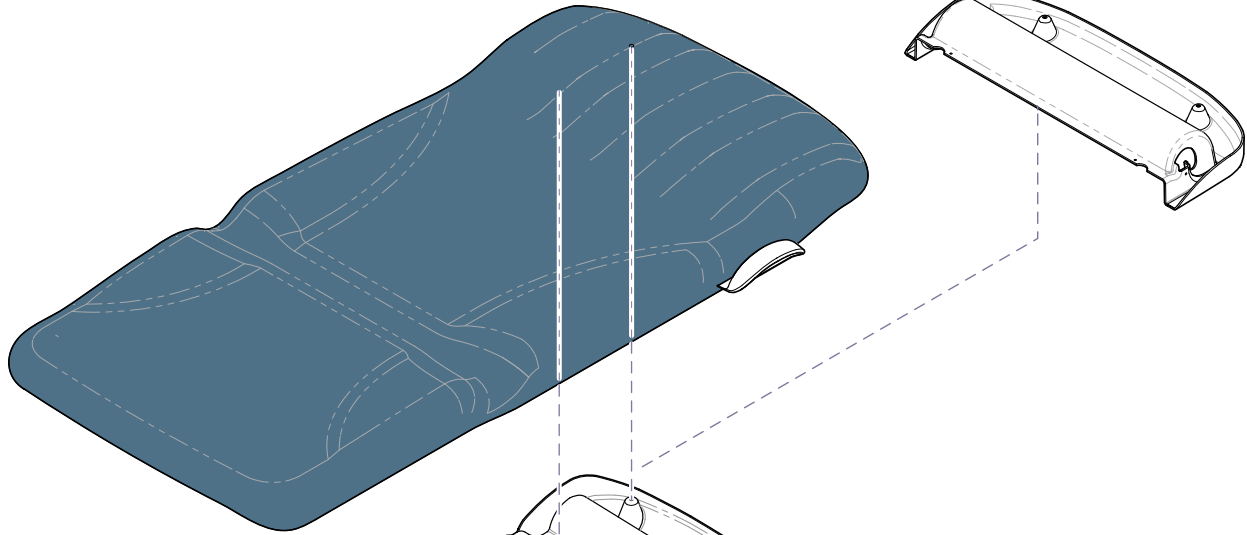

Step 2: Secure new paper roll holder with four screws.
Place dowel rod in position.

MA722100i