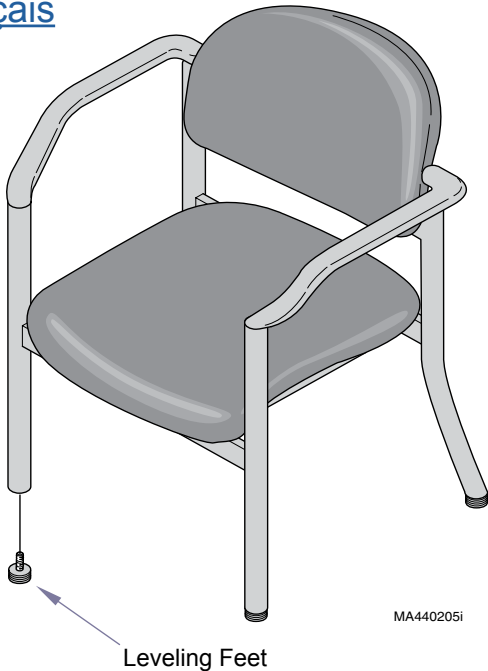
Side Chair Setup & Care

[English](#)

[Español](#)

[Français](#)

For Models:

680:

(-001) w/o Arms (Std. Uph.)

(-002) w/ Arms (Std. Uph.)

(-003) w/o Arms (Cal 133 Uph.)

(-004) w/ Arms (Cal 133 Uph.)

To level chair...

- A. Screw leveling feet into chair legs.
- B. Place chair on floor surface.
- C. Adjust leveling feet as necessary.

Cleaning

Upholstery

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on www.midmark.com in the Technical library under the User Information tab for your product.

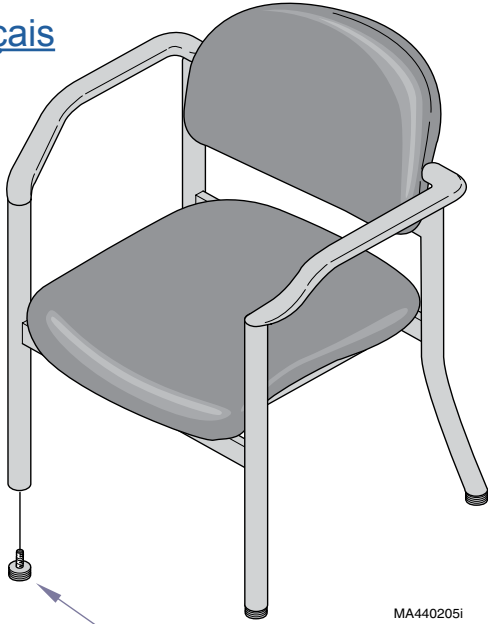
Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner. Periodic applications of common furniture wax will ease cleaning, and maintain the finished luster of the surfaces.

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Midmark Corporation