


**Pelvic Tilt feature (optional)**  
**To elevate seat section...**  
 Lift seat section until pelvic support drops into place.

**To return to flat position...**  
**A.** Lift seat section slightly.  
**B.** Pivot trigger handle backward.  
**C.** Lower seat section.

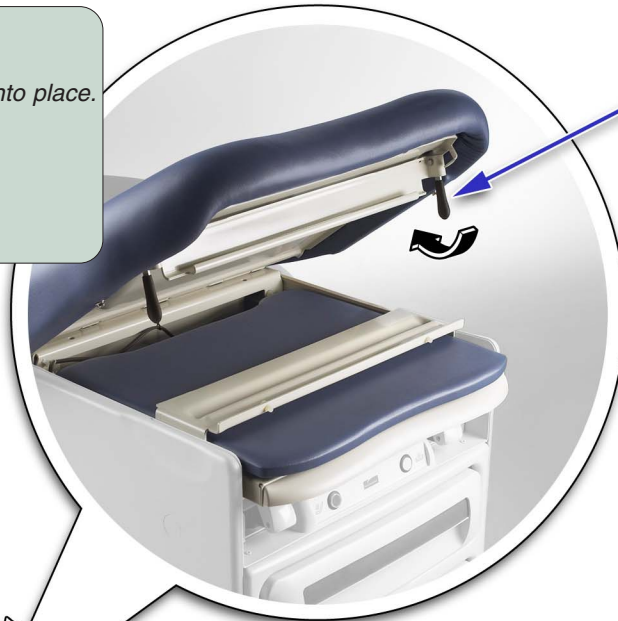

Trigger Handle

**To position the back section...**  
**A.** Squeeze the Back Release Handle (either side).  
**B.** Move the back section to desired position.  
**C.** Release handle to lock back section in position.


Back Release Handle



Stirrups



MA709002p

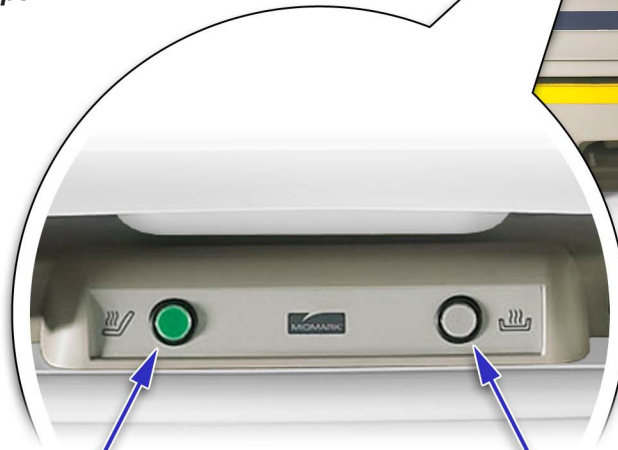

Upholstery Heater Switch  
(green)

Drawer Heater Switch  
(white)

**Exam Assistant® Drawer System**  
**To lower drawer front...**  
 Pull drawer front down to release magnetic latch.

**Equipment Alert**  
 The max. weight the shelf will support is 10 lbs (4.5 kg).


**If drawer front folds past horizontal...**  
 Pull drawer front upward until it snaps back into the horizontal position.

**Drawer Heater (optional)**  
**To turn the drawer heater ON / OFF...**  
 Press the drawer heater switch.

**Upholstery Heater (optional)**  
**To turn the upholstery heater ON / OFF...**  
 Press the upholstery heater switch.

**Table Receptacle (optional)**  
If the maximum load is exceeded, the circuit breakers remove power from the receptacle.  
Maximum Load: 5 amps

**To reset circuit breaker(s)...**  
Push circuit breaker toggle switch.

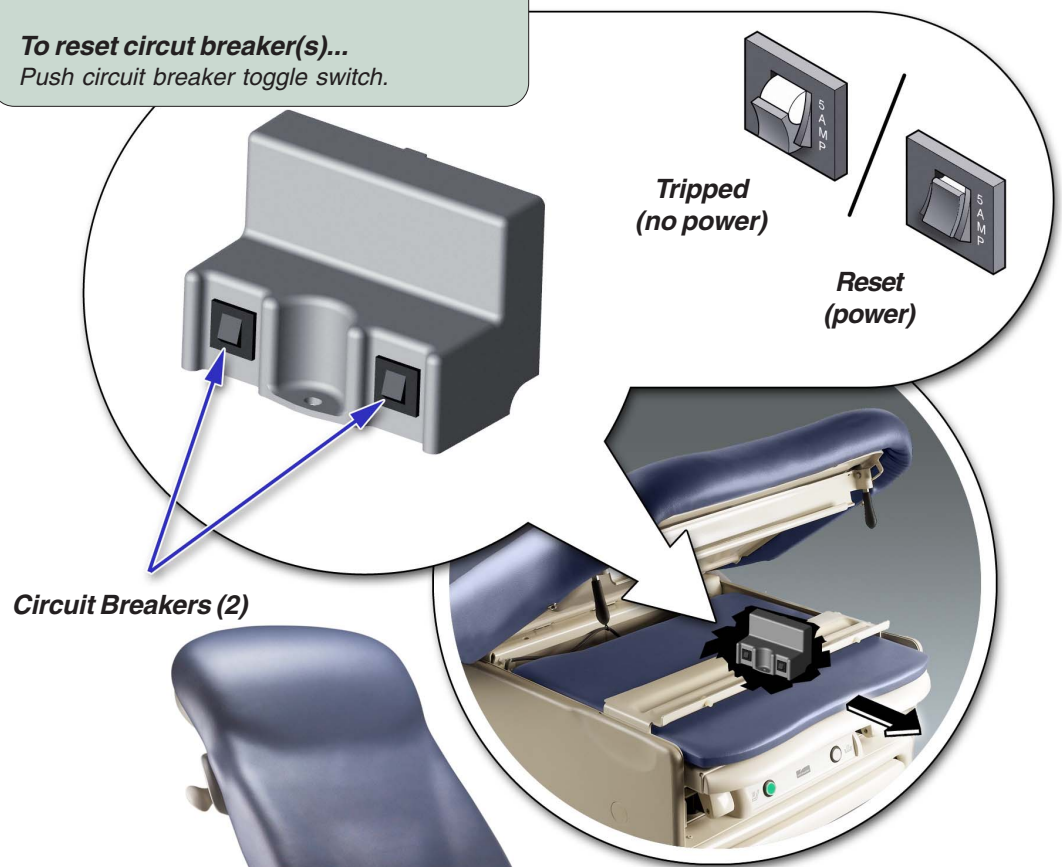

### Periodically inspect the following:

- Power cord must be free of visible damage.
- All fasteners must be present and secure.
- All mechanical functions should operate.

### Cleaning

- Clean upholstery with diluted bleach solution.  
Ratio: ..... 10:1 (water:bleach)
- Clean the table weekly, wiping the painted metal and plastic surfaces with a clean, soft cloth, and mild cleaner. Periodic applications of common furniture wax will ease cleaning and maintain the finished luster of the table.

### Calling for Service

If service is required, contact your Midmark dealer.

To contact Midmark directly:

1-800-Midmark (1-800-643-6275)  
8:00 am until 5:00 pm Mon-Fri (EST)



- **Français :** Référez-vous [www.midmark.com](http://www.midmark.com)
- **Deutsch:** Siehe [www.midmark.com](http://www.midmark.com)
- **Español:** Refiera a [www.midmark.com](http://www.midmark.com)

Midmark Corporation  
60 Vista Drive  
Versailles, OH 45380-0286  
Phone: 937-526-3662  
Fax: 937-526-5542  
[midmark.com](http://midmark.com)



<b>Model Number:</b>	<input type="text"/>
<b>Serial Number:</b>	<input type="text"/>
<b>Dealer Info:</b>	<input type="text"/>
<b>Purchase Date:</b>	<input type="text"/>
<b>Upholstery Color:</b>	<input type="text"/>

003-1617-00 Rev. E 8/31/15

This guide outlines basic operation of the table. The enclosed User Guide provides additional instructions on operation and care, as well as regulatory compliance information. If service is required, contact your authorized Midmark dealer.

