

**To operate the Base / Back / Tilt functions...**

Press & hold the appropriate button on the hand control.

**Note:** Movement automatically stops at the max./min. position.




**Base Cover**



**Attention!**

The table is equipped with safety switches to stop all movement if any pressure is applied to the base cover.

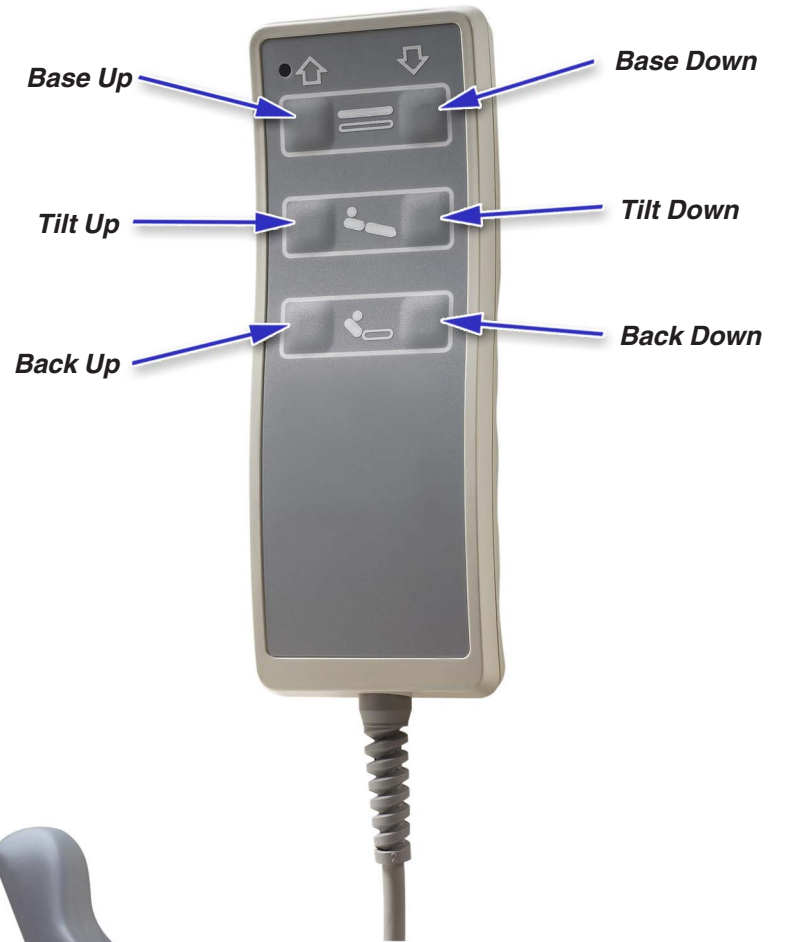

**Quick Reference Guide**  
**244 Barrier-Free™**  
**Bariatric Power Treatment Table**

**Model Number:**   
**Serial Number:**

**Dealer Info:**

**Purchase Date:**

**Upholstery Color:**

003-1641-00 Rev. D (7/10/15)

*This guide outlines basic operation of the table. The enclosed User Guide provides additional instructions on operation and care, as well as regulatory compliance information. If service is required, contact your authorized Midmark dealer.*



**Max. Patient Weight\* ..... 850 lbs (385 kg)**

\* If the optional casters are installed, the maximum patient weight is 650 lbs (295 kg).

**Periodically inspect the following:**

- Power cord must be free of visible damage.
- All fasteners must be present and secure.
- All mechanical functions should operate.

**Cleaning**

**Upholstery**

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on [www.midmark.com](http://www.midmark.com) in the Technical library under the User Information tab for your product.

**Painted Metal / Plastic Surfaces**

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner. Periodic applications of common furniture wax will ease cleaning, and maintain the finished luster of the surfaces.

**Calling for Service**


*If service is required, contact your Midmark dealer. To contact Midmark directly:  
1-800-Midmark (1-800-643-6275)  
8:00 am until 5:00 pm Mon-Fri (EST)*

Midmark Corporation  
60 Vista Drive  
Versailles, OH 45380-0286  
Phone: 937-526-3662  
Fax: 937-526-5542  
midmark.com



**Power Cord / Hand Control Cord Connection:**

Plug the cords into the connection ports as shown.

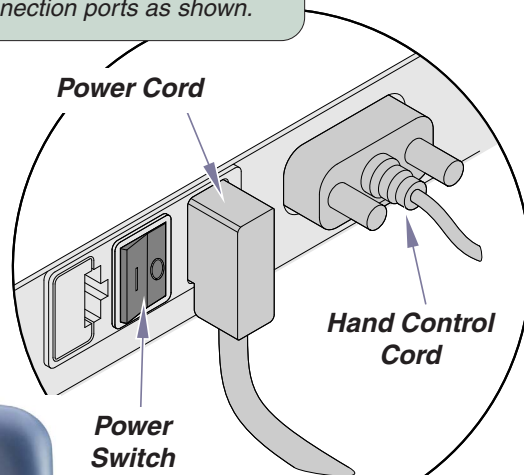

**To turn the table ON...**

Press power switch to ON (I).

**To turn the table OFF...**

Press power switch to OFF (O).



MA742100i

[Table shown with the Adjustable Arm System accessory installed]