

## Drawer Stop Installation [9A372]

**Components:**

Slide Clips ..... 3

**Special Tools:**

none

**Restriction**

Early models have a drawer hole cover in the bottom of the table.  
The drawer stop **cannot** be installed on the bottom slide of these models.

**Attention**

Slide clips must be installed on the RH cabinet slides.  
Each drawer requires one slide clip.


**Step 1:** Hook long "leg" of slide clip over sheet metal edge as shown.

**Step 2:** Align notches with cabinet slide.

Notches

**Step 3:** Press slide clip down until it snaps into place.

**Note:** Slide clip should rest flat on slide / cabinet.

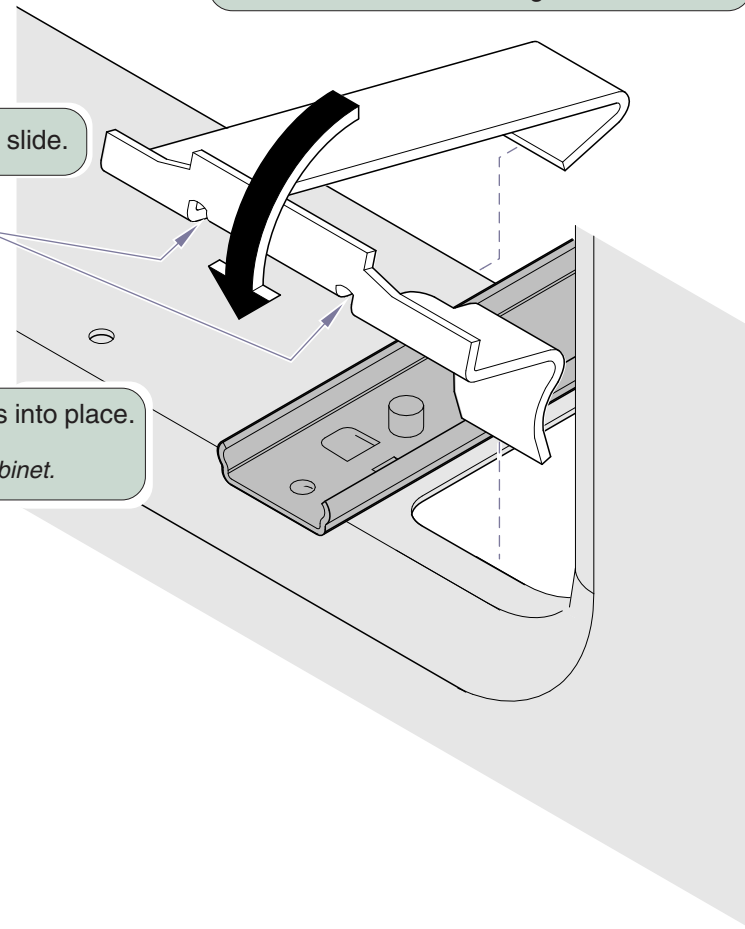

MA747500i