




# UPT Actuator Grounding Kit

- Components:**  
Ground Wire ..... 1  
Ground Screw ..... 1  
Star Washer ..... 1
- Special Tools:**  
none

**Models w/ Grounding Plate:**

To ensure proper ground, the metal braided insulation on the actuator wires must be in direct contact with the grounding plate.

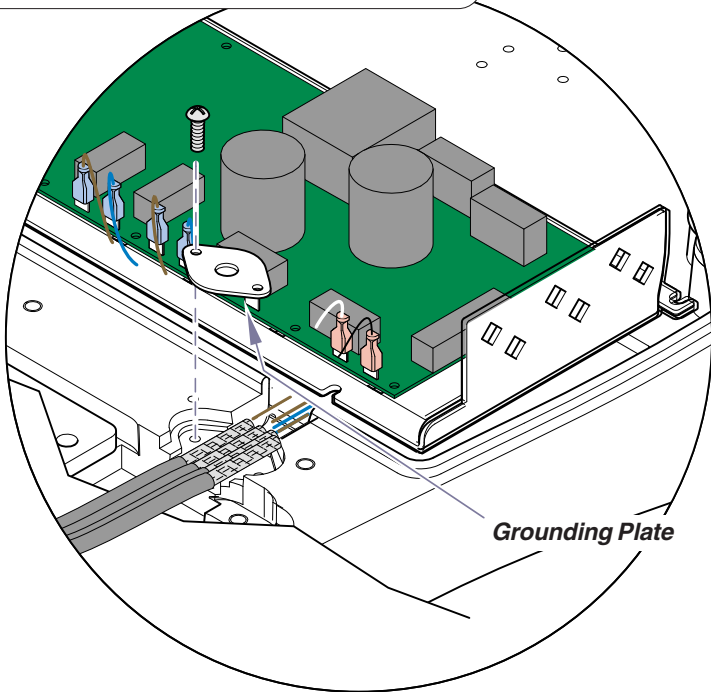

**Models w/o Grounding Plate:**

Separate the strands of the braided insulation on actuator wires.  
Twist the strands together.  
Connect the ground wire (provided) as shown.  
Connect ground wire to base casting w/screw (provided).

