

Canis Major® Exam Lift Table Setup Guide

Applies to Models:
 CMWO (without scale)
 CMWS (with scale)

Step 7: Raise and lower table top
 Note: Operating range should be 34" (86.4 cm).
 (9.5" to 43.5" (24.1 to 110.5 cm) from floor)

Caution - Equipment Alert
 Dropping unit off skid could result in equipment damage.
Step 4: Unplug unit. Remove table from skid. Move to final location.
 Note: Slide unit off rear of skid until legs are almost off, then lift front end of table top and slide skid clear of legs.

Step 1: Unfasten and remove rear retaining block.

Step 2: Remove [2] packaging straps and any remaining packaging material.

Step 5: Adjust front leveling feet as necessary to stabilize unit.

Step 8: Wipe down entire table using mild cleaner solution.

Step 6: Plug unit into power source. Plug foot control into table. Raise table top with foot control or hand switch.
 Note: A dedicated supply circuit is required for this table and needs to be 115 VAC, 60 Hz, 10 Amp.

Note: Plug unit into power source. 115 VAC, 60 Hz, 10 Amp.
Step 3: Lift table using hand switch, remove shipping brackets and DISCARD [2]. Brackets used for shipping purposes only.

