



# Sink / Tap Master Bottom Support Installation Instructions [002-1063-XX]

Components:	
Screws:	Hole Plug ..... 4
#8 - 32 x 1/2" ... 2	RH Bracket ..... 1
#10 - 24 x 3/8" . 4	LH Bracket ..... 1
Base Bottom ..... 1	Cross Brace ..... 1

  

Special Tools:	
Level	

**Step 1:** Remove base bottom.  
*Note: Keep screws to replace new base bottom.*

**Step 6:** Install hole plugs (4).

**Step 5:** Install base bottom with screws (2)

Tap Master Sink Bottoms ONLY  
**Step 4:** Position cross brace as shown. Secure with screws (2). Proceed to **Step 5** if cross brace does not apply.

**Step 2:** Position RH bracket in cabinet. Install screw in front hole. (*Do not tighten*) Repeat for LH bracket.

**Step 3:** Level RH bracket in cabinet. Secure back of bracket with self drilling screw. Repeat for LH bracket.

