



Door Seal Installation Sheet [002-1030-00]

Components:

Wingnuts	2
Tray	1
Gasket	1
Handle	1

Special Tools:
none

Caution:
Always disconnect power to sterilizer before removing cover. Allow sterilizer to cool then drain reservoir.

Step 1: Remove tray from M3 chamber.

Step 3: Remove gasket.

Step 2: Remove wing nuts.

Step 4: Inspect and clean mounting surface and place new gasket on handle assembly and secure by replacing wing nuts.

Step 5: To install gasket reverse steps.

